NUK®

Handmilchpumpe

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Manual Breast Pump

with NUK Perfect Match bottle

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MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

Contents

1. Important information before using for the first time	12
2. Pump description	13
2.1 Intended use	13
2.2 Description of function	13
2.3 Symbols on the pump/packaging	13
3. Safety information	13
3.1 Explanation of terms	13
3.2 General information	13
3.3 Hygiene	14
3.4 Notes for the Breast Milk Container and Teat	14
4. Key parts (Figure Q)	14
5. Taking the pump apart (Figures A to H)	14
6. Cleaning and sterilising	15
6.1 Cleaning	15
6.2 Sterilising	15
7. Reassembling the pump (Figures I to P)	16
8. Using the breast pump	16
8.1 Expressing milk	16
8.2 Feeding expressed milk	16
9. Storing milk	17
9.1 Using the Breast Milk Container	17
9.2 Important information for handling breast milk	17
10. Tips for successful expressing	18
10.1 Quantity of milk	18
10.2 Let-down reflex	18
11. Help with problems	19
12. Disposal	19
13. Warranty	19
13.1 Service Centres	20
13.2 Manufacturer	20

Dear Mother,

The NUK Manual Breast Pump is designed as an aid for you and your baby to store up breast milk.

Before using your NUK Breast Pump for the first time, please allow a few minutes of your time to carefully read these instructions for use.

1. Important information before using for the first time

Warning! Before using the NUK Breast Pump for the first time, clean and sterilise all parts (see “6. Cleaning and sterilising“ on page 15).

Please carefully read the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

2. Pump description

2.1 Intended use

The NUK Breast Pump is an aid to expressing and collecting breast milk.

Once expressed, milk can be directly collected in the NUK Breast Milk Container and then stored, frozen, warmed and fed to your baby.

The pump is suitable for all NUK bottles:

- NUK First Choice bottles are screwed directly onto the pump;
- NUK Perfect Match bottles, NUK Nature Sense bottles and NUK for Nature bottles are screwed onto the pump using the adapter provided.

2.2 Description of function

The NUK Breast Pump is a manual suction pump and offers the following benefits:

- simple to use and needs no power supply;
- breast pump comes apart for easy cleaning;

- silicone cushion for gentle massaging effect and softer feel;
- collects expressed breast milk.

After expressing, the pump can be securely placed in the stand provided together with the bottle.

To store the expressed breast milk, the bottle can be sealed securely with the sealing disc provided and the screw ring.

2.3 Symbols on the pump/ packaging

Symbol	Meaning
	Refer to the operating instructions
	Keep away from sunlight
	Batch identification
	Attention! Observe the safety instructions in the operating instructions

3. Safety information

3.1 Explanation of terms

The following keywords are used in these user instructions:

Warning!

Ignoring this warning could result in a serious risk to life and limb.

Caution!

Ignoring this warning can cause minor personal injury and / or property damage.

Note:

Points to be noted when handling the appliance.

3.2 General information

- Before using the pump, please read these instructions for use carefully. These instructions are part of the unit and must be kept to hand at all times.

- Use the NUK Breast Pump and accessories for their intended purpose only (see “2.1 Intended use”)

Warning! To avoid risk of choking, keep small parts out of the reach of children.

- We recommend removing ointments/creams containing grease or oil from the breast before expressing, as these may damage the surface of the silicone cushion.
- To guarantee that the pump will function correctly, only use original NUK accessories.
- Please note that any claim under warranty or liability will be void if accessories are used which are not recommended in these operating instructions, or if any parts other than original spare parts are used for repairs.

This also applies to repairs carried out by unqualified persons.

- In the event of any malfunctions, please contact the manufacturer.
- If the breast milk is for a premature baby, it is essential to follow your doctor's instructions.

3.3 Hygiene

Warning! When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.

- To prevent infection or bacterial contamination (microbial growth) and to prevent damage to the pump, please read and pay attention to "6. Cleaning and sterilising" on page 15.

- **Only a breast pump which has been thoroughly cleaned will achieve full suction strength.**
- Remove all body jewellery (e.g. piercings) before using the breast pump.

3.4 Notes for the Breast Milk Container and Teat

For your child's safety and health: WARNING!

Please carefully read the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

4. Key parts (Figure Q)

- | | | | |
|---|-------------------|----|--------------------------------|
| 1 | Silicone cushion | 9 | Lip valve |
| 2 | Breast shield | 10 | Adapter |
| 3 | Slot for pump rod | 11 | Breast milk container / bottle |
| 4 | Pump lever | 12 | Stand |
| 5 | Pump rod | 13 | Silicone sealing disc |
| 6 | Handle fixture | 14 | Protective cap |
| 7 | Pump diaphragm | 15 | Screw ring |
| 8 | Catch for handle | 16 | Feeding teat |

5. Taking the pump apart (Figures A to H)

1. Take the silicone cushion **1** out of the breast shield **2** (Fig. **A**).
2. Holding the end of the handle, pull the pump lever **4** straight up until the handle fixture **6** slips out of the catch for the handle **8** (Fig. **B**).
Attention! Don't pull/turn the pump lever to the side, because the pump rod may break.
3. Carefully remove the pump lever **4** from the pump rod **5** (Fig. **C**).
4. Take the pump diaphragm **7** and pump rod **5** out of the breast shield **2** (Fig. **D**).
5. Carefully ease the pump rod **5** out of the pump diaphragm **7** (Fig. **E**).
6. Unscrew the breast milk container **11** (Fig. **F**).
7. Unscrew the adapter **10** (Fig. **G**).
8. Pull the lip valve **9** out of the breast shield **2** (Fig. **H**).
9. Finally, clean the individual parts as described in "6. Cleaning and sterilising".

6. Cleaning and sterilising

Warning!

Hygiene is of the utmost priority when handling breast milk and all parts that come into contact with it. **The pump should therefore only be used in a clean and sterile condition.**

Clean, sterilise and quickly dry **all parts** before using for the first time and immediately **after each use.**

6.1 Cleaning

Caution! To avoid damaging the lip valve **9**, carefully clean it. We recommend just rinsing off the lip valve and boiling it to sterilise. Cleaning implements could cause damage.

1. Clean all parts with warm water and a little washing up liquid (or NUK Baby Bottle Cleanser).
2. Rinse off all parts in running water.

Warning! Milk residues may damage the material and result in microbial growth.

Note: When rinsing with dishes (for example), certain foodstuffs (e.g. carrot juice) may discolour the parts or affect their odour (e.g. fennel tea). This has no effect on the fitness for use or safety of the bottle.

3. Next, sterilise all components (see “6.2 Sterilising”).

6.2 Sterilising

Before sterilising, always clean as described in chapter “6.1 Cleaning” on page 15.

Warning! Risk of scalding during sterilisation.

Caution!

- Dismantle each part into its individual components for boiling or steam-sterilising (temperature above 80 °C) and leave open – otherwise deformation can result.

- Incorrect treatment can damage the parts.
- Before using again, always allow the sterilised parts to **cool down properly and dry completely** before reassembling.
- Do not use any chemical disinfectants for sterilising:

1. Use a NUK Steam Steriliser, following the instructions for use of the device;
- Or
2. Sterilise the parts in boiling water for 5 minutes.
Ensure there is sufficient water in the pan to avoid damage.
- Or
3. Use a NUK Micro Express Plus Microwave Steam Steriliser for sterilisation in a microwave oven.
Only use the NUK Micro Express Plus Microwave Steam Steriliser for microwave sterilisation. Follow the instructions for use of the device.

7. Reassembling the pump (Figures I to P)

1. Insert the pump rod **5** in the pump diaphragm **7**. The lower end of the pump rod **5** must fit in the hole of the pump diaphragm **7**. (Fig. **I**).
2. Position the pump diaphragm **7** in the breast shield **2** and press firmly in place (Fig. **J**).
3. Insert the head of the pump rod **5** in the pump lever **4** (Fig. **K**).
4. Engage the handle fixture **6** in the catch for the handle **8** (Fig. **L**).
5. Insert the lip valve **9** into the breast shield **2** (Fig. **M**).
6. Screw the adapter **10** onto the breast shield **2** (Fig. **N**).
Note: NUK First Choice bottles can be screwed directly onto the breast pump without the adapter.
7. Screw on the breast milk container **11** under the adapter **10** with the breast shield **2** (Fig. **O**).
8. Position the silicone cushion **1** in the front of the breast shield **2** and pull the edge of the cushion over the breast shield (Fig. **P**).

8. Using the breast pump

8.1 Expressing milk

1. Place the funnel of the breast shield over your breast.
 - Make sure that your nipple is in the centre of the silicone cushion **1**. This prevents the nipple becoming sore.
 - Gently press the breast shield **2** against the breast so that the silicone cushion **1** fits closely and no air is sucked in from the side.
 - **Always** hold the breast milk container with the adapter and breast pump upright.
2. Press the pump handle **4** then slowly release.

Note: The further you press the pump lever **4**, the greater the suction. To reduce the suction, do not press the pump lever **4** as far.

Caution!

- To achieve optimum results, express only enough milk into the container so that it does not come into contact with the bottom edge of lip valve **9**.
- When freezing breast milk, do not fill the container with more than 150 ml (5 fl oz),

as breast milk expands as it freezes and the container could burst.

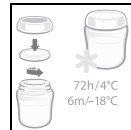
3. After expressing, place the breast pump in the stand **12**. The pump must not be allowed to topple over as milk might escape.
4. Unscrew the breast shield **2** and the adapter **10** from the breast milk container **11**.
5. Firmly seal the breast milk container **11** using the supplied screw ring **15** and silicone sealing disc **13**.
6. Immediately after expressing, wash the individual parts (see “6. Cleaning and sterilising“ on page 15).

8.2 Feeding expressed milk

Screw feeding teat **16** and screw ring **15** onto breast milk container **11**.

If your collected breast milk is not for immediate use, firmly seal the filled breast milk container **11** with the sealing disc **13** and screw ring **15** and store it in your refrigerator or freeze it (see “9. Storing milk“ on page 17).

9. Storing milk

9.1 Using the Breast Milk Container

Breast milk may be chilled, frozen and defrosted in the breast milk container. The breast milk container may also be used for warming and feeding. It is important that you follow the instructions about handling breast milk which is provided in “9.2 Important information for handling breast milk” on page 17.

Warning! For hygienic reasons:

If the breast milk is not used immediately, you must seal the filled breast milk container firmly and store it in the refrigerator or freeze it.

Using a suitable pen or label, write the date and the time when the breast milk was expressed. Be sure to note the retention periods in “9.2 Important information for handling breast milk” on page 17.

Caution! To avoid any damage: When freezing breast milk, do not fill the container with more than 150 ml (5 fl.oz), because breast milk expands as it freezes and the container might burst.

For a spacesaving solution you can use NUK Breast Milk Bags for freezing and storage of breast milk.

9.2 Important information for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 3-4 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, defrost any frozen breast milk in the container by immersing it in lukewarm water (max. 37°C), then feed immediately.	

Warning! To ensure adequate hygiene at all times:

If transporting chilled breast milk make sure that it stays chilled at all times, otherwise the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.

Do not use a microwave to defrost breast milk, as this can destroy valuable nutrients and damage the container (risk of explosion). Uneven heating may result in scalding!

Defrosted, warmed or unused milk must be disposed of and not refrozen.

1. To achieve the correct drinking temperature of approx. 37 °C, put the

container of defrosted breast milk in the baby food warmer.

Caution! The container must be open to prevent a pressure build-up as the milk warms up.

2. Shake the milk gently before feeding so that milk and milk fat are properly mixed again.
3. If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk. When doing so, the temperature must be the same (cold on cold). Freshly expressed milk must therefore be cooled down before adding it to milk that is already cool.

10. Tips for successful expressing

Correct expressing takes practice. You will first need to get used to your new pump.

10.1 Quantity of milk

Milk production adapts itself to daily requirements.

This means that the quantity of milk produced may vary considerably.

- Always make sure that you drink plenty of liquids.
- Milk should be expressed from each breast in turn.

10.2 Let-down reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down reflex and milk begins to flow. You can also stimulate the let-down reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when using the breast pump.
- Adopt a comfortable and relaxed sitting position.
- The let-down reflex triggers milk flow in both breasts simultaneously. You can use the NUK Breast Shell Set to collect breast milk leaking from the breast you are not currently expressing.
- Try to maintain the suction pressure at the pressure level that is comfortable for you. This will keep the milk ducts in the breast open, allowing the milk to flow freely from the breast.

11. Help with problems

Problem	Reason	Remedy
Pump has no suction or too little suction	The pump rod 5 is not seated properly in the pump diaphragm 7	Make sure that the pump rod 5 is fully inserted in the hole in the pump diaphragm 7 (see “7. Reassembling the pump (Figures I to P)” on page 16).
	Pump diaphragm 7 missing, fitted incorrectly or damaged	Insert pump diaphragm 7 correctly (see “7. Reassembling the pump (Figures I to P)” on page 16). If anything is damaged, please contact our Service (see “13.1 Service Centres” on page 20).
	Lip valve 9 is incorrectly positioned	Position lip valve 9 correctly (see “7. Reassembling the pump (Figures I to P)” on page 16).
	Lip valve 9 missing, fitted incorrectly or damaged	Check that the lip valve 9 is correctly positioned. If anything is damaged, please contact our Service (see “13.1 Service Centres” on page 20).
	Breast milk container 11 is full and lip valve 9 is touching the expressed milk	Change the breast milk container 11 .
	No vacuum is produced	Make sure that the breast shield 2 forms a complete seal around the breast.
		Make sure that the silicone cushion 1 is inserted correctly and is pulled over the edges of the breast shield.
	Pump is being held at an angle	Hold the pump as upright as possible.

12. Disposal

If at some stage the appliance is no longer fit for use, the parts can be disposed of along with normal household waste.

13. Warranty

Subject to careful treatment of your appliance and observance of these instructions for use, the manufacturer offers a 24 month warranty from the date you purchase the product.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the device.

Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

Under all circumstances, the device must be returned accompanied by this completed warranty certificate or, failing this, the purchase receipt or other proof of the purchase date must be supplied.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty form immediately after purchase and keep this, or the till receipt, in a safe place as proof of purchase. Lost warranty certificates will not be replaced.

Please keep the address, Art. No. and LOT No. for future reference.

13.1 Service Centres

The addresses for service can be found under "Contact addresses" on page 114.

Article no.: 10.252.147 / 10.252.148

13.2 Manufacturer

Please note that the address below **is not a service address**. In the event of problems or queries relating to your product, contact one of the Service Centres (see "Contact addresses" on page 114).

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Artemis Brands (PTY) LTD