935 PERLWEISS™ Dental Bleaching (4.0)

24.01.2024

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24.01.2024

MANUAL

FRONT	PERLWEISS™ logo Dental Bleaching Professional tooth whitening <i>Illustrations of syringe, tube and mouth trays</i> Bleaching system with mouth trays for a 7-day home use Manual
INSIDE LEFT	Pictogram brightened tooth in box -> before / after. PERLWEISS™ 3-phase tooth whitening gel For best results, please read instructions carefully before starting treatment. Application 1. Shaping the mouth trays: immerse it in hot, <u>non-boiling</u> water for approx. 3 seconds, shake it off and place it over the teeth. Press the splint with tongue and fingers against the teeth. At the same time, suck air away to achieve the best possible fit. If this does not work, repeat the procedure with the same splint. Only rinse the splint in cold water and allow it to harden when the best possible fit has been achieved. Then cut off the grip lip. 2. Before each treatment, brush teeth lightly without toothpaste. 3. Application of the tooth whitening gel: Carefully apply a thin strip of tooth whitening gel from the syringe from one side to the other directly into the front edge of the splint so that the front side of the teeth is covered. For application on both rows of teeth, one unit of measurement - approx. 1,5 ml - is sufficient. 4. Place the mouth tray on the row of teeth to be treated. Both rows of teeth can be whitened simultaneously or one after the other. Leave on for 15 minutes! Then remove the mouth tray and rinse the oral cavity with water. Brush the teeth lightly without toothpaste. 5. Application of Repair & Protect Gel: For longlasting results, apply a small amount of Repair & Protect Gel to the appropriate row of teeth using the enclosed applicator or clean fingertip after the bleaching treatment. It is not necessary to rinse the gel. Recommendation: For best results, please do not eat food or discoloring drinks (e.g coffee, tea, wine) for about 1-2 hours after the bleaching treatment and the application of the Repair & Protect Gel. 6. Repeat treatment once a day for 7 consecutive days or until satisfied (maximum treatment: 14 days). The content of the syringe is sufficient for 7 treatments of both rows of teeth.

INSIDE RIGHT	For a successful result, please note
	<ul style="list-style-type: none"> • The splints do not have to fit perfectly. Their sole purpose is to keep the gel on the teeth during the treatment so that it is not prematurely detached by tongue or saliva. Salivation during treatment is normal and important for the gel to be fully effective. We recommend that you perform the whitening treatment in the evening before going to bed so that food and drinks will not affect the result.
	<ul style="list-style-type: none"> • The number of treatments required to achieve the desired result is individual. The result depends on the discoloration and the genetic conditions. Small white spots may appear immediately after treatment. These usually dissolve after 4-6 hours. Repeat the treatment until the desired result is reached, but for a maximum of 14 consecutive days, then pause for 3 months. For optimal and lasting results, daily dental care with the PERLWEISS™ Schönheits- or Raucher Zahnweiss or Expert Weiss is recommended.
	<ul style="list-style-type: none"> • Avoid nicotine, fruit juices, tea, coffee and other colored drinks for up to two hours after treatment.
	Storage
	Keep all teeth whitening products and accessories out of reach of children and animals.
	Side effects
	There are no known side effects.
BACK	PERLWEISS™ Repair & Protect Gel
	Repairing protective gel for the teeth. The Repair & Protect gel prolongs the degree of whiteness achieved by bleaching. It remineralises the enamel, dissolves dirt and reduces harmful bacteria and plaque. Thanks to its innovative formula, the gel gives your teeth a fantastic shine. This makes it an ideal protection after bleaching. For a feeling like after a visit to the dentist - fresh and clean. Perlweiss™ Repair & Protect gel can also be used daily after the 7-day treatment. Whenever you want to show your most beautiful smile.
	Safety instructions
	For adult use of 18 years and older only. Keep out of the reach of children. Avoid eye contact. Do not use on: Gum disease, periodontal disease, bleeding gums, gingivitis or receding gums, sensitive teeth, pregnancy or lactation. Do not use these products when wearing braces. Use for more than 14 consecutive days only after consultation with your doctor. Do not wear the malleable splint longer than recommended. Stop use immediately if irritation or allergic reactions occur. Do not swallow. If you have accidentally swallowed a large amount, consult a doctor. Do not wear the malleable splint longer than recommended. Teeth whitening does not work on crowns, fillings or veneers. Store products in a cool and dry place.
	www.perlweiss.de
	Facebook- icon
	Instagram icon
	Murnauer Markenvertrieb GmbH, D-63324 Egelsbach
	MADE IN SWEDEN
	<i>Illustration of VITA scale</i>