WHITENING STRIPS

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WHITENING STRIPS

14 x 2 Professional Whitening Strips
1x Whitening Pen

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Aqua, Glycerin, Aloe Barbadensis Leaf Juice, Sodium Phytate, PEG-40 Hydrogenated Castor Oil, Xylitol, Cellulose Gum, Sorbitol, Citric Acid, Sodium Hypochlorite, Mentha Piperita Oil, Menthol, Vaccinium Macrocarpon Seed Oil, Citrus Paradisi Peel Oil, Punica Granatum Seed Oil, Limonene, Rosmarinus Officinalis Leaf Extract

facebook.com/smilepen
instagram.com/smilepens

www.smilepen.ch
info@smilepen.ch

SwissWhite GmbH
Blegistrasse 11A
CH-6340 Baar/ZG – Switzerland

Kühl und Trocken aufbewahren
Store in a cool and dry place
Conserver dans un endroit frais et sec
Conservare in luogo fresco e asciutto
Koel en droog bewaren

Made in P.R.C.

14 x 2 Strips / 1 x 5ml Pen

Our product contains 14 x 2 Whitening Strips sets & 1 x Whitening Pen for professional teeth whitening. They are ideally suited for teeth with a slight discolouration or as a follow-up treatment for our Whitening Kits.

Application Whitening Strips

1. Thoroughly clean your teeth roughly half an hour before use.
2. Open the foil package at the marked point and remove the whitening strips.
3. Remove the strip from the base and stick the larger one onto your upper jaw. The adhesive strip should be centred on your gums. Use your fingers to apply. Fold the protruding end of the strip behind your teeth.
4. Use the smaller whitening strip for your lower jaw.
5. To achieve results that resemble professional teeth whitening, we recommend wearing the strips for one week, 1–2 hours daily. You do not have to worry about the long exposure, the bleaching process is gentle and does not hurt your gums or teeth.
6. After wearing, you can easily peel off the bleaching strips and clean your teeth briefly.

Application Whitening Pen

1. Clean your teeth with a toothbrush.
2. Now press the end of the Whitening Pen until bleaching gel comes out.
3. Evenly apply the gel to all teeth. Make sure the gel does not touch your gums.
4. Close your mouth and wait 10 to 30 minutes. During this time you can continue with your morning or evening routine.
5. Now rinse your mouth to remove the gel residue. Done.

Warning

- Avoid contact with eyes
- Rinse eyes immediately if the product has come into contact with eyes.
- Consult your dentist before using for the first time.
- Read internal users and safety information carefully before use.
- This product is not intended for use by persons under the age of 18.
- Use for a maximum of 7 consecutive days and not more than once a day

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