
<u>Languages:</u>	<u>Page:</u>
ENGLISH	2
FRANCE	11
ITALIAN	20
SPANISH	29
DEUTSCH	38
SUOMI	48
LIETUVIŠKAI	56
ČESKÝ	65
LATVIEŠU	73
EESTLANE	81
ქართული	89
УКРАЇНСЬКА	100
ҚАЗАҚША	109
БЕЛАРУСКАЯ	118
РУССКИЙ	127

ELECTRIC BREAST PUMP XN/MD-2255A

USER GUIDE

The Baboo™ electric breast pump package includes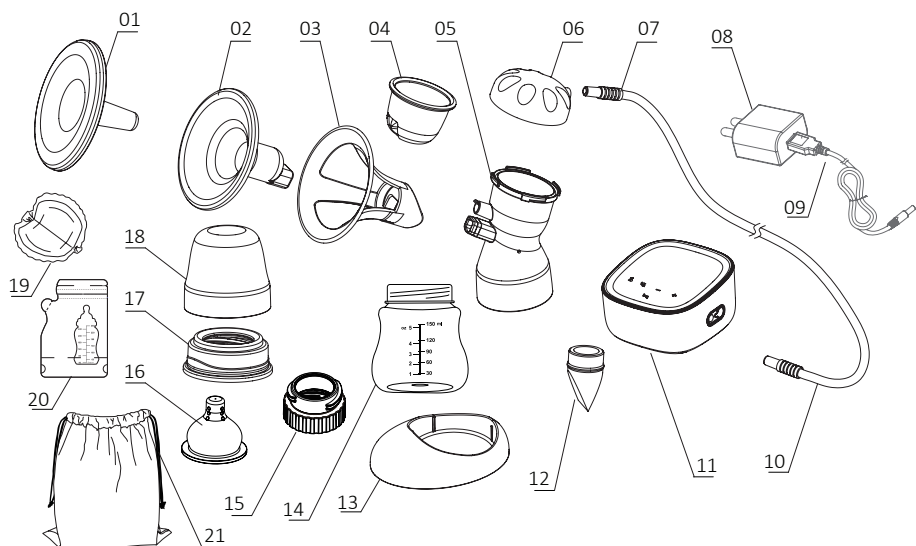

ENGLISH

BABOO™ ELECTRIC BREAST PUMP

Model NO.: XN/MD-2255A

USER GUIDE

1. Introduction	3
2. Intended use	3
3. Important safety information	3
4. Description	4
5. Get ready for the electric breast pump	4
6. How to use the electric breast pump	5
7. Breast milk storing conditions	7
8. Cleaning	8
9. Troubleshooting	9
10. Technical details	9
11. Guarantee and support	10

Introduction

It is time for you to relax and let your new Baboo™ electric breast pump do a job for you! Everyone knows that a mother's breast milk is the best natural food for a baby; therefore, we collaborated with the breastfeeding experts to make the beginning of your parenthood a little bit easier. Applying the knowledge and consideration of breastfeeding mothers who want nothing but the best for their babies, just like you, Baboo decided to help and fulfil their needs with this fantastic Baboo™ electric breast pump which helps to feed your baby even when facing some immediate circumstances which suggest that it wouldn't be possible.

By choosing the Baboo™ electric breast pump, you get quite a few unique features, such as:

- 3D™ soft flange to stimulate the baby's natural sucking
- Double functionality – works with the portable battery and can be plugged into any electrical household socket
- Easy to assemble and clean
- Two modes– stimulation and expression
- 9 suction levels
- Built-in rechargeable Li-ion battery
- Easy one-way milking flow system
- Modern design with touch screen system
- Soft milking termination when you are done using it
- All parts are BPA free

*Before using our product, please read this manual carefully and save it for future reference.

Intended use

Baboo™ electric breast pump is made to be used by women to express milk from the breast.

The electric breast pump may be used to relieve engorgement, a painful condition whereby the breasts are overfull, possibly preventing a proper latch by an infant. Baboo™ electric breast pump can be used anytime and anywhere, especially when circumstances do not allow a mother to breastfeed naturally.

Baboo™ electric breast pump is made only for a single user.

Important safety information

Failure to follow safety instructions can damage the device and become hazardous to your health.

- When you finish using the device, please unplug the Baboo™ electric breast pump from the adapter.
- Do not leave the Baboo™ electric breast pump unattended when plugged into a power source.
- Keep the device dry, and do not try to immerse it in water.
- Do not reach for an electrical device if it has fallen into the water. Unplug immediately.
- The authorised service agencies must conduct repairs.
- Do not rinse the membrane lid with water. If water gets into the lid, let it get dry before use.
- Before using the Baboo™ electric breast pump, ensure the connection hose is clean and completely dry.
- None of the components can be sterilised in a microwave or any other sterilisation device.
- Based on the battery characteristics, it is expected that the battery life will shorten after the usage of half a year. If the adapter or pump unit is damaged, please send it back to the purchasing store or the authorised professional for repair. There are no user-serviceable parts inside the motor unit and adapter of the Baboo™ electric breast pump. Do not attempt to open the device or repair the electric breast pump by yourself. If you do, your guarantee becomes invalid.
- If symbols E1 or E2 appear on the touch screen, it means there is an open or short circuit in the internal battery; please get in touch with your dealer or the store where you bought the Baboo™ electric breast pump.
- The Li-ion battery is built-in. DO NOT remove the battery by yourself.
- If you are not using the Baboo™ electric breast pump for a significant amount of time, please turn off the Li-ion power. If this is the case, we encourage you to charge the battery at least once per month. The charging time should not exceed 6

hours to extend its service life.

- Do not use other adapters and batteries that do not come with the Baboo™ electric breast pump package.
- If the E3 symbol appears on display, it means that the internal temperature of the Baboo™ electric breast pump unit is above or below the optional working temperature (from 5° to 60°).
- Please position the device in the optimal working space under optimal working temperature.

*We reserve the right to any changes to the product design.

Warning

- This is a single-user product. If used by more than one customer, it may present health risks.
- Supervision is required when the Baboo™ electric breast pump is used around children.
- Use the Baboo™ device only for its intended use, as described in this manual.
- Do not attempt to remove the working electric breast pump from the breast. Firstly, turn off the device, place your finger next to the breast to break the vacuum between your breast and the pump funnel and only then remove the electric breast pump from your breast.
- Never use the electric breast pump while pregnant, as pumping can induce labour.
- Do not use the electric breast pump while sleeping or when you are overly tired.
- Do not continue pumping for more than five minutes at a time if you fail to express any milk.
- If the pressure generated is uncomfortable or causes pain, switch off the appliance, break the vacuum between the breast and the pump with your finger and remove the breast pump from your breast.
- Do not operate the Baboo™ electric breast pump if it is physically damaged or misused.
- Do not place the Baboo™ electric breast pump near appliances generating a robust magnetic field.
- Do not expose the Baboo™ electric breast pump to extreme heat and do not place it in direct sunlight

Description

The Baboo™ electric breast pump package includes:

- | | |
|--|--------------------------------------|
| 1. Dust cover | 12. Duckbill valve* |
| 2. Silicone cushion | 13. Bottle stand |
| 3. Plastic brace | 14. Wide-neck bottle |
| 4. Membrane | 15. Bottle converter** |
| 5. Bottle connector | 16. Teat |
| 6. Membrane lid | 17. Disc collar |
| 7. Hoses connector | 18. Dust cover |
| 8. AC adapter | 19. Breast pads 4in*** |
| 9. Adapter cable | 20. Breast milk storage bags 3in1*** |
| 10. Hoses | 21. Carry bag |
| 11. Pumping unit with a lithium battery included | |

* The breast pump package may include a spare duckbill valve.

** Only use a bottle converter (15) for Baboo™ wide neck bottles.

*** Optional.

Because of constant improvements, there is a possibility of slight differences in the package content, including the breast pump and the ones shown on the package. Please check the content of the package before taking a decision.

Get ready for the electric breast pump

Preparing the Baboo™ electric breast pump

The Baboo™ electric breast pump comes pre-assembled. However, you need to re-assemble it after each cleaning session. Before assembling, the parts which come in contact with the breast milk directly, such as the milk bottle, silicone cushion, breast milk bag, duckbill valve, disc collar, nipple, and bottle converter (if used Baboo Wide Neck Feeding Bottle), must be cleaned and sterilised before each use.

Assembling the Baboo™ electric breast pump

Make sure you have cleaned and sterilised the appropriate parts of the electric breast pump mentioned before. Wash your hands thoroughly before you assemble the electric breast pump. Be careful, some parts of the electric breast pump may still be hot after sterilisation.

Assembling milk collection kit


1. Assembling of cushion/bottle connector

Put the silicone cushion into the plastic brace (3) (to stimulate milk excretion (2)). Connect tightly with the breast pump part connected to the bottle (5). Always put the dust cover (1) if you are not using the breast pump.

2. Assembling of the membrane

Put the membrane (4) into the breast pump connector (5), cover the membrane with a lid (6) and spin it tightly. Then connect to the hoses connector (7).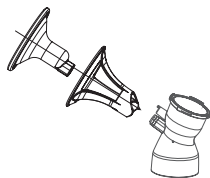

3. Assembling the duckbill valve

Push the duckbill valve (12) into the breast pump side, where the bottle connects. Push it tightly to avoid air leakage.

4. Assembling of the bottle


Spin the bottle (14) to the breast pump connector (5) and put it onto the bottle stand (13).

The breast pump kit for collecting milk connection to the electric breast pump side


1. Pump unit and sucking device

The vacuum hoses have two connector ends. Connect one end to the air vent of the membrane lid (6) and the second one next to the electric pump side (11). Make sure they are firmly fixed.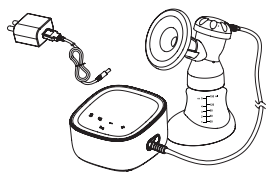

2. Breast pump and AC adapter

With the pump use adapter, this is added to the kit. The adapter reference code is indicated on the bottom of the appliance. Connect the AC adapter (8) to the electric breast pump side (11), then plug the adapter into an electrical socket. At that same instant, the pump LED display flashes, which means it is in standby mode.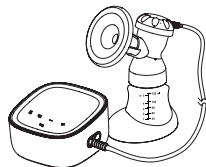

Attention


- The Anti-dust plug should be removed first before inserting the hose connector into the airway hole of the pump unit.
- The Anti-dust plug must be removed first before turning on the pump unit. Otherwise, it will affect the service life of the pump unit.
- After finishing pumping, insert the Anti-dust plug back into the airway hole of the pump unit to prevent any dust.

How to use the Baboo™ electric breast pump

How to use the electric breast pump

- Before each use, ensure that the Baboo™ electric breast pump side that collects milk is clean and completely dry.
- Clean your breast and massage your breast and areola. Relax yourself to relieve possible pain resulting from engorged milk.

Control panel description

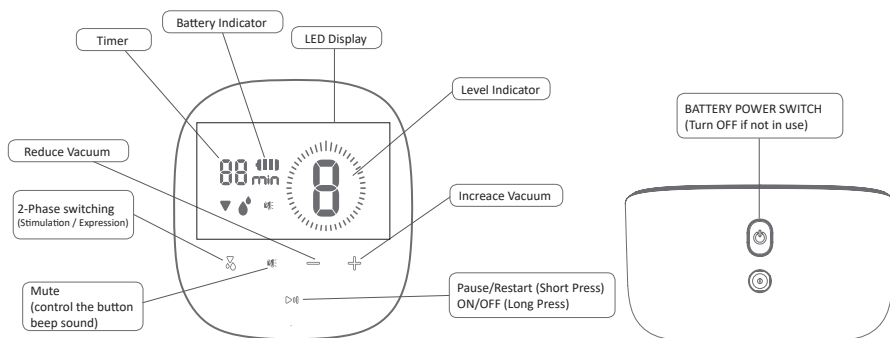LED Display

1. Fully charge the Baboo™ electric breast pump, then press the battery power switch  at the top of the electric breast pump unit to enter standby mode.
2. Long press the button  2 seconds to start or finish pumping.
3. Under standby mode, long press  to enter stimulation mode level 1. Under working status, short press  to pause working. The timer  will stop counting and blinking. Short press  again to restart working.
4. Under working status, long press  to stop working and enter standby mode.
5. Under stimulation or expression mode, press +/- to adjust the suction level. There are 9 adjustment levels for each mode.
6. Under stimulation or expression mode, press  to switch stimulation to expression or expression to stimulation mode. The Baboo™ electric breast pump will remember the last level during the mode switching.
7. The  will show the stimulation or expression mode level.
8. Under stimulation mode, LED shows , and under expression mode, LED shows .
9. Under working status, press  and the LED screen indicates . There is no beep until you press  again. Then  on the LED screen will disappear.
10. When the Baboo™ electric breast pump is in use through the Li-ion battery power, the battery power indicator  will provide a real-time display of your battery status; when the pump is in use through the A/C adaptor and is charging, the battery power indicator  will provide an actual status of the charging and the battery power level.
11. When the battery power runs out, the indicator  will flash. The buzzer will beep every 2 seconds, and the LED will go off after 10 seconds.
12. Ensure you charge the Li-ion battery before the first use. It can be used around 4–6 cycles\ (30mins per cycle) after fully charging. Usually, it takes 4–5 hours to charge the battery when the power has run out entirely.
13. The timer counts will start at 00 min. After 30mins, the electric breast pump will stop working automatically. The timer indicates 00 min again, and the stimulation and expression will return to level 1, and then the pump will enter standby mode.

When to start expressing milk

If you are breastfeeding, it is advisable (unless advised otherwise by your healthcare professional / breastfeeding advisor) to wait until your milk supply and breastfeeding schedule are established (generally at least 2 to 4 weeks after giving birth).

Exceptions:

- If breastfeeding does not immediately work for you, a regular expression can help establish and maintain your milk supply.

- If you are expressing milk for your baby to be given in a hospital.
- If your breasts are engorged (painful or swollen): you can express a small amount of milk before or between feeds to release the pain and help your baby latch on easier.
- If you have sore or cracked nipples, you may wish to express your milk until they are healed.
- If you are separated from your baby and want to continue breastfeeding when being together, you should express your milk regularly to stimulate your milk supply.

You need to find the optimum times of the day to express your milk, for example, just before or after your baby's first feed in the morning when your breasts are full or after feeding, if your baby has not emptied both breasts. If you have returned to work, you may need to express yourself during a break.

Using Baboo™ electric breast pump requires practice, and it may take several attempts before you succeed effortlessly. Fortunately, the Baboo™ electric breast pump is easy to assemble and use. Therefore shortly, you will be able to spend more time with family and baby and have some time for yourself while Baboo™ will do a job for you.

LED Display

1. Wash your hands thoroughly and make sure your breasts are clean.
2. Relax in a comfortable position. Make sure you have a glass of water nearby.
3. Press the assembled Baboo™ electric breast pump against your breast. Ensure your nipple is centred so the massage cushion seals airtightly.
4. Press the on/off button. Choose stimulation or expression mode and press +/- to adjust the suction level.
5. Once your milk starts flowing, you can adopt a slower rhythm by pressing the lowest suction level. Note: do not worry if your milk does not flow immediately. Relax and continue pumping. The first few times you use the breast pump, you may need to use a higher suction mode to get your milk flowing.
6. Depending on your comfort, you may wish to use a higher suction level that provides deeper suction. You can return to a lower suction level or switch off the electric breast pump whenever you like by pressing the appropriate button. Note: you do not need to use all suction modes, only the comfortable ones.
Always switch off the Baboo™ electric breast pump before removing the electric breast pump body from your breast. Do not continue pumping for more than 5 minutes if you fail to express any milk. Try to express the milk at another time during the day. Stop using the pump and consult your breastfeeding advisor if the process becomes uncomfortable or painful.
On average, you must pump for 10 minutes to express 60-125ml of breast milk from one breast. However, this is just an indication and varies from woman to woman.
7. When you have finished expressing, switch off the Baboo™ electric breast pump and carefully remove the electric breast pump body from your breast.
8. Unscrew the bottle from the electric breast pump body. The milk inside the bottle is ready for feeding/storing. Clean the rest used parts of the breast pump according to the instructions.
9. Unplug the device unit.

Breast milk storing conditions

Breast milk storage and thaw

Only store the breast milk that has been collected with a sterile breast pump. You should store and thaw your breast milk correctly. This is the only way to ensure that all the essential nutrients are preserved and that bacterial contamination is minimised.

Breast milk can be stored in a feeding bottle with a sterilised teat, disc collar and dust cover or in a Baboo™ breast milk storage bag.

Guidelines for storing / thawing breast milk

We recommend using the following guidelines for storing and thawing breast milk.

Storing conditions	Freshly expressed breast milk	Thawed breast milk (previously frozen)
Room temperature	4 hours at below 25°C	Do not store
Refrigerator	5 days at approx. 4°C	10 hours
Refrigerator freezer compartment	2 weeks at approx. -15°C	Do not refreeze after thawing!
Freezer	3-6 months at approx. -18°C	Do not refreeze after thawing!

Tips for freezing / thawing breast milk

- Never fill bottles / bags more than full to allow space for possible expansion.
- Label the bottles / bags with the date you expressed the milk.
- Do not store breast milk in the refrigerator door. Store it in the lower back area of the refrigerator, where the temperature is coolest.
- Thaw frozen breast milk in the refrigerator overnight or hold the bottle or bag under running warm water (max.37°C).
- Shake the bottle or bag gently to blend any fat that has separated.
- Do not thaw frozen breast milk in a microwave oven or boiling water pan (danger of burning)

Do's

- Always refrigerate or freeze expressed milk immediately.
- Only store milk collected with a sterilised breast pump in sterilised bottles.

Don'ts

- Never refreeze thawed breast milk.
- Never add fresh breast milk to frozen breast milk.

Feeding

Always check the temperature of the milk before you feed it to your baby.

Always discard any breast milk left at the end of a feed.

Never heat breast milk in a microwave, as this may cause uneven heating, leading to hot spots in the milk. It may also destroy nutrients in the milk.

Cleaning

Before using the Baboo™ electric breast pump for the first time

1. Disassemble the parts that come in contact with the breast and the breast milk.
2. Wash them in warm soapy water using any fragrance-free baby liquid dish detergent.
3. Rinse each piece thoroughly with warm water for 10 to 15 seconds.
4. You may sanitise the electric breast pump parts (except the adapter, vacuum hose, and electric part of the pump) through a sterilisation method of your choice, allowed by this instruction. But not more than for 3 minutes.
5. Do not use antibacterial or abrasive cleaning agents when cleaning the electric breast pump parts.

How to clean the Baboo™ electric breast pump

1. To prevent bacteria growth rinse each part that comes in contact with the body and milk with cool water immediately after pumping. This avoids the build-up of milk residue in the collecting bottles and parts.
2. Parts that come in contact with the mother's breast and breast milk are the bottle, silicone cushion, membrane, duckbill valve, disc collar, teat, and bottle converter (if using Baboo™ Wide Neck Feeding Bottle). Wash them in warm soapy water using any fragrance-free baby liquid dish detergent.
3. Use a separate cleaning sponge and bottle brush with a smaller brush to reach hardly reachable parts of the pump. Wash the parts separately from adult dishes, pots and pans, etc.
4. Rinse each piece thoroughly with warm water for 10 to 15 seconds. You may sanitise the parts through a sterilisation method of your choice each use, but not more than 3 minutes. Excessive and frequent boiling or exposure to heat may cause wear and tear to the plastic and silicone parts.
5. Check for wear and tear on hoses and all plastic or silicone parts.
6. Replace worn out or damaged parts immediately as this will affect the suction.
7. Parts may be washed in the dishwasher using the top rack. Place all the cleaned parts on top of a paper towel or hand on a drying rack and allow to air dry.
8. Avoid using cloth towels because they can carry germs and bacteria that may contaminate the breast milk.
9. Make sure that all parts are arid before use. Wet parts will cause poor suction. Once all the parts are dry, re-assemble the breast shield set.

Cleaning the hose

The Baboo™ electric breast pump is a closed system; therefore, cleaning the hose is unnecessary. Remove the hose from the breast shield if you see moisture in the hoses after pumping. Turn the motor on for a few minutes until the moisture is gone.

Cleaning the motor

From time to time, sanitise the Baboo™ electric breast pump motor using disinfectant wipes by wiping off the hoses connector jack. Wipe the electric breast pump with a clean, damp cloth or paper towel before use.

Safety recommendations

1. Make sure to wash your hands before touching the electric breast pump and the electric breast pump parts.
2. Avoid touching the inner part of the collection bottle where breast milk comes in contact.
3. Wash all the parts that come in contact with your breasts and breast milk immediately after each use to avoid the growth of any bacteria from the milk residue.
4. The electrical unit must never be exposed to water or other liquids for cleaning. It must not be sterilised in a microwave steriliser.

Troubleshooting

Problem I experience pain when I use the Baboo™ electric breast pump.

Solution

Stop using the electric breast pump and consult your breastfeeding advisor or doctor. If the electric breast pump body does not come away easily from the breast, it may be necessary to break the vacuum by placing a finger between the breast and the massage cushion.

Problem There is low or no suction.

Solution

- Check all connections if they are all properly attached at all connection points.
- Check the duckbill valve if it is not installed or assembled in place. Or the valve is deformed and damaged.
- Check the membrane is correctly attached to the bottle connector.
- Check whether the hoses are not bent and if there is not any water in the hoses.

Problem There is low or no suction.

Solution

- Check if the power cord is correctly attached to the Baboo™ electric breast pump.
- Check if the battery power switch is ON without the adapter.
- Ensure the battery power is available without the adapter.

Problem Milk backflow.

Solution

- If milk or water is found in the membrane or hoses, please rinse the hoses with clean hot water. BEFORE USE, hang them to dry completely. Also, check if the membrane is good and not broken.
- If E1 or E2 appears on the LED display, there is an open or short circuit in the internal battery; please contact your dealer or the store where you bought the electric breast pump.

Technical details

Technical Data

Power Adapter

Power Input: 100 V - 240 V ~ 50/60 Hz

Power Output: 5 V $\overline{\text{---}}$ 1 A

Power Rating: 5W

Li-ion Battery: 3.7 V 1800 mAh

Stimulation Velocity: 100 T/min

Expression Velocity: 26-60 T/min

Stimulation Strength: -0.004~ -0.015 Mpa (max), 9 levels vacuum

Expression Strength: -0.016~ -0.033 Mpa (max), 9 levels vacuum

Timer: 30 mins

Overall Noise: $\leq$ 58 dB(A)

Operating Conditions: 5-40 °C, RH $\leq$ 90 %

Electric Shock Protection: AC adapter Class II, Host machine Class III

Electromagnetic fields (EMF)

This Baboo™ appliance complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

Storage

Keep the Baboo™ electric breast pump out of direct sunlight as prolonged exposure may cause discolouration. Store the electric breast pump and its accessories in a safe and dry place, away from heat sources and out of the reach of children.

Recycling

The product is covered by European Directive 2012/19/EU. The product contains batteries covered by European Directive 2006/66/EC which cannot be disposed of with regular household waste.

Inform yourself about the local separate collection system for electrical and electronic products and batteries. Follow local rules and never dispose of the product and batteries with regular household waste. Correct disposal of old products and batteries helps prevent negative consequences for the environment and human health.

Guarantee and support

If you are facing technical difficulties while using our product, we recommend checking our device instruction manual first. For more information, please visit our website www.baboo.baby. You can ask any questions about the guarantee of our product at the place where you purchased the goods.

If any defect occurs because of faulty materials or assembling, our representative in your country will repair or replace the product at its expense. Also, you can reach our Baboo™ production team in contact by e-mail at returns@baboo.baby.

The warranty does not cover product parts that are subject to wear that can be considered consumable parts by their nature.

The warranty service does not apply for our goods if the damage was caused by:

- the consumer violating the rules of operation, storage or transportation of the product;
- the product was used not according to the recommendations of the operating rules;
- if non-original accessories and/or spare parts that are not described in the operating instructions were used;
- unauthorised persons performed repairs, design or layout has been changed, and software changes were made that were not approved by the manufacturer;
- there were deviations from State Technical Standards and norms of electricity, telecommunications and cable networks;
- the product has been incorrectly assembled and/or turned on;
- there were force majeure circumstances (natural disaster, fire, lightning, etc.).

The warranty does not cover parts of the product that are naturally worn out and can be considered consumables by their nature.

Baboo™ electric breast pump has a 2-year warranty and a 2-year service life, which are calculated from the date of delivery of the product to the consumer or the date of manufacture if the date of delivery cannot be determined. The product's date of manufacture is indicated on the product's packaging.

Baboo™ electric breast pump. Model: XN/MD-2255A. For breast milk suction, domestic purposes. Manufacturer: Guangdong Horigen Mother & Baby Products CO., Ltd. Add: No.18. Pingnan Industrial Zone, Mianbei Street, Chaoyang District, 515100 Shantou, Guangdong, People's Republic of China. TEL: +86 754 83613668 Baboo™ brand owner: BABY'S KINGDOM LTD 10 John Street London WC1N 2EB UK, www.baboo.baby. Tel. +44 7429 602241.

Composition: milk extractor - polypropylene, silicone, thermoplastic elastomer; built-in lithium-ion battery, basket - polyester. Service life - 2 years. Warranty period - 2 years. The teat warranty period is 2 months. Disposable breast pads (cellulose, gel, polyethylene). Disposable bags for breast milk (polyethylene, polyethylene terephthalate). Country of manufacture: China.

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