baboo®
BECAUSE WE CARE

baboo.baby


BABOO® W10 ELECTRIC WEARABLE BREAST PUMP

USER GUIDE

Package Contents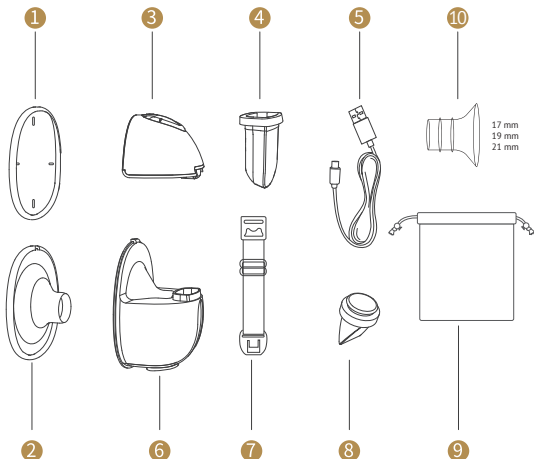

Languages:

Page:

ENGLISH

3

FRANÇAIS

16

ITALIANO

29

ESPAÑOL

42

DEUTSCH

55

SUOMI

69

ČEŠTINA

82

Languages:

Page:

LIETUVIŠKAI

95

LATVIEŠU

108

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121

EESTLANE

136

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149

ROMÂNĂ

162

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175

Introduction

Thank you for choosing the Baboo® W10 Electric Wearable Breast Pump. It is known that breast milk is the best natural nutrition for the baby. Through our collaboration with breastfeeding experts and the experience of nursing mothers, the Baboo® meets the needs of breastfeeding mothers in circumstances that do not allow breastfeeding. Celebrate Freedom with Baboo® Hands-Free Wearable Breast Pump because every moment matters!

Read this manual carefully before using your breast pump and save it for future reference.

By choosing the Baboo® Electric Wearable Breast Pump, you get unique features, such as:

- Hands-Free: Wearable Breast Pump design for busy mums.
- Sleek, Discreet Design: Unique and lightweight for discreet and stylish usage.
- Whisper-Quiet Operation: Extra low noise level below 50 dB ensures peace and privacy.
- Effortless Maintenance: only 5 parts easy assembly and cleaning for busy moms.
- Versatile Modes: 4 modes include stimulation, massage, expression, and auto mode for convenient use.
- Customized Suction: Individually adjustable 9 suction levels for each mode to suit your comfort.
- Auto Shutdown: 20-minute automatic shutdown for safety and convenience.
- Gentle Expression: Super soft, flexible silicone shell for a natural and comfortable milk expression.
- User-Friendly Display: LCD screen and convenient one-finger buttons for effortless and intuitive control.
- Long Battery Life: Built-in Li-ion Polymer battery for extended use.
- Anti-Backflow Design: Ensures hygienic milk collection.
- Travel-Friendly: Dust-free cover, carry-on bag, and self-standing base for portability and convenience.
- Efficient Milk Collection: High 180 ml. capacity, Spill-proof, Easy-pour design.
- Safe Materials: Skin-friendly and BPA-free materials protect both mother and baby's health.

Recommendations for use

Intended use: Baboo® Electric Wearable Breast Pump is used to pump milk from the breast of breastfeeding women. The breast pump can be used when the breast is swelling and overfull, which prevents baby from properly fitting to the breast during feeding. A breast pump can also be used if the mother does not have the opportunity to breastfeed her baby directly, but she considers it necessary to feed her baby with useful breast milk. The breast pump is intended for personal use only.

Important Safety Information

Failure to follow the instructions / safety information may result in a hazardous situation, damage to the appliance or harm your health.

- Always unplug your pump immediately after charging.
- Do not leave your pump unattended when it is connected to a power source.
- Keep device dry and never immerse it in water.
- Repairs must be carried out by an authorized service center only.

- Before use, make sure that the diaphragm is clean and dry, without water droplets inside.
- No parts of the breast pump can be sterilized in a microwave or in a sterilization cabinet.
- Battery life of a breast pump may be reduced after six months of use. If USB cable or the motor is damaged, please contact your dealer or service center. Pump motor contains no parts that can be replaced or repaired by the user.
- Do not open, maintain or repair the pump yourself. Otherwise it will void the warranty.
- The lithium-ion battery is built-in, do not remove the battery yourself.
- Switch off the pump with a lithium-ion battery if the pump is not used for a long time. Charging time must not exceed 2.5 hours in order to prolong the life of the appliance.
- Do not use any batteries that are not included with the pump.

We reserve the right to make changes to the design of the product.

Caution

This product is for individual use. Use by more than one person can be harmful to health.

- You need to be very careful when using Baboo® Electric Wearable Breast Pump in close proximity to children.
- Use Baboo® Electric Wearable Breast Pump only for its intended purpose as described in this manual.
- Do not attempt to remove a working breast pump from the breast. Switch off the appliance, place your finger between your breast and the pump funnel to break the vacuum and remove the pump from your breast.
- Do not use a breast pump during pregnancy as pumping can cause premature birth.
- Do not use Baboo® Electric Wearable Breast Pump during sleep or excessive sleepiness.
- In the absence of milk flow, the pumping should be continued for no more than 5 minutes.
- If pressure seems to be excessive or painful during pumping, lift the pump with your finger to break the vacuum and remove it from your breast.
- Do not use damaged Baboo® Electric Wearable Breast Pump.
- Do not use Baboo® Electric Wearable Breast Pump near devices that create a strong magnetic field.
- Do not expose the Baboo® Electric Wearable Breast Pump to high temperatures and direct sunlight.


Package Contents

- | | |
|--|--|
| 1. Dust cover | 6. Milk collector |
| 2. Silicone breast pad (main) | 7. Bra adjustment buckle |
| 3. Pump motor part (with built-in lithium-ion battery) | 8. Valve |
| 4. Silicone diaphragm | 9. Carrying bag |
| 5. USB Cable | 10. Additional silicone breast pads 17 mm, 19 mm, 21 mm (optional) |

Notes:

The contents of your breast pump may differ depending on the article number. Please refer to the product packaging for exact information on the pump package contents.

Control Panel Description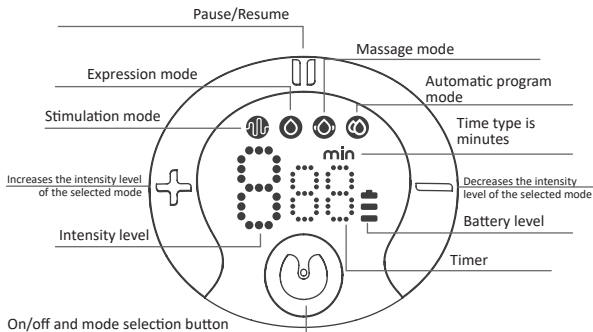

When to express your milk

If you are already breastfeeding, it is advisable (unless otherwise recommended by your doctor/ breastfeeding advisor) to wait for the stabilization of a feeding regimen and milk flow (usually 2-4 weeks after birth).

Exceptions:

If you are already breastfeeding, it is advisable (unless otherwise recommended by your doctor/ breastfeeding advisor) to wait for the stabilization of a feeding regimen and milk flow (usually 2-4 weeks after birth).

- If breastfeeding regimen is not yet stabilized, regular pumping will help stimulate and maintain sufficient milk flow.
- If you pump milk for the baby feeding in the maternity hospital.
- If you and your child are in separate rooms, but you are planning to breastfeed in the future, for milk flow stimulation milk must be pumped regularly.
- If you have strong milk flushes (pain and swelling present): you can pump a small amount of milk before or between feedings to give pain relief and to help your baby to take the breast sooner.
- If your nipples are inflamed or cracked, it is recommended to start pumping after their healing.

It is necessary to choose the most appropriate time for milk pumping, for example before or after the first morning feed when there is a lot of milk in your breast, or after feeding if your baby took only one breast. If you're back at work, you can use a lunch break for milk pumping.

Using a breast pump requires some skills, so it may take several attempts to get your pumping started. The Baboo® Electric Wearable Breast Pump is easy to assemble and use, so you can quickly learn how to use it.

Preparing your breast pump for use

The Baboo® Electric Wearable Breast Pump comes pre-assembled. However, you will need to assemble it by yourself after each cleaning. Before assembly, all parts that come in direct contact with breast milk, such as the breast pad, diaphragm, valve, milk collector, and protective cap, must be cleaned and sterilized before each use.


Assembling the pump for milk pumping

Caution! Corresponding parts of the breast pump must be cleaned and sterilized. Wash hands thoroughly before parts assembling. Be careful, items may be hot after sterilization.


1. Place the valve into the milk collector. Note the direction of the valve.


2. Insert the silicone diaphragm into the milk collector and press it with fingers to ensure a tight seal of the diaphragm. **Caution!** Before use, make sure the diaphragm is clean and dry, with no water droplets inside.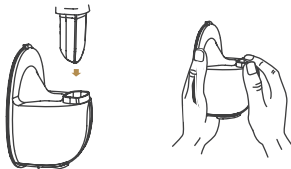

3. Insert the breast pad into the milk collector and make sure that the notch on the breast pad aligns with the notch (for emptying the milk) on the milk collector. If the main breast pad size is not comfortable for you, please choose one of additional breast pads that best fits your nipple size for a comfortable fit. Insert the additional breast pad into the main breast pad. Push it down. Make sure that the connection is tight. Attention! Don't use any additional breast pads without main breast pad.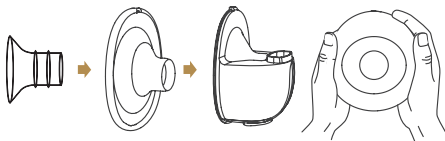

4. Place the pump motor part with built-in battery on the milk collector, align the notch under the pump with the silicone diaphragm and push it down until it connects to the milk collector. Make sure that the connection is tight.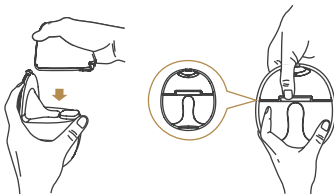

If you are not going to use the breast pump immediately, put the protective dust cover on the pump.


CAUTION! Pump does not operate during charging. Do not use the pump while charging.

How to use the electric wearable breast pump?

Before each use: Make sure the pump is clean and completely dry. Wash the breasts thoroughly, massage the breasts and areolas. Relax to relieve any pain that may result from the milk flush.


Milk pumping

Place the breast pad firmly on the chest, leave no gaps, and make sure the breast nipple is centered in the funnel-shaped breast pad. Keep the pump vertically. Avoid changing the position of the breast and breast pump while pumping, otherwise the pump will not pump.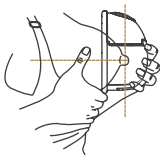


Get into a comfortable position.


Relax and sit on a comfortable chair, lean straight forward (use a pillow to support your back). Make sure that the pump, including the milk collector, is in the upright position.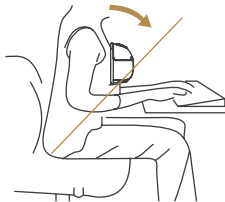

Press the on/off button. A long press of the pump button  activates the pump.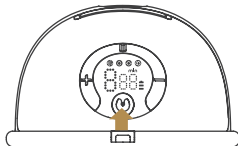

A short press of the button  will start the activated breast pump or PAUSE working breast pump.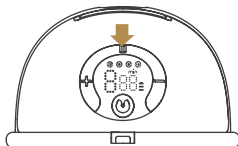

Select a stimulation mode or an expression mode and set it to one of nine levels. You will feel “suction”.

Mode selection


A short press of the button  is used to switch between modes after the pump starts to operate. Press this button to select one of four modes (stimulation mode, expression mode, massage mode and combo mode).

If the indicator  is on, it means that the STIMULATION MODE is on. The stimulation mode helps to produce milk in case of milk absence.


If the indicator light  is on, the expression mode is activated. Use the expression mode to pump your milk.

If the indicator  is on, it means that the MASSAGE MODE is on, the massage mode is used to relax the breast and to relieve possible pain in the event of a strong milk flush and swelling of the breast and to prepare the breast for pumping.

If the indicator  is on, it means that the AUTOMATIC MODE is activated. Automatic mode is a combination mode that combines massage and expression modes. The massage mode is first activated and the massage intensity automatically increases. Then the pump is activated. This mode is most suitable for initial use of the breast pump until you have customized the settings to suit your needs.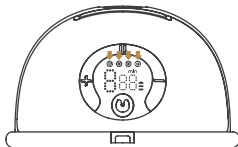

There are nine levels of intensity for each breast pump operation mode. You can set the intensity level that is comfortable for you. A short press of the “+” button increases the intensity level, and a short press of the “-” button decreases the level.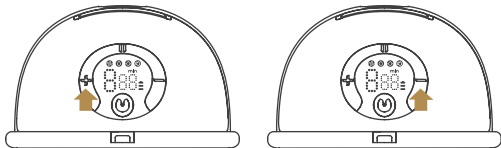


When you choose an expression mode, you may select a slower expression mode after milk started to flow. Note: Don't worry if the milk doesn't start to flow right away. Relax and continue pumping. When you start using milk pump, you may need a more intense expression mode.

Depending on your feelings and preferences, you can use a more intense expression mode, which increases the speed of milk pumping. You can switch off the breast pump or select a less intense mode at any time. Note: There is no need to use all expression modes. Only use the modes that are comfortable for you.

If pumping does not produce results, continue trying for no more than 5 minutes. Try pumping at a different time of day. If you feel pain or discomfort while pumping, stop using the pump and consult your doctor/breastfeeding practitioner. On average, it takes 10 minutes to pump 60 to 125 ml of milk. However, this is approximate amount.

When you have finished pumping, long press the button  to switch off the pump.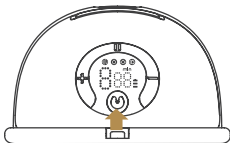


Caution! This model has a built-in memory function, when you switch on the appliance for future use, the mode and intensity settings you last used will automatically be set.


If you do not use the pump for 20 minutes, the pump will automatically switch off.

Completion of pumping

When you have finished pumping, carefully remove the pump from your breast when you have switched off. To relieve vacuum, place your finger between your breast and the breast pad.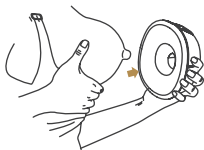

Hold the pump upright. Separate the motor part from the milk collector.

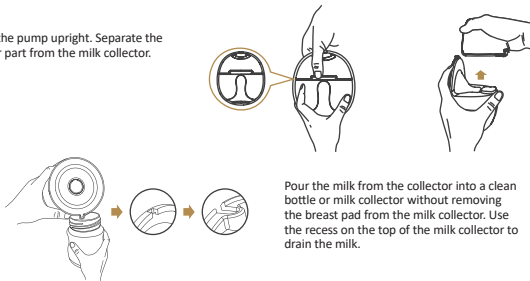Pour the milk from the collector into a clean bottle or milk collector without removing the breast pad from the milk collector. Use the recess on the top of the milk collector to drain the milk.

Disassemble the breast pump and clean all parts as recommended.

Cleaning and maintenance

Before using the pump for the first time:

1. Detach the components that come into contact with breast and breast milk.
2. Wash all these components in warm soapy water using any odorless baby liquid dish cleaner.
3. Rinse each component thoroughly with warm water for 10-15 seconds.
4. You can disinfect components of your breast pump (except pump unit) using your chosen sterilization method, but no more than 3 minutes.
5. Do not use antibacterial or abrasive products to clean the components of the breast pump.

Cleaning the pump motor part

From time to time, clean the pump motor part with disinfecting cloth moistened with 70% ethanol or isopropyl alcohol. Clean the pump motor part with a clean damp cloth or paper towel before each use. *Never immerse the pump motor part with the built-in battery in water.*

How to clean your breast pump after regular use

1. Wash each part that comes into contact with your body and/or milk with cool water immediately after pumping. This avoids the accumulation of milk residues in the milk collectors and in the parts of pump to prevent the spread of bacteria.
2. Parts that come into contact with breast and breast milk: protective cover, breast pad, silicone diaphragm, milk collector, valve. Wash all these components in warm soapy water using any odourless baby liquid dish cleaner.
3. Use a separate cleaning sponge and bottle brush with a small brush to reach all hard-to-reach areas. Clean the parts of the breast pump separately from adults' dishes, pots and pans, etc.

WARNING

- Be careful when removing and cleaning the valve. If the valve is damaged, the pump will not function properly.
 - Use warm water with a few drops of detergent for valve cleaning. Do not clean the valve by inserting foreign objects (brushes, etc.) into the valve. Doing so may damage the product.
4. Rinse each part thoroughly with warm water for 10-15 seconds. You can disinfect parts with your chosen sterilization method each time you use them, but not more than 3 minutes, excessive and frequent boiling or exposure to heat can cause plastic and silicone parts to wear.
 5. Carefully inspect all plastic or silicone parts for damage.
 6. Replace damaged parts immediately as they will affect the performance of the pump.
 7. The parts of the breast pump, except pump motor part, can also be washed on top shelf of a dishwasher. Place all washed parts on paper towel or place into dryer and allow to air dry.
 8. Avoid using fabric towels, as they may contain germs and bacteria that can get into breast milk.
 9. Make sure all parts are completely dry before you use the pump. Wet or damp parts can cause poor suction. Once all parts are completely dry, reassemble the pump.

Safety recommendations

1. Always wash your hands thoroughly before touching the pump and its parts.
2. Do not touch the inside of the areas where breast milk enters.
3. Clean all parts of the pump that come into contact with your breast and breast milk immediately after each use to avoid bacteria growth in the area of milk residues.
4. Do not expose the pump motor part to water or other liquid cleaning agents. It cannot get in contact with water or sterilized in a microwave / sterilizer.

Instructions for bra adjusting buckle

Adjusting the length of the bra strap with the adjusting buckle will help you to pump more comfortably. Unfasten breastfeeding bra cup (Fig.1). Attach one end of the adjusting buckle to the buckle of the breastfeeding bra (Fig. 2). Attach the other end to the cup of your bra. Adjust the length of the adjusting buckle to ensure that there is sufficient space for a comfortable positioning of the breast pump (Fig.3).

Fig.1

Fig.2

Fig.3

Battery charge indicator

Indication of the battery level indicator	Remaining pump operating time
Three bars light up continuously	At least 100 minutes
Two bars light up continuously	At least 70 minutes
One bar lights up continuously	At least 30 minutes
One bar blinks	At least 10 minutes

Charging the pump motor

The Type-C charging port is protected by a silicone cover. To charge the device, gently lift the silicone cover and connect with USB cable. Once charging is complete, disconnect motor from USB cable, securely push the silicone cover back into place, ensuring it is correctly positioned. You can now assemble the pump motor with sterile milk collector and use the pump.

Storing and defrosting breast milk

Only store milk collected with a sterile breast pump. You should properly store and defrost your breast milk. Only in this way you can be sure that all important nutrients are preserved and the risk of bacterial contamination is minimized. You can store milk in the bottle covering it with a sterilized teat, coupling ring and lid, or by pouring it into a breast milk storage bag. Recommendations for storing/defrosting breast milk:

We recommend that you use the following guidelines for storing and defrosting breast milk:

Storage conditions	Freshly expressed milk	Thawed milk (previously frozen)
At room temperature	Up to 4 hours at temperatures below 25°C	Do not store at room temperature
Refrigerator	5 days at a temperature of about 4°C	10 hours
Refrigerator freezer	2 weeks at a temperature of about -15°C	Do not freeze repeatedly after thawing
Freezer	3-6 months at a temperature of about -18°C	Do not freeze repeatedly after thawing

Tips for freezing/defrosting breast milk:

- Never fill milk bottles/bags completely, as milk may expand when frozen.
- Mark milk bottles/bags with the date of milk collection.
- Do not store pumped milk in the refrigerator door. Store it at the bottom of the refrigerator close to the back (coldest temperature).
- Defrost frozen breast milk leaving it in a refrigerator overnight or hold a bottle or a milk storage bag under running warm water (max. 37°C).
- Gently shake the bottle or bag to mix all separated fats.
- Do not defrost frozen breast milk in a microwave oven or in a pot of boiling water (risk of burning).

Recommended actions

- Put collected milk into a fridge or freezer immediately after collection.
- Should be stored only milk collected by a sterilized breast pump and only in sterilized bottles.

Not recommended actions

- Never freeze milk repeatedly.
- Never add fresh milk to frozen milk.

Feeding:

Always check milk temperature before giving milk to your baby.

Always dispose milk residues after feeding.

Never heat milk in a microwave oven, microwaves heat liquids unevenly, which can lead to overheating points and destruction of the health benefits of milk.

Troubleshooting

Problem: Pain occurs during pumping with a breast pump.

Solution:

Stop using the breast pump and consult your doctor/breastfeeding practitioner. If it is difficult to remove the pump from the breast, try inserting a finger between the breast and the massage cushion of the breast pump

Problem: The breast pump suction is weak or no suction at all.

Solution:

- Check all connections for correctness and tightness at all connection points.
- Check the valve, its presence in the place and the presence of deformation or damage.
- Check that the diaphragm is correctly inserted into the pump.

Problem: The pump will not switch on.

Solution:

- Make sure the battery is charged.
- Disconnect the breast pump from the charging cable. Caution! The pump does not operate during charging.

Specifications and parameters

USB: Type C.

Charging time: approx. 2,5 hours.

Operation time of fully charged pump: approx. 90 min.

USB adapter Input: 100-240V (50/60Hz)

USB adapter Output voltage/current: DC 5V 1A

Power: 5W

Lithium-ion battery: 3.7 V 1200 mAh

Stimulation mode rate: 77 – 95 T/min.

Expression mode rate: 30 – 92 T/min.

Pressure in stimulation mode: -100mmhg ~-150mmhg, -0,01Mpa~-0,02Mpa

Pumping pressure: -100mmhg ~-280mmhg, -0.01Mpa~-0.04Mpa

Timer: 20 minutes

Noise level: ≤50dB(A)

Operation temperature: +5°C to +40°C.

Storage temperature: -20 to +60°C.

Operation and storage humidity 15-90%

Operation and storage pressure 90-106 kPa

Protection against electric shock: Main unit Class III.

Electromagnetic fields (EMF)

This Baboo® appliance complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

EMC Directive 2014/30/EU

Essential Requirements		Harmonized Standards
EMCD Clause 1(a)	Emission	N IEC 55014-1:2021; EN IEC61000-3-2:2019+A1:2021; EN 61000-3-3:2013+A2:2021
EMCD Clause 1(b)	Immunity	EN IEC 55014-2:2021

Materials used:

- Dust cover – polypropylene
- Breast pad – silicone
- Pump motor part, with built-in lithium-ion battery – ABS plastic
- Diaphragm – silicone
- Milk collector – polypropylene
- Bra adjustment buckle: PET (polyethylene terephthalate)
- Valve: silicone
- The carrying bag – polyester

Storage

Do not expose the breast pump to direct sunlight for long periods of time, as this may cause discoloration. Store the pump and its accessories in a safe and dry place away from heat sources and away from children.

Disposal

The product is subject to Directive 2012/19/EC of the European Parliament and Council.

This product contains batteries that are subject to Directive 2006/66/EC of the European Parliament and Council and cannot be disposed of with household waste.

Read the local regulations for separate disposal of electrical and electronic products and batteries. Follow these guidelines and do not dispose of the product and batteries with household waste. Proper disposal of products and batteries helps to prevent possible negative consequences for the environment and human health.

Warranty & Support

If you encounter any difficulties while using the product, we recommend that you first read the user manual. For support or information, visit www.baboo.baby. For warranty service, contact the place of purchase.

If your product is found to be defective, the manufacturer's representative in your country will assess the issue. If necessary, they will either repair the defective product or, if repair is not possible, replace it or provide a refund, in accordance with the laws of the country where the product was purchased, throughout the warranty period. Additionally, you can contact our Baboo® production team via email at returns@baboo.baby.

This warranty applies only to Baboo® Electric Wearable Breast Pump used for personal purposes.

Warranty service does not apply to products whose defects arose due to:

- violation by the consumer of the rules of operation, storage or transportation of the product;
- absence or improper maintenance of the product according to the recommendations of the operating instructions.
- use of non-original accessories and/or consumables not provided for in the operating instructions;
- repair by unauthorized persons, making structural or circuit changes and changes to the software unauthorized by the manufacturer;
- deviations from state technical standards and norms of power, telecommunications and cable networks;
- improper installation and/or connection of the product;
- actions of force majeure (natural forces, fire, lightning, etc.).

The warranty does not cover parts subject to natural wear and tear, which by their nature may be considered consumables.

The Baboo® Electric Wearable Breast Pump is provided with warranty period of 1 year and a service life of 1 year, which is calculated from the date of transfer to the consumer or from the date of manufacture, if the date of transfer is impossible to determine. The date of manufacture is indicated on the packaging of the product. For more information, please contact baboo@sentija.com.

Baboo® Electric wearable breast pump. Model: W10. For breast milk collection. For domestic needs.

Baboo® brand owner: BABY'S KINGDOM LTD 10 John Street, London, WC1N 2EB UK, www.baboo.baby. Tel. +44 7429 602241. Importer and representative in European Union: UAB „Sentija“, J. Kubiliaus st. 4b - 1, Vilnius, Lithuania, LT-08241, baboo@sentija.com.

Country of origin: China. Manufacturer: Shenzhen TPH Technology Co., China.

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