GET READY!

babynono

EN	COMPACT PLUS electric breast pump with unique nasal aspirator mode.	2
PL	Laktator elektryczny z aspiratorem do nosa COMPACT PLUS.	8
BG	Електрическа помпа за кърма COMPACT PLUS.	14
	Уникален режим на аспиратора за нос.	
CZ	Elektrická odsávačka mateřského mléka COMPACT PLUS.	20
	Jedineční režim nosní odsávačky.	
DE	Milchpumpe COMPACT PLUS. Einzigartiger Modus des Nasenaspiretors	26
EE	Elektriline rinnapump COMPACT PLUS. Unikaalne ninaaspiratori režiim.	32
ES	Extractor eléctrico de leche materna COMPACT PLUS.	38
	Modo único del aspirador nasal.	
FR	Tire-lait électrique COMPACT PLUS. Mode unique de l'aspirateur nasal.	44
GR	Ηλεκτρική συσκευή άντλησης μητρικού γάλακτος COMPACT PLUS.	50
	Μοναδική λειτουργία του ρινικού αναφροφτήρα.	
HR	Električni laktator COMPACT PLUS. Jedinstveni način nazalnog aspiratora.	56
HU	PLUS elektromos mellszívó. Országos egyedülálló üzemmódja.	62
IT	Tiralatte elettrico COMPACT PLUS. Una modalità particolare dell'aspiratore nasale.	68
LT	Elektrinis pienatraukis COMPACT PLUS. Unikalus nosies aspiratoriaus režimas.	74
LV	Elektriskais laktators COMPACT PLUS. Unikāls deguna aspiratora režīms.	80
NL	Electrische borstkolf COMPACT PLUS. Unieke neusreinigingstand.	86
PT	Bomba tira leite eletrica COMPACT PLUS. Modo único do aspirador nasal.	92
RO	Pompă de sân electrică COMPACT PLUS. Modul unic de aspirator nazal.	98
RU	Молокоотсос электрический COMPACT PLUS.	104
	Уникальный режим назального аспиратора.	
SE	Elektrisk bröstpump COMPACT PLUS. Unikt nässugsläge.	110
SK	Elektrická odsávačka materského mlieka COMPACT PLUS.	116
	Jedinečný režim nosovej odsávačky.	
UA	Молоковідсмоктувач електричний COMPACT PLUS.	122
	Уникальний режим назального аспіратора.	

COMPACT PLUS electric breast pump with unique nasal aspirator mode

INTENDED USE

- Expressing milk from a woman's breasts
- Expressing excess milk from overfilled breasts
- Storage of excess milk to be used later

NOTE!


- Before using the breast pump for the first time make sure you read the user manual and warnings. Keep the user manual for future reference.
- Use the breast pump in accordance with its intended use described in the user manual.
- Do not use the breast pump if it is physically damaged.
- Do not place the breast pump near devices generating a strong magnetic field.
- Do not expose the breast pump to direct sunlight.
- Keep the breast pump out of reach of children.
- Keep the pumping unit of the breast pump in a dry place and do not immerse it in water.
- Before each use, check all the elements visually for cracks, chips, tearing, stains, discolouration and signs of wear since they may affect the efficiency of the device. If the breast pump is damaged it cannot be used until the damaged parts are replaced with new ones.
- Keep the breast pump clean.

BEFORE EACH USE

Find a quiet place and make sure that all the elements of the breast pump coming in direct contact with the food (silicone breast funnel, breast pump body, silicone membrane, silicone valve and bottle) have been sterilised and correctly assembled.

- Wash your hands thoroughly with soap and warm water before touching any element of the breast pump.
- After taking a comfortable sitting or semi-recumbent position, gently massage the breasts and heat them with a warm towel.
- Before using the breast pump make sure that all the parts have been cleaned, sterilised and dried.
- Make sure there is no liquid in the tubing. Do not switch the breast pump unless the tubing is empty.

LIST OF BREAST PUMP PARTS

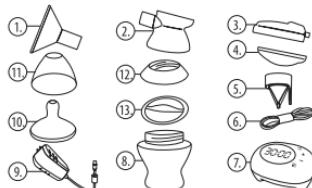1. Silicone breast funnel
2. Breast pump body
3. Pump body cap
4. Silicone membrane
5. Silicone valve
6. Silicone tubing
7. Pumping unit
8. Feeding bottle
9. Charger
10. Teat
11. Bottle cap
12. Cap
13. Bottle sealing disc

ASSEMBLY INSTRUCTION

1. Insert the breast funnel (1) into the breast pump body (2).
2. Put the silicone membrane (4) on the breast pump body.
3. Put the pump body cap (3) over the membrane, turning it to the right until it locks.
4. Put the silicone valve (5) into the bottom of the pump body (make sure it's been done correctly).
5. Fix the feeding bottle (8).
6. Fix the silicone tubing (6) onto the splined aperture in the pump body and connect the other end to the pumping unit (7).

USER MANUAL

- 1) 2) 3) 4) 5)

a) battery level indicator, b) clock and suction strength indicator, c) level down, d) level up, e) operating mode
1) Expression, 2) Double frequency, 3) Stimulating massage, 4) Two in one, 5) Aspirator mode.

Take the correct and comfortable position. Put the funnel to the breast so that the nipple is in the funnel tube. Make sure the tubing is not bent or tangled.

Touch the ON/OFF button for 2 seconds to turn on the power. The display will start. Starting the breast pump activates the time switch set for 1 minute. Unless you start pumping the milk within that time, the pump will turn off automatically.

Touch e) symbol to select the operating mode.

Select 1) to turn on the EXPRESSION mode. The suction strength can be set in the range from 1 to 7. Individual levels differ in terms of suction strength, from low to high.

Select 2) to turn on the DUAL FREQUENCY mode. The mode combines the advantages of alternating high to low suction strength. The suction cycle with a variable strength (low – break – high) effectively helps in stimulating inverted nipples. The suction strength can be set in the range from 1 to 7.

Select 3) to turn on the STIMULATING MASSAGE mode. It is recommended to start a stimulating massage approx. 2-5 minutes before pumping the milk. The suction strength can be set in the range from 1 to 5. Each level offers a massage with a different frequency.

Select 4) to turn on the TWO IN ONE mode. The breast pump performs 8 massage cycles and 1 suction cycle. This mode is recommended for soothing the pain of

overfilled breasts and for stimulating them to express more milk in less time. The suction strength can be set in the range from 1 to 7. Each level gently massages the breast and offers different suction strength, from low to high.

Select 5) to turn on the NASAL ASPIRATOR MODE. Mode adapted to work with the aspirator. The suction strength is adjustable in the range of 1 to 3.

Sleep mode: Unless you start pumping the milk within a minute after turning on the pump, it will turn off automatically. The default pump operating time is 30 minutes. After working for 30 minutes, the breast pump turns off automatically.

Memory mode: The breast pump automatically remembers the last used operating mode and selected suction strength.

Warning

USE ONLY THE CHARGER SUPPLIED WITH THE BREAST PUMP. MAKE SURE THE CHARGER IS COMPATIBLE WITH THE POWER SOURCE.

CHARGING THE BATTERY

NOTE! Before first use, the battery must be fully charged. The breast pump is equipped with a 2600mAh lithium battery, so it does not require the use of (AA type) batteries or permanent connection to power supply. It is recommended to charge the breast pump before starting to express milk.

The battery provided with the breast pump can be charged approx. 500 times. Stop charging when the battery is full in order to avoid harming the battery. In order to extend the life of the breast pump and reduce the electricity consumption by the battery it should not be charged for more than 6 hours.

BATTERY PARAMETERS

Battery capacity	2600 mAh
Max. negative pressure	-0.04 MPa
Rated voltage	5V
Rated current	1A
Rated power	5W
Charging time	approx. 5 hours
Operating time	approx. 2.5 hours

The breast pump is equipped with a battery. To charge it, connect the pump to the charger and then to the power outlet. It will be fully charged after approx. 5 hours. During charging the indicator alternates between blue and white. When the battery is charged, the indicator turns white. The red indicator means the battery is low and needs to be recharged immediately. The battery discharge rate depends both on the mode and operating time of the breast pump and on the suction strength set. Proper operation of the built-in battery allows to extend its life. It is recommended to turn off the pumping unit of the breast pump while the battery is charging. The battery should be replaced by a specialist. Do not disassemble the built-in battery without the consent of the manufacturer. The breast pump can be recharged at any time, without having to wait for the battery to discharge.

SAFETY RULES

The following safety rules should be always complied with:

- In the case of inflammation of the mammary glands and inverted or flat nipples the battery-powered breast pump should be used only after consultation with a physician.
- The breast pump is intended for personal use. Sharing it by more people poses a health risk.
- Store the battery-powered breast pump out of the reach of children.
- Do not use the battery-powered breast pump when you are asleep or grow drowsy.
- Do not use the battery-powered breast pump if it is damaged.
- Do not use the battery-powered breast pump equipped with parts from other manufacturers. It may reduce its efficiency.
- Do not attach a glass feeding bottle to the battery-powered breast pump. It can get broken if falls down.
- Do not leave the battery-powered breast pump unattended when it is connected to a power source.
- Do not reach for the battery-powered breast pump if it falls into the water. In such a situation, immediately disconnect it from the power source.
- Do not expose the battery-powered breast pump to direct sunlight.
- All repairs of the breast pump should be made by an authorised service provider.

CLEANING AND STERILISATION

Before using the battery-powered breast pump, dismantle, wash and sterilise all its components.

Improperly dried parts can interfere with the proper operation of the battery-powered breast pump. Before use, make sure all the parts are dry.

Cleaning:

- Do not clean or sterilise the pumping unit, charger, silicone tubing and tube connector.
- Clean and sterilise all the parts coming in direct contact with the food: silicone breast funnel, breast pump body, silicone membrane, silicone valve and bottle.
- Wash the parts of the breast pump with warm water to avoid contaminating the food with bacteria.
- Do not clean with solvents or abrasives.
- Wipe the pumping unit with a cloth lightly moistened with water or water mixed with a mild soap.

Cleaning the tubing:

- Wipe the outside of the tubing with a clean, damp cloth. Make sure that no water or other liquid gets into the tubing.
- If there is any fluid in the tubing, rinse it with clean, hot water. Then hang until completely dry. Do not insert any objects into the tubing.

TROUBLESHOOTING

TOO LOW OR NO SUCTION STRENGTH

- Make sure that all the parts of the breast pump are correctly assembled.
- Make sure that the membrane is properly fixed and placed flat against the breast pump body.
- Check that the silicone valve and the silicone breast funnel of the breast pump are correctly assembled.
- Make sure that the silicone breast funnel tightly adheres to the breast.

BREAST PUMP IS WET

- Turn off the breast pump. Disconnect the charger from the power source.
- Do not put the breast pump upside down. The control panel must be facing up.
- Leave the breast pump for 24 hours in a dry, warm place.
- Do not expose the breast pump to direct sunlight.

- Do not accelerate the drying process by heating the breast pump elements.

BREAST PUMP DOES NOT START

- Make sure that the power cord is properly connected to the breast pump.
- Make sure the power source is live.
- Contact the store where the breast pump was purchased.

NATURAL NURSING FEEDING BOTTLE AND TEAT

The teat supplied with the breast pump is intended for feeding bottles from the Natural Nursing product family. Before each use, please check the teat by pulling it in all directions. If the teat is damaged, it should be immediately replaced with a new one. Do not enlarge the apertures in the teat. It may damage the silicone structure. For reasons of hygiene and safety, it is recommended to replace the teat every 1-2 months

IMPORTANT FOR YOUR BABY'S HEALTH AND SAFETY

- Never leave your child unattended during feeding.
- Do not use feeding teats as soothers.
- Prolonged sucking of food may cause tooth decay.
- Always check the food temperature before feeding.
- Always assemble sterilised bottle elements with clean hands.
- The product complies with the requirements of the EN 14350 standard.

EXPRESSED BREAST MILK STORAGE

IMPORTANT: Please make sure any expressed breast milk is correctly stored! Incorrectly stored breast milk loses its nutritive value and may even be harmful to your baby.

We recommend Babyono sterile bags for breast milk storing and freezing.


STORAGE PLACE	STORAGE TIME	
	FOR HEALTHY BABIES	FOR PREMATURE AND ILL BABIES
In room temperature	8 – 12 hours	1 – 4 hours
In a fridge (5°C)	3 – 4 days	24 – 48 hours
In a freezer compartment (-10°C)	2 weeks	1 week
In a freezer (-18°C)	6 – 12 months	3 – 6 months

NOTE

- Thawed milk must not be frozen again. Any unused milk should be discarded.
- Do not mix any thawed breast milk with freshly pumped breast milk.
- You can freeze only fresh breast milk.
- Breast milk should be defrosted slowly. To do this, frozen milk should be placed in a refrigerator until it takes a liquid form.
- To warm breast milk, it is recommended to use a water bath with a temperature not exceeding 50°C.
- To preserve its nutritive value, breast milk must not be boiled, heated up directly on a cooker or in a microwave!
- Ready to serve breast milk should have the human body temperature, i.e. no more than 37°C.

ASPIRATOR

When using nasal aspirator mode, please replace the suction device accessories for matching use. When installing the accessories, the sealing ring should be fully assembled to ensure good air tightness of the accessories. Pay attention that the amount of secretions absorbed should not exceed the capacity of the collector. And please ensure that accessories and sponges are dry and free of liquid when using, in order to avoid liquid backflow to damage to the pumping unit.

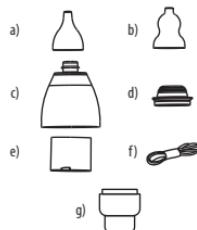- a) suction tip: Tip for removing nasal mucus secretion,
- b) suction tip: Tip for removing nasal mucus fluid,
- c) body of the aspirator: Housing protecting the interior of the aspirator,
- d) mucus chamber plug: prevents the reverse flow of the sucked out mucus secretion,
- e) mucus chamber: a container for collecting the sucked

- out mucus secretion,
- f) silicone tubing,
- g) base with a filter.

PREPARATION BEFORE USE

- Having carefully taken the device out of the packaging, make sure that:
 - the device is not damaged
 - no parts are missing
 - if the device is damaged or if parts are missing, contact the seller.

HOW TO USE THE ASPIRATOR

1) Aspirator assembly

- The mucus chamber should be closed with the plug.
 - The assembled chamber should be inserted into the device and twisted.
 - The suction tip should be attached to the body of the aspirator.
- Connect the aspirator with a silicone tube to the breast pump. Turn on the pump and select the aspirator mode. Adjust the suction strength using the +/- buttons.
 - The device is NOT a toy. Children MUST NOT play with the device. Each component of the device can be swallowed by a child and the child can choke.

AFTER USE

- Disassemble the aspirator by disconnecting it from the pump.
 - Remove the secretion collected in the mucus chamber
 - Sterilise the disassembled components of the aspirator by washing them in boiling water or precisely clean them under running water.
 - Dry carefully after washing.
- When sterilising the aspirator's components in boiling water, prevent them from touching the hot bottom of the water container.

SAFETY MEASURES

- You are kindly requested to comply with the "Safety recommendations", which describe proper and safe use of the device and protect you from any hazards.
- The "Safety recommendations" are marked as "Warning" or "Caution" in accordance with the description below.

WARNING A potentially hazardous situation, which, if not prevented, can result in critical or serious damage.

CAUTION A potentially hazardous situation, which, if not prevented, can result in minor damage.

This symbol means you should take special care because there is potential damage in particular circumstances. You are requested to follow the recommendations - this way you will avoid any potential hazards.

- Persons suffering from the below listed diseases should consult a doctor before use.
- Persons suffering from heart diseases, malignant tumours and diseases requiring rest and relaxation
- Persons suffering from cardiac disorders, with sensitive skin or other skin diseases
- Persons suffering from psychological disorders, with scars on the nose, suffering from frequent swelling of the body or high fever
- Persons under regular medical care due to other health problems or persons who regularly use products in connection with present or past health problems
- Persons under medical care due to health problems or allergies concerning the nose, the eyes and the ears or persons who require urgent medical intervention
- The device must NOT be used for any purposes contrary to its intended purpose.

CAUTION:

- Avoid the overflowing of the suction pump chamber. It must always be cleaned after use.
- Switch the power off before detaching the aspirator from the body of the device.
- Having checked the switch status, make sure the device operates properly.
- When using the device, pay particular attention to the following issues:
 - Check the condition of the device and check if its use is not uncomfortable for the patient.
 - If the patients feels uncomfortable with the device use or if the device does not operate properly, switch the aspirator off to make sure the patient is safe, then undertake further steps.
- Before using the device, read the instruction manual carefully.
- During the device storage period, pay particular attention to the following issues:
 - Keep the device away from the reach of children

- Always keep and use the device in dry environment
 - Always keep the device in a place free of any adverse effects of its storage conditions, such as polluted air, adverse temperatures, moisture, ventilation, dusting, wind, presence of salts or other pollutants.
 - Pay attention to the sloping surface, vibrations or shocks.
 - The device must not be stored near any chemicals or gases.
 - The device must not be stored near any sources of heat e.g. fireplaces, radiators, stoves, tv sets or other household appliances.
 - The device must be stored in cleaned and ready to use condition.
7. If you find any fault in the device operation, do not try to disassemble the device but contact customer service.
8. NEVER modify the device in any way.
9. Maintenance
- Regularly inspect the condition of the device and its components.
 - Avoid continuous use of the device for a longer time period and make sure the device operates as usual if prior stored.
 - Never allow children to play with the device. Some of the device components can be swallowed, which can lead to choking.
 - Consult a doctor regarding the manner and frequency of use of the device on children.
 - NEVER insert the device too deeply into the nose. The tips of the device must NEVER be pressed too hard to the nostril walls.
 - NEVER insert the device into the ears, mouth or eyes. The aspirator that is described here is intended for the removal of mucus secretions from the nose.
 - The device must NOT be used for any purposes non-compliant with its intended purpose.
 - In case of use of the baby cleaning tweezers, use them with protective tips or otherwise baby's nose can get injured.

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