GET READY!

babyono

GB	MEGAN hands-free electronic breast pump.	3
PL	Laktator muszlowy elektryczny MEGAN	6
BG	Електронна помпа за изцеждане на кърма MEGAN.	9
CZ	Elektronická odsávačka mléka MEGAN.	12
DE	Elektrische Milchpumpe MEGAN.	15
EE	Elektriline karprinnapump MEGAN.	18
ES	Extractor de leche en forma de concha MEGAN.	21
FR	Tire-lait MEGAN.	24
GR	Ηλεκτρική αντλία μαστού με χούνη MEGAN.	27
HR	MEGAN elektronička pumpica za dojenje oblika školjke.	30
HU	MEGAN Elektromos, kagylós mellszívó készülék.	33
IT	Tiralatte elettronico a conguaglia MEGAN.	36
LT	Elektrinis pientraukis MEGAN.	39
LV	MEGAN gliemežnīcas elektroniskais krūts piena sūknis.	42
NL	Elektrische borstkolf MEGAN.	45
PT	Extrator de leite eletrônico MEGAN	48
RO	Pompă de sân electrică cupă MEGAN	51
RU	Молокоотсос электронный MEGAN	54
SE	MEGAN Snäckskaal elektronisk bröstpump.	57
SK	Elektronická odsávačka mlieka MEGAN.	60
UA	Молоковідсмоктувач електронний MEGAN.	63
AR	فد ص ل ك ش ي ل م EMEAN ة ي ا ب ر ه ك ل ا ي د د ث ل ا ة خ ض م	66

MEGAN hands-free electronic breast pump

ORIGINAL FEATURES

-  • Creative hands-free and wireless design, easy to carry
-  • Exclusive pumping mode to further satisfy the mother
-  • Digital screen clearly displaying all operating parameters
-  • Built-in battery for use at the time and location of your convenience

INTENSION TO USE

- expressing milk from lactating breasts
- relieving engorgement when breasts are full
- storing excessive breast milk to nurse at any time


BEFORE EACH USE

Choose an undisturbed place and ensure all parts that have direct contact with breast milk (breast shield, milk bottle, silicone gas storage, duckbill silicone valve) have been sterilized and assembled properly.


1. Wash your hands thoroughly with soap and warm water before handling breast pump attachments.
2. Do a hot compress with warm towel and massage your breasts gently and while sitting or lying in a comfortable position.
3. Make sure attachments is cleaned, sterilized and dried before pumping.

PART OF WEARABLE BREAST PUMP

No	Name of parts	Material	Heat-resistance temp.
a	main pump unit	ABS, silicone	/
b	milk bottle	tritan	-10~90°C
c	silicone gas storage	silicone	-20~180°C
d	tee joint connector	PP	-20~80°C
e	duckbill silicone valve	silicone	-20~180°C
f	breast shield (26 mm)	silicone	-20~180°C
g	insert funnel (21, 24 mm)	silicone	-20~180°C
h	USB charging cable	/	/
i	bra adjustment buckle	/	/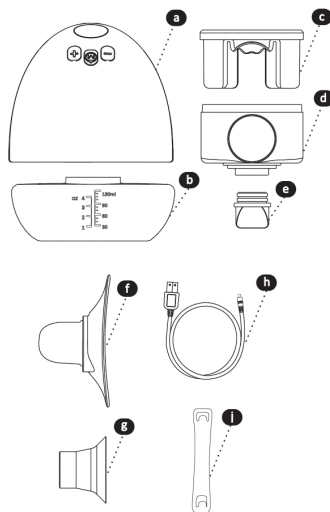


CONTROL PANEL INSTRUCTION

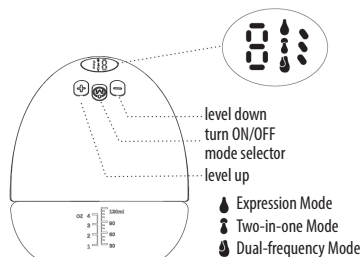
PRODUCT PARAMETER

Model	598
Input Power	5V --- 1A
Rate Power	5W
Suction Strength	≤ -0.036 MPa (-270 mmHg)
Battery Capacity	1100 mAh
Charging Time	≈ 2.5 hours
Running Time	≈ 2 hours


CONTROL PANEL INSTRUCTION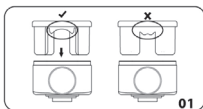


Install the silicone gas storage into the tee joint connector.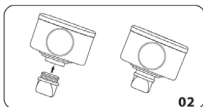


Insert the duckbill silicone valve firmly into the tee joint connector.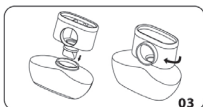


Rotate the tee joint connector to the milk bottle.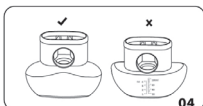

Please check if the milk bottle is assembled in the correct direction.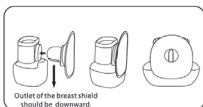


Insert the breast shield into the tee joint connector.

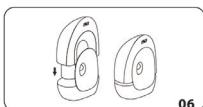Install the main pump unit on the tee joint connector.

Choose the 21 mm insert funnel depending on your personal needs.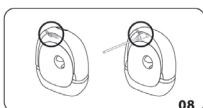

Open the rubber plug on the back of the main pump unit, insert the USB-C port and connect with power to recharge it.

HOW TO OPERATE THE BREAST PUMP

1. Maintain correct posture and ensure that the nipple is properly centered in the breast shield and the cup fits tightly to the breast.
2. Long press  on top of main pump unit to switch on the power, the digital display screen would be lit up and 1 minute timer would start once turning on main pump unit. If do not operate main pump unit, it would switch off automatically if it had been idle for 1 minute.
3. Press  to choose desired mode.
4. Press "+" and "-" to increase and decrease levels of each mode.
5. Choose  to enter EXPRESSION MODE. Suction range is from L1 to L9. Each level offers different pumping strength from weak to strong.
6. Choose  to enter TWO-IN-ONE MODE. Suction range from L1 to L9. May be helpful to relieve pain of overfull breast and to promote breast milk in short time smoothly.
7. Choose  to enter DUAL-FREQUENCY MODE. Suction range is from L1 to L9. May be helpful in assisting mothers to resolve inverted and short nipples.
8. When pumping finish, please bent down to take out the breast pump. When taking out the breast pump, please grab the main pump unit and the breast shield together, to avoid pulling out the main pump unit only.
9. Sleep Mode: The main pump unit default running circle 30 minutes. After working 30 minutes, the pump would switch off automatically.
10. Memory Mode: The main pump unit will automatically memorize the using mode and level which was last use.

BATTERY INSTRUCTION

- When the battery has only 1 bar left, it means the battery is low and should be recharged immediately. We recommend that you use a certified 5V 1A adapter to charge the main pump unit.
- When charging, three bars of the battery will be flow flashing. When three bars are always on, it means it's fully charged.
- The running time will vary with the different modes and levels used by each mother.
- Replace battery under professional helping. Do NOT remove built-in battery without authoriz

CLEANING AND STERILIZING ATION

Non-washable Part: main pump unit

The main pump unit should be kept dry. A clean, damp cloth can be used to clean the main pump unit. Please keep this area dry and clear of residue. Any other cleaning method will be classed as misuse under the warranty.

Washable Parts: breast shield, milk bottle, tee joint connector, silicone gas storage, duckbill silicone valve. Cleaning washable components that come into contact with milk prevents the build up of dry milk residue, which can cause the growth of bacteria that may be harmful to you and your baby.

AFter EACH USE

CLEAN WASHABLE COMPONENTS

1. Soak in a mild cleaning solution for 2 minutes and rinse under clean running water, or
2. Clean directly with drinking water or purified water, or
3. Place on the top shelf of the dishwasher


ONCE PER DAY

STERILIZE WASHABLE COMPONENTS

1. Boil in water for five minutes. Make sure the water is deep enough to completely cover the parts. In the process of boiling, pay attention to stirring accessories to avoid deformation which caused by the uneven heat.
2. We do not recommend microwave or steam sterilizing. Please be aware that some steam and microwave sterilizers reach high temperatures which may risk warping plastic parts.
3. We also do not recommend ultraviolet sterilization.

DRY COMPLETELY BEFORE ASSEMBLY

Bra adjustment buckle

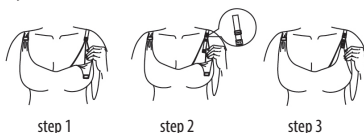1. Adjust the length of bra straps, make the pumping more comfortable.
2. One end of the bra adjustment buckle hung on the inside buckle of nursing bra.
3. The length of the bra strap can be adjusted by hanging the nursing bra buckle on the other end of the bra adjustment buckle.

SAFETY GUIDELINE

Failure to follow these instructions/safety

- Please use the Wearable Breast Pump under doctor's instruction while users have mastitis, inverted or flat nipple.
- Breast pump is personal product. Use by more than one person may cause a health risk.
- Use the breast pump out of reach of children.
- Do not use breast pump while sleeping or overly drowsy.
- Do not operate breast pump if it is physically damaged.
- Do not use breast pump with components of other brands for avoiding bad influence result.
- Do not leave breast pump unattended when plug into a power source.
- Do not reach for breast pump if it has fallen to water. Unplug immediately.
- Do not expose breast pump under direct sunlight.
- Repairs must be conducted by authorized service agencies.

MAINTENANCE & CARE GUIDE

The following problems may occur during use, please refer to the method provided for inspection and handling.

Troubleshooting	Cause Analysis	Processing Method
low or no suction	1. The silicone gas storage and pump unit are not installed in place. 2. Duckbill silicone valve is not installed in place or damaged.	1. When installing the pump into the silicone gas storage, it needs to be fully pressed down. 2. Replace a new duckbill silicone valve and install it in place.
milk back flow	If silicone gas storage is damaged or deformed, the milk will flow back into the pump unit. Please stop using it immediately.	Replace a new silicone gas storage.
Painful or unable to pump milk	1. Breast engorgement or clogged milk duct. 2. Inappropriate nipple size. 3. The pumping level used is too strong.	1. Ask a professional breast milk instructor to massage the breast before using the breast pump. 2. Choose a breast shield with correct size. 3. Start adjustment from the lower level, so that the breast have a process of adapting.

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