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**INSTRUCTION MANUAL**  
**EXPERT 2-Phase Electronic Breast Pump LOVI**

Please read this instruction manual carefully before using the device. Retain this instruction for later use.

**INTRODUCTION**

**Congratulations on your purchase of the EXPERT 2-Phase Electronic Breast Pump LOVI!**

Expert Breast Pump is a professional support for breastfeeding mothers who systematically express their milk, wanting to stimulate lactation, and solve problems of lactation. Development of innovative funnel was inspired by a real way in which the child lips and tongue are placed at the nipple - the child's mouth include breast, tongue and palate compress the nipple stimulating lactation and causing milk flow.

Expert Breast Pump enables painless, comfortable milk expression in a short time (even 200 ml during 20 min, or 100 ml during 10 min of one session). It is easy to use thanks to its compact dimensions and intuitive touch panel display.

Thanks to built-in Li-ion battery Expert Breast Pump can be used any place at any time. It doesn't require regular batteries (e.g. type AA) or permanent access to the electricity line. The built-in battery lasts for about 3 hours of continuous operation and can be recharged.

**Based on natural sucking rhythms, Expert Breast Pump is equipped with two expression cycles: stimulation phase and expression phase.**

Each of two phases can operate up to nine levels depending on the individual needs and preferences of the user.

**Stimulation phase** – imitates fast suckling moves that stimulate the production and milk let down.

**Expression phase** – reflects slower and deeper effective suckling. Under the influence of oxytocin milk flows and the child has to put an effort to suck it. Milk slowly becomes more dense and filling, and calorific - satisfies hunger.

**Range of rhythm and suckling strength of** Expert Breast Pump has been developed to reflect the natural suckling process and how the rhythm of sucking changes during breastfeeding.

**Using both phases, you will express more milk in less time.**

Additional elements included in the set, enables converting Expert Electronic Breast Pump LOVI into manual breast pump.

Breast milk is the healthiest and most valuable food for newborns and infants. It is recommended as a sole food given to your child in the first 6 months of its life. After six months, it should be combined with solid food.

**NOTE!** Do not use a breast pump in circumstances when a hungry baby fails to empty the breast itself, i.e. if there is any possibility that your milk ducts are blocked.

**In the case of physiological problems, consult your doctor immediately.**

## BASIC PRODUCT INFORMATION

### Breast Pump XN-2219MB/XH

Expert 2-Phase Electronic Breast Pump LOVI (later referred to in the manual as ,a breast pump'), is a product designed for breastfeeding mothers in order to express breast milk. It enables collection of milk for later usage i.e. in case of separation from the child. The breast pump helps maintain lactation and prepares for your planned absence or random events such as power outages, treatment, cracked and sore nipples.

- The device is intended for personal use. Using by more than one person may pose a health risk.
- The device warranty is 24 months from the date of sale. Please keep a proof of purchase.
- Before using, read the instruction manual and follow the rules of safe use.
- Keep the manual together with the device.
- The expected life of the product is 5 years. The service life of the silicone elements is no longer than 2 years. Silicone elements may change colour permanently. These elements must be replaced after finding discoloration or after the estimated time of use.

**FOLLOW THE RECOMMENDATIONS ON THE CORRECT USE OF THE DEVICE AND ITS ACCESSORIES. THE MANUFACTURER IS NOT LIABLE FOR DAMAGES RESULTING FROM THE MISUSE OF THE DEVICE, THE RESULTING DAMAGE IN THIS WAY CANNOT BE THE BASIS FOR THE CLAIM OR WARRANTY.**

**Making any repairs or modifications to the product by unauthorized persons, results in loss of warranty and may result in risks to health and life.**

## SAFETY RULES

When using electrical devices, special safety precautions must be taken including:

- **CAUTION!** To protect against electrical shock, do not immerse the cord, AC adapter or device's control panel in water or other liquid. Protect the device from water and NEVER operate the device with wet hands.
- Before plugging in the device for the first time ensure the voltage is compatible with your electricity supply by checking the nameplate on the base of the main body. The correct voltage for LOVI Breast Pump is 100-240V.
- Connect the device only to an efficient and surge-protected power supply.
- Extension cords may be used if during their use they are adequately protected and are under appropriate supervision.
- If extension cords are used:
  1. The marked electrical voltage of the extension cord should be compatible with the electrical voltage specified on the device.
  2. The extension cord must be adapted to the device's electrical load, according to the information provided on the nameplate.
  3. The extension cord must bear the CE marking.
  4. The extension cord cannot bear signs of any damage.
  5. The cord should be arranged so that it does not drape over the countertop or table top where it can be pulled by children or tripped over unintentionally.

- If the device has a grounding plug, the extension cord must be a 3-wire cord type with a grounding pin.
- Do not leave the device unattended when it is plugged in.
- When removing the plug from the socket hold its periphery. Never remove the plug from the socket by pulling on the cord.
- Do not place the power cord under furniture or other obstacles.
- DO NOT place on / near a hot gas or electric burner or in a heated oven. Make sure that power cords are away from hot surfaces, radiators and central heating pipes. Do not leave the device in direct sunlight.
- Always remove the plug from the socket before cleaning the device.
- **WARNING!** Do not operate any device with a damaged cord or plug, after the device malfunctions or if it has been damaged in any manner.
- Never use an incomplete / deficient appliance. Damaged or worn elements must be replaced immediately to those provided by the manufacturer.
- Observe the correct use of the appliance itself, as well as various accessories of the breast pump.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Close supervision is necessary when any appliance is used by a disabled person or near children.
- ALWAYS keep the appliance out of the reach of children when not in use.
- ALWAYS unplug the breast pump from electric socket when not in use.
- Do not use a breast pump AC adapter to power other devices, as well as do not connect breast pump to other AC adapter than the one provided by the manufacturer.
- If the power adapter is damaged, it must be replaced by another, purchased from the manufacturer of the device, in order to avoid danger.
- All repairs should be performed by the manufacturer or distributor service only.
- Making any repairs or modifications to the product by unauthorized persons, results in loss of warranty and might cause danger to life and health.
- The manufacturer reserves the right to change the design of the product.

**WARNING! FAILURE TO FOLLOW THE INSTRUCTIONS IN THIS MANUAL COULD CAUSE SERIOUS PERSONAL INJURY, LOSS OF LIFE, OR DAMAGE TO THIS EQUIPMENT.**

1. The device should only be used in accordance with its intended use as described in this manual. Other uses violate the terms of the warranty.
2. Check the device before each use. Do not use if the device is damaged in any way. Instead, contact the manufacturer or distributor service.
3. Never use a product that has been dropped or has fallen on a hard surface. In this case, the product must be returned to service in order to be checked.
4. It is forbidden to use the device while taking a bath.
5. It is forbidden to use the product in locations exposed to contact with water or other liquids, e.g. near bathtubs, wash bowls, kitchen sink or shower enclosures.
6. DO NOT reach for the unit if it fell into the water or other liquid. Immediately disconnect the unit from the electrical outlet.
7. It is forbidden to use the device during sleep or in a state of excessive fatigue.
8. It is forbidden to dip the control panel and power supply equipment in the water, or clean with a wet sponge or wet material.
9. Improper use can destroy the device.
10. Battery charging time should not exceed 6 hours to extend battery life.
11. The fact that after 6 months of use, the working time of battery will be shorter is normal. This is due to the characteristics of the battery.
12. Use only indoors. DO NOT use outdoors.
13. The device is designed for domestic use only.
14. If the appliance has been transported at low temperatures, it should be acclimatized before use to the ambient temperature.
15. Before turning on the device, make sure it has been assembled properly.
16. To stop the pump during the cycle, touch the ON/OFF button again and disconnect the pump from the mains.
17. DO NOT use any solvents or chemicals to clean the product.
18. The product is not a toy. Use and store the pump and all its parts out of the reach of children.

19. Do not disassemble the product.
20. Li-ion battery is built into the device. DO NOT remove it by yourself.
21. DO NOT use a power supply or batteries that have not been attached to the device.
22. It is forbidden to redesign and modify the product. Any action contrary to this guideline invalidate the WEEE declaration of conformity.
23. If you have any doubt about the safety of the product, please contact with distributor or manufacturer.
24. The information contained in this manual may be subject to appropriate changes on the technical side.

This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. The requirements of the WEEE directive 2012/19/EU are fulfilled.

The manufacturer declares that the product complies with European Union directives and harmonized standards. It has been marked by conformity assessment procedures.

The product meets the requirements of the directive for equipment intended for use in certain voltage limits (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30 / EU, directive RoHS 2011/65 / EU.

The product is made in the second class protection.

This product is not a toy. Keep all parts away from children. Retain the instruction for later use. Retain the instruction for later use.

TECHNICAL DESCRIPTION	
Battery operation	Li-ion battery
Power operation	100-240 V / 50/60 Hz / 0,35A max; outlet: = 10V / 1,5A
Maximum power	84 W
Weight with AC adaptor	1100 g

**USEFUL HINTS**

**Important safety information:**

- Never use a breast pump during pregnancy because it may start a labour pains.
- If you experience problems while breastfeeding, or feeding causes pain, it is necessary to seek medical advice, before using the breast pump.
- The first use of the breast pump is best done under the supervision of specialists.
- It is recommended to read specialist literature and get to know the principles of baby feeding.

Before use, make sure that the pump has been disassembled, washed and disinfected. Disassemble, wash and disinfect the breast pump components which come in contact with mother's milk, always before and after each use of the breast pump.

**DO NOT WASH OR DISINFECT CONTROL PANEL OR AC ADAPTER**

- Ensure the overall purity of the product. Prevent the build-up of dust and other contaminants that may adversely affect the work of the product.
- Taking special care of hygiene is important to ensure the safety of mother and child.
- Elements of the device not intended for cleaning must be protected against water and moisture.

**NOTE!** *Insufficient suction may result in a smaller amount of milk, but too high suction can cause soreness and reduce production of milk. If you notice this type of failure, you should adjust the suction power (see SUCTION LEVEL).*

**When to use a breast pump?**

If possible, wait until lactation stabilizes, which usually occurs within 2-4 weeks after birth. Exceptions to this rule:

- If your breasts produce too much milk, they become swollen and hard (engorged). Expressing a small amount of milk before or during a feed can give you relief and help your baby to grasp the nipple.
- If you have inverted nipples (which may occur because of long intervals between feeds.)
- If you have a concave, flat or damaged nipples (using a breast pump can improve nipple shape).
- If you feel overflow between breastfeeds and do not want to wake the baby.
- The baby is sick and does not want to suckle.
- If your breasts produce too little milk, using the pump between feeds will help increase milk production.
- When you are working or away from home and want to continue breastfeeding, you should regularly express milk using a breast pump to sustain the natural process of lactation.

**How often should you express milk?**

If mother temporarily cannot breastfeed it is recommended to express milk using breast pump at least 8 times for every 24 hours. This will keep the lactation at the appropriate level. In the beginning you may experience difficulty in expressing larger amount of milk, but after a few days you will be expressing more.

Also during stabilized lactation, possibility of single expression is smaller (30-60 ml), that is why collecting one portion of milk may require several sessions.

**Pic. 1 – SET PARTS**

1. Pump unit
2. AC adapter
3. Tubing for connecting breast pump and pump unit
  - a. Tubing
  - b. Tubing connector
4. Expression collection combo kit
5. Milk storage kit
  - a. Hood/cup with scale
  - b. Screw cap
  - c. Teat
  - d. Sealing disc
6. Manual pumping parts
  - a. Inner cap stopper
  - b. Silicone bladder
  - c. Handle holder
  - d. Handle
7. Spare parts
  - a. Cap plug – 1 pc
  - b. Tubing connector – 1 pc
  - c. Tubing – 1 pc
  - d. Valve membrane – 2 pcs.
8. Storing case
9. Disposable Breast Pads „Discreet Elegance“ – 10 pcs.

**Pic. 2 – EXPRESSION COLLECTION COMBO KIT**

- A) Bladder Cap
- B) Cap Plug
- C) Silicone bladder (for electronic breast pump)
- D) Dust cover
- E) Silicone massaging cushion
- F) Plastic brace
- G) Breast pump body
- H) Valve body
- I) Valve membrane
- J) Bottle converter
- K) Bottle
- L) Bottle stand

**Pic. 3 – PUMP UNIT**

1. LED display
2. Vacuum increase button
3. Vacuum decrease button
4. Phase changing button
5. ON/OFF button
6. Tubing connector inlet
7. AC adapter inlet
8. ON/OFF switch

**Pic. 4 – LED DISPLAY**

1. Battery power indicator
2. Stimulation phase indicator
3. Expression phase indicator
4. Suction level indicator
5. Time indicator

## CLEANING AND DISINFECTION INSTRUCTIONS

Before first and each next use disassemble the breast pump. Wash all components of breast pump and bottle in warm water with mild detergent, then rinse thoroughly and disinfect in accordance to rules below.

Before first use, disinfect all components which come in contact with mother's milk (silicone massaging cushion (fig. 2E), breast pump body (fig. 2G), valve body (fig. 2H), valve membrane (fig. 2I), silicone bladder (fig. 2C; 1.6b), inner cap stopper (fig. 1.6a), bottle converter (fig. 2J), bottle (fig. 2K) by placing them for 5 minutes in boiling water. This is to ensure hygiene.

**NOTE!** Do not leave components on the bottom of the pot. Stir the water during disinfecting to prevent damage to components.

- Be sure that all components are completely submerged in boiling water during disinfection.
- **Do not boil silicone tubing with connectors (fig. 1.3), plastic brace (fig. 2F), bladder cap (fig. 2A) and electric elements (fig. 1.1; 1.2).**
- Remove all components from water immediately after boiling.

Before subsequent use, disinfect all above mentioned elements of breast pump and bottle by pouring with boiling water or placing in steam sterilizer. This is to ensure hygiene.

**WARNING!** There are no elements that can be disinfected in **microwave sterilizer**.

- Do not pour water on bladder cap (fig. 2A). If you will notice water on or in the cap, pull out the cup plug (fig. 2b) and leave both elements to dry, so that water won't enter the tubing and the pump unit.
- **WARNING!** Do not disinfect silicone tubing with connectors (fig. 1.3), plastic brace (fig. 2F), bladder cap (fig. 2A) and electric elements (fig. 1.1; 1.2). Do not submerge these elements in water. Wipe with damp cloth.

After each disinfection completely air- or towel-dry before reassembly and next use. Dried parts should be stored in an airtight container or a plastic bag. You can also use a clean towel.

**WARNING!** Partially dry components may affect the operation of your Electronic Breast Pump. Make sure that all parts are completely dry before use.

### Washing instructions:

- Always wash the product immediately after use to remove any residual milk.
- Disconnect the tubing with connectors. These elements don't need cleaning, because the silicone bladder protects them from contact with feed.
- **Clean all components of the breast pump in accordance with the instructions mentioned above.**
- DO NOT use any solvents or chemicals to clean the product.
- Rinse with clean, hot water.
- If you use electric sterilizer for disinfecting elements please follow the instructions given by its manufacturer.

**WARNING!** Always disassemble valve membrane (fig. 2I) by catching it near the "hinge" with two fingers as close as possible. Never pull the edges of the membrane, because it can damage it and in consequence weaken the suction strength.

**WARNING!** Before each use, check the condition of all elements for cracks, tears, discoloration or deterioration, as they may affect the effectiveness of the device.


### How to disinfect the teat?

Before first and subsequent use wash the teat with warm water then place it in boiling water for 5 min. This is to ensure hygiene. Clean before each use. Do not use any antibacterial agents! Inspect carefully before each use especially when baby has teeth. Pull the teat in all directions. Throw away at the first signs of damage or weakness. Do not tighten up bottle locking ring, as it may disturb its ventilation. Always check for leakage. Never enlarge the teat flow holes by any means, as it may damage the teat. Do not leave the teat in a direct sunlight, close to a source of heat, or in disinfectants (sterilizing solution) longer than recommended, as this may weaken the teat. It is recommended to replace a teat max. after 2 months of use to ensure safety and hygiene.

After use, clean the bottle and the teat from food remains, rinse and then wash thoroughly with hot water; do not boil. The bottle is dishwasher safe (max. temp.65°C). Do not wash the teat in the dishwasher because detergents / cleaning agents can cause damage to the product.

**For your child safety and health WARNING!** Always use this product under adult supervision. Never use feeding teats as soothers. Continuous and prolonged suckling of fluids may cause tooth decay. Always check food temperature before feeding. Warming in microwave oven may result in locally high temperature. If warming feed in microwave, oven remember to leave the bottle open. Take particular care when heating in a microwave. Always stir heated feed to ensure even distribution of temperature and check liquid temperature before giving it to the child. Warming breast milk in microwave is not recommended. This product is not a toy. Keep all components out of the reach of children when not in use.


### ASSEMBLING INSTRUCTIONS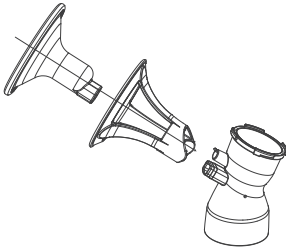

Place silicone massaging cushion (fig. 2E) in the plastic brace (fig. 2F). Push the narrow part of the cushion to the Narrow Neck of plastic brace. Then place connected elements on the breast pump body (fig. 2G)

**Important!** Make sure that the edges of cushion fit evenly to the edges of plastic brace. Take a note that the both elements are profiled and should be properly matched to each other.

Silicone cushion can be placed in the plastic funnel fast and easily after placing it on dust cover (fig. 2D) which should be taken off after assembling.

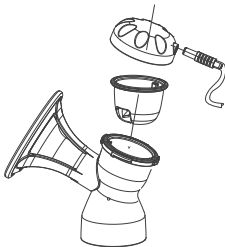
Place silicone bladder (fig. 2C) in the goblet of breast pump body. Make sure that bladder cap (fig. 2A) has the cap plug (fig. 2B). Then place the bladder cap on the breast pump body and rotate in a clockwise direction in such a way that the hole for the tubing connector will be located on the opposite side of the funnel.


Place the valve membrane (fig. 2I) in the valve opening (fig. 2H). Then place valve with the membrane inside the breast pump body (fig. 2G).


**Important!** Make sure that membrane fits tight to openings of the valve and all components are interconnected.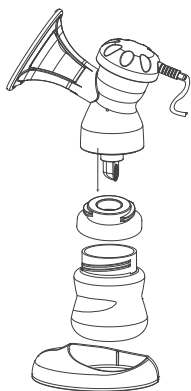

Place bottle converter (fig. 2J) on the bottle (fig. 2K). Screw this set into breast pump body (fig. 2G). A complete set can be placed in the bottle stand. (fig. 2L).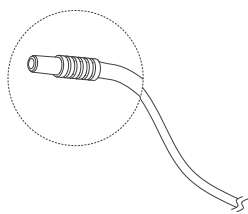

Connect the tubing (fig. 1.3a) with tubing connectors (fig. 1.3b) by placing their thinner ends in the tubing. Note that elements should tightly fit to each other.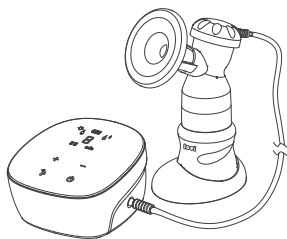

Connect tubing set with breast pump (fig. 1.4) and pump unit (fig. 1.1) Put one end in the opening of bladder cap (fig. 2A) and the second in the port (fig. 3.6) placed on the control panel. Press connectors to ensure that they fit in the openings tightly.

## POWER OPTIONS

### Using your AC adapter (included)

Place AC adapter ending (fig. 1.2) in the port placed on the control panel (fig. 3.) Plug the AC adapter into an electrical outlet.


Electrical safety can only be achieved when using the AC adaptor which was provided with the pump. Use of any another adaptors causes concern for the safety of the product and/or user and voids the warranty.

**NOTE!** AC adapter pins may vary depending on the country of destination.

### Using Li-ion battery

Breast pump can be powered by the built-in Li-ion battery. Charging the battery is done by using the AC/DC adapter (fig. 1.2)

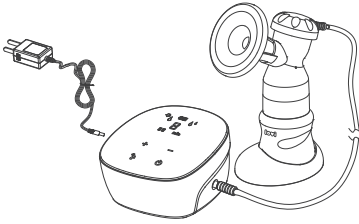**WARNING!** Before first use, fully charge the battery.

When pump unit is powered by the battery, the display shows current charge status . After connecting the power supply charging starts, and current battery charge status is displayed in the dynamic form. When the battery runs low you will hear a beep and the icon  will flash.

**It is recommended to carry out the process of charging / trickle charging before the process of pumping.**

AC adapter (fig. 1.2) connected with the pump unit (fig. 1.1) will charge the battery, no matter if the switch (fig. 3.8) on the basis of pump unit is set to ON or OFF. When the switch is set to ON, and the AC adapter is not connected to pump unit, it will automatically work on the built-in battery.

To recharge the battery, connect the AC adapter pump unit when it is not in use. Charging a completely discharged battery pack takes approx. 4 hours. When the battery is fully charged, it will discontinue charging to avoid damage. In order to extend the life of equipment and reduce electricity consumption by the battery, do not charge the battery for more than 6 hours.

The battery in pump unit can be recharged approx. 500 times. One full charging of the battery allows the pump to operate for approx. 3 hours. When the battery is low you will hear a beep, and the control panel will turn off automatically within next 10 seconds.

If the breast pump is not in use, put the switch (fig. 3.8) to OFF position. This will help to save the battery power and extend its life. If the device is not used for a long time, recharge the battery once a month.

## EFFECTIVE MILK EXPRESSION WITH EXPERT ELECTRONIC BREAST PUMP

1. Before expressing milk wash your hands thoroughly and make sure the pump is clean and disinfected (remember its parts should not be hot or wet after scalding) and assembled according to the instructions. Check all the parts for correct fixing.
2. To improve effectiveness of milk expression use the breast pump between feedings, and not immediately after feeding.
3. Before expressing shower your breasts with **warm water** - this will stimulate milk flow. Gentle breast massage toward the nipple or wrapping the breast with warm towel may also be helpful.
4. Sit comfortably with good back support and relax. Place the baby's photo in front of you or express milk in his presence - eye contact with your child stimulates lactation. Do not express milk when you are stressed or tired, as this inhibits milk flow.
5. Place the nipple deep into the funnel and press the breast massage pad so that it tightly covers the breast, preventing air outflow. Otherwise milk expression is ineffective.
6. Pumping should create vacuum, necessary for proper breast pump performance.
7. Do not try to move away from the main part of the pump at a time when the device produces a vacuum. Turn the machine off, place your finger between the breast and silicone massaging cushion, unhook the funnel of the breast and move the pump.
8. You should not feel the pain while expressing the milk. If you feel discomfort, reduce the suction level or turn the pump off. Then carefully check the breast. If you see a red or hot areas of it, contact your midwife or doctor.
9. To increase the speed of the flow during expressing **bend slightly forward** (when the breast is well filled, milk drops may often flow out in this position even without compressing the areola). However, remember to hold the bottle in an upright position, and make sure you are sitting comfortably.
10. In the beginning express milk from each breast for only a few minutes. Do not expect to immediately fill the bottle. As you get used to expressing the amount of milk will increase.
11. When the milk stops flowing, put the breast pump to the other breast.
12. Remember that you need about 10 minutes to express 60/120 ml of milk. The amount of milk expressed at different times can vary.
13. When you finish expressing, remove the breast pump body, disassemble the device, wash its parts in warm water with mild detergent, and rinse thoroughly. Dry all parts thoroughly on a cloth or on a bottle and teat drainer.
14. You can feed your child immediately with the expressed milk after screwing on the cap with the teat included in the package.

GB

## PUMPING INSTRUCTIONS


**WARNING!** Be very careful during the assembly and disassembly of the pump to avoid damaging the components.

### Before each use:

- Wash your hands thoroughly before using the pump.
- Ensure that the pump is pre-washed and dried, if not, follow the instructions located in section "CLEANING AND DISINFECTION INSTRUCTIONS".
- Ensure that there is no liquid or moisture in the silicone tubing. Never operate the pump if you notice any water in the silicone tube.


**NOTE!** Do not mouth blow the silicone tube in order to remove water.

**WARNING!** Before expressing check the charge status of the battery. If the battery is running low, connect the AC adapter (fig 1.2) to the port on the pump unit (fig. 3.7) and let the battery recharge. Remember, that expressing during charging the battery is not recommended.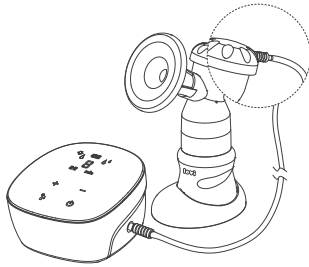


Connect tubing set with breast pump (fig. 1.4) and pump unit (fig. 1.1). Put one end in the opening of bladder cap (fig. 2A) and the second in the port (fig. 3.6) placed on the control panel. Press connectors to ensure that they fit in the openings tightly.

**WARNING!** Ensure that there is no liquid or moisture in the silicone tubing. (eg. water remaining after the last wash).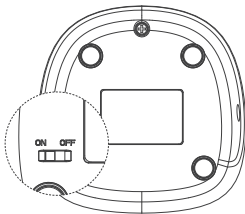


If the battery is charged, put the switch (fig. 3.8) from OFF to ON position and start the process of milk expression.

**WARNING!** Remember, if device is not used for a longer time, set the switch to the OFF position to save battery life.

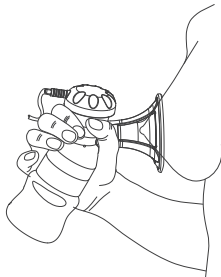Hold the expression collection combo kit to your breast so that your nipple(s) are centred inside the breast shield.

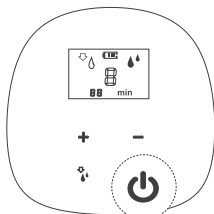
**NOTE!** It is important that the silicone cushion clings tightly to the breast, otherwise the suction may be reduced.


Pump unit is equipped with a touch panel.

Touch the ON/OFF icon  to start expressing process. You will hear a beep sound (it will occur during every changing of settings on the control panel).

At the same time pump unit starts 30 min countdown. When the timer reaches 0 minutes, the engine switches off and goes into standby mode with the backlight turned off (the display of battery status is not visible). If you touch ON/OFF icon  during expressing, the pump will turn off. Second touch of the  icon, will start counting down again.

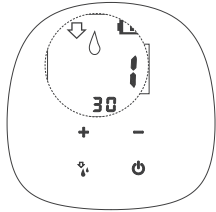
After turning on the breast pump will start working on stimulation phase and suction strength level „1“.

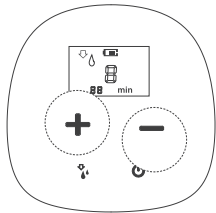

To adjust the suction strength to your needs, regulate it's level in stimulation phase by touching icons of (+) or (-).


There are 9 levels of suction strength available in stimulation phase.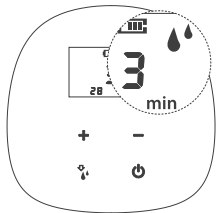


After 2 minutes of stimulation phase, pumping unit will automatically switch to expression phase on suction level "3" regardless of the suction strength set on during stimulation phase. The following icon  will appear on the display.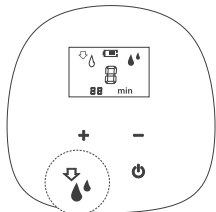

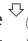If you prefer to skip the stimulation phase  or when the milk begins to flow earlier than two minutes from the start, tap the icon  to change the phase. Expression phase will automatically start at suction level "3".

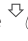
You can return from expression phase  to stimulation phase  by touching the same icon on the pumping unit.

*Regardless of the current suction strength, after switching phases, the pumping unit will always automatically adjust the suction strength level „1“ in the stimulation phase and level „3“ in the expression phase.*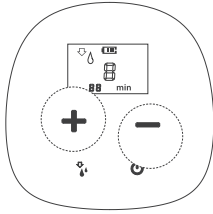

To adjust the suction strength to your needs, you can regulate it's level in expression phase by touching icons of (+) or (-).

There are 9 levels of suction strength available in expression phase.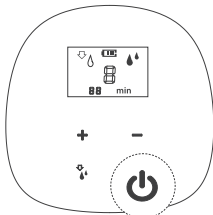

If you need to stop expressing, simply touch ON/OFF icon and turn off the pumping unit.

**WARNING!** Make sure there is no suction on the nipple(s) before removing the breast shield. If necessary, break the suction by inserting a finger between the breast shield and the breast, similar to the way you break away from your baby's suction.

**NOTE!** The breast pump turns off automatically after 30 minutes of pumping.

## SUCTION LEVEL

Whenever you select a suction level (in both stimulation & expression cycle), your Expert Electronic Breast Pump remembers it until you change it next time.

When you finish expressing after switching the pump ON again it will start to work at the last memorized phase and suction level (on the condition that the switch placed on the basis of pumping unit (fig 3.8) will not be set to OFF. In this case, if you switch to ON position the pump will start to run from the default settings).

If the breast pump stops, after turning on again it will start from last memorized phase and suction level

### To set your preferred suction level

1. In stimulation phase, adjust the suction to the level you find most comfortable by touching the icons up (+) or down (-).
2. In deep expression phase, adjust the suction level to the highest effective, yet still comfortable, level by touching the icons up (+) or down (-).

**If milk gets under the silicone massaging cushion, reduce the suction level so that milk keeps up flowing into the bottle.**


## MANUAL EXPRESSION OF MILK

A change to Manual Breast Pump


Set contains of elements allowing normal expression of milk. With the help of these elements Electronic Breast Pump easily changes into Manual Breast Pump.

To assemble and properly use Manual Breast Pump read the instructions below.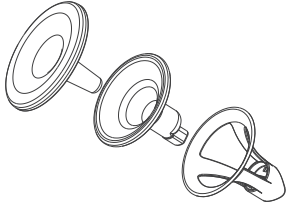


Place silicone massaging cushion (fig. 2E) in plastic brace (fig. 2F). Push the narrow part of the cushion to the narrow neck of plastic brace. Then place connected elements on the breast pump body (fig. 2G).

**Important!** Make sure that the edges of cushion fit evenly to the edges of plastic brace. Take a note that both elements are profiled and should be properly matched with each other. Silicone cushion can be placed fast and easily in the plastic funnel after placing it on dust cover (fig. 2D) which should be taken off after assembling.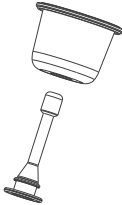


Holding manual silicone bladder (fig. 1.6b) in one hand (concave side down), put the inner cap stopper (fig. 1.6a) through the bladder so that the wide end of a stopper becomes the bladder base.

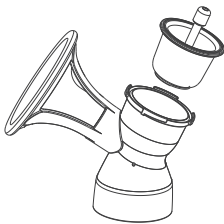
Place the bladder with stopper inside the goblet of breast pump body (fig. 2G). Pushing the edges of the bladder, remove any irregularities. Make sure that the bladder adheres tightly to the edge of the goblet.

Then place the handle holder (fig 1.6c) on the breast pump body (fig. 2G) and rotate in a clockwise direction in such a way that the hinge for the handle will be located on the opposite side of the funnel.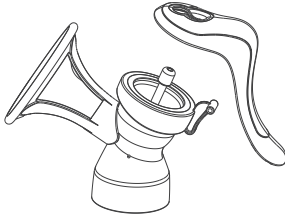

Put the handle (fig. 1.6d) on the hinge so that the inner cap stopper (fig. 1.6a) will go through the opening placed on the top side of the handle. Then push to the hollow until you hear a click sound.

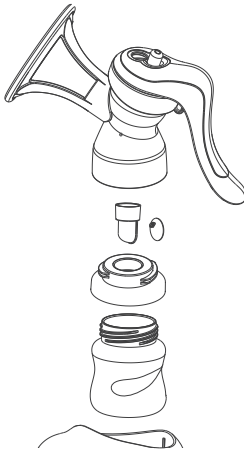Place the valve membrane (fig. 2I) in the valve opening (fig. 2H). Then place valve with the membrane inside the breast pump body (fig. 2G).

**Important!** Make sure that membrane fits tight to the openings of valve and all components are interconnected. Place bottle converter (fig. 2J) on the bottle (fig. 2K). Screw this set into breast pump body (fig. 2G). A complete set can be placed in the bottle stand. (fig. 2L).

To disassemble the breast pump follow the steps of the above assembly instruction in reverse order.

## MILK STORAGE

- Due to the active biochemical and microbiological processes occurring in the milk, it is important to maintain a constant temperature of storage.
- Milk can be stored in the bottle, or in the breast milk storage bags.
- Mark the date of each container with expressed breast food.
- Freeze only fresh milk. Do not fill the container completely as frozen breast milk increases its volume.
- Breast milk can be heated in the same way as any other feed (but never in the microwave as this may destroy valuable nutrients and antibodies).
- Frozen breast milk can be defrosted overnight in the refrigerator then warmed gradually in bottle warmer or hot water.
- Shake the container to combine separated fat molecules with the remaining volume of milk.
- DO NOT store milk on the shelves placed on the refrigerator door.
- NEVER refreeze defrosted milk.
- DO NOT add fresh breast milk to already defrosted milk.
- DO NOT add fresh breast milk to already frozen milk for storage.

- NEVER heat breast milk by boiling or putting it in the microwave.
- ALWAYS check milk temperature before feeding.
- Throw away any left-over breast milk after the feed.

Milk status	Temperature	Recommended time of storage
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum 24 hours
	+19°C to +25°C (room temperature)	optimally up to do 4 hours
	from +25°C to +37°C	maximum up to 4 hours
Frozen milk	-20°C	optimally 3-6 months
Thawed milk	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	maximum up to 4 hours

The Breast Pump is a personal equipment. It can't be exchanged or refunded and it is covered by a manufacturer's warranty. Remember that the basis of the complaint must be accompanied by proof of purchase.

EACH MODEL OF THE BREAST PUMP IS TESTED IN TERMS OF USABILITY BEFORE INTRODUCING IT TO THE MARKET.

**Please remember that if you experience lactation problems resulting in your child inability to empty the breast, the breast pump may also prove ineffective.**

**WARNING!**

- Do not use this product for purposes other than those listed in the attached instruction.
- Do not place the product near sources of fire.
- Before using the pump, make sure that all parts were assembled correctly.
- Immediately stop using the breast pump if you feel breast or nipple discomfort.
- Wash your hands before using the product.
- Keep the product out of the reach of children.
- Do not drop the product to avoid any damage.
- To avoid burns do not touch the product elements with bare hands immediately after sterilization in boiling water.
- Do not resale your device.

**MANUFACTURER DECLARATION**

The manufacturer declares with full responsibility that the product complies with the essential requirements of the New Approach Directives, leading to CE marking. For confirmation the declaration of conformity is issued.

## TROUBLESHOOTING

While using the breast pump you may encounter small difficulties. Before you decide to place the claim, try to solve these minor problems by yourself. Here are some useful tips.

### Decreased suction:

If it feels like the suction has decreased, try the following:

- Detach the tubing and tubing adapter from the back of your Electronic Breast Pump and place a finger over the pump inlet. If you do feel suction, the electric pump base is operating properly. If you do NOT feel suction, try unplugging the AC adapter from the electrical outlet. If problem persists, change electrical outlets.
- Make sure the expression collection combo kit is assembled correctly.
- Check if the valve membrane is placed firmly, flat in valve and is not damaged. This element is crucial for achieving proper suction strength. If this element is damaged, replace it with a new one. Spare membranes are attached to the kit.
- Check, if the cap plug was correctly placed in the bladder cap.
- Check, if the silicone tubing was not bent and that there are no impurities inside.
- Check, if the breast tightly adheres to silicone massaging cushion.

### Breast milk build up in the breast shield body:

- If breast milk doesn't appear to be draining fast enough into the collection bottle, reduce the suction setting on your Electronic Breast Pump to allow the milk to drain before gradually increasing.
- Make sure that you keep breast pump properly. Bottle should be placed in an upright position.

### Reverse of milk:

- If the milk or water is in the silicone bladder or tubing, clean it according to the instructions. Stick to the principle of „dry completely before use“.
- Check if silicone bladder is not damaged.

### Power problems:

If there is a power problem during charging / using a breast pump, follow these steps:

#### The battery does not charge

- Check if the AC adapter is correctly connect with pumping unit.
- Change electrical outlets.

#### You cannot run pumping unit

- Check if the switch on the base is in the ON position.
- Check the battery charge level.

### LED display

- If E1 or E2 appear on the LED display, it means there is an open or short circuit in the internal battery. Please contact with sales representative or place of purchase. Do not use the device.

### Radio frequency energy:

This equipment generates, uses, and can radiate radio frequency energy. If not installed and used in accordance with the instructions, may cause harmful interference with other devices nearby. Try the following:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a different circuit from the one that the other device(s) are connected.
- Consult the manufacturer or a field service technician for help.
- There is no guarantee that interference will not occur. A device meeting the requirements of the electromagnetic compatibility, always generates minimal disruption in the normal range. There are no devices which do not produce distortion.

## DECLARATION OF CONFORMITY EU No. 7/2016

In scope of directives: 2014/35/EU, 2014/30/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o. SKA  
Puławska Street 430, 02-884 Warsaw, Poland

We hereby declare under our responsibility that our breast pump with adaptor (cat. no 50/000)

**brand: Lovi, breast pump - model: XN-2219MB/XH, adaptor type: GEO151T-1015**

is in accordance with the following directives:

- directive relating to electrical equipment designed for use within certain voltage limits  
2014/35/EU
- directive relating to electromagnetic compatibility 2014/30/EU
  - directive ErP 2009/125/EC
  - directive RoHS 2011/65/EU

and is in accordance with the following harmonized standards:

- EN 60335-1:2012 (PN-EN 60335-1:2012)
- EN 55014-1:2006 (PN-EN 55014-1:2012)
- EN 55014-2:2015 (PN-EN 55014-2:2015-06)
- EN 61000-3-2:2014 (PN-EN 61000-3-2:2014-10)
- EN 61000-3-3:2013 (PN-EN 61000-3-3:2013-10)
- EN 55024:2010 (PN-EN 55024:2011)
- EN 62233:2008 (PN-EN 62233:2008)
- EN 61558-2-16:2009 (PN-EN 61558-2-16:2010)
- EN 60529:1991 (PN-EN 60529:2003)
- EN 60950-1:2008 (PN-EN 60950-1:2007)

as well as other technician specifications:

The Commission Regulation (EC) No. 278/2009

The last two digits of the year in which the **CE** marking was affixed: 15

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Warsaw, 16.12.2016

.....  
(place and date of issue)

.....  
(name, surname and signature of authorized person, position)

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