## INSTRUCTIONS FOR USE

### ELECTRONIC NASAL ASPIRATOR MODEL NO. DQ22

Please read this manual carefully before using the device. Retain for future reference.

#### NOTE

1. Make sure that the device and head are well assembled before use.
2. When any abnormality occurs, turn off the device.
3. Read the manual carefully before the first use.
4. Important!
  - Keep out of the reach of children when not in use.
  - Do not store near heat sources and do not expose to extreme temperature, high humidity, or dust.
  - Avoid storing with chemicals or flammable gases.
5. If the device is damaged, immediately contact the manufacturer.
6. Do not change device settings.
7. Maintenance:
  - Check the device and all its accessories regularly.
  - Avoid long-term use. Check the device before each use.
8. Don't insert the tip into nostrils too deep.
9. Never use on ears, mouth, or eyes.
10. **For non-commercial home use only.** This nasal aspirator is for removing nasal secretions only. Do not use this device other than for its intended purposes. The manufacturer is not responsible for product misuse.
11. **For charging, use the adapter included.**

#### PARTS OF THE DEVICE, FIG. 1

- |                      |                           |                         |
|----------------------|---------------------------|-------------------------|
| 1. Connector         | 7. Collection cup         | 12. USB cable           |
| 2. ON/OFF button     | 8. Narrow tip             | 13. Adaptor             |
| 3. Mode button       | 9. Large tip              | 14. Cleaning brush      |
| 4. Mode indicator    | 10. Storage chamber cover | 15. Silicone drain pipe |
| 5. Battery indicator | 11. Headpiece             | 16. Charging port       |
| 6. Plastic cover     |                           |                         |

#### BEFORE FIRST USE

Check the aspirator for any obvious damage and missing parts. If the device is damaged or some parts are missing, contact the device manufacturer.

- A low battery charge is indicated by the battery indicator (5). If the indicator lights red, connect the device to the power supply.
- Connect the USB cable included in the set with an original 5V/1A adapter.
- Plug the USB cable into the device port.
- Plug the adapter into a 230V AC outlet. Do not use the device during charging.
- Once the device is fully charged, the battery indicator (5) will light green.
- The device turns off automatically after 2 minutes from the start.

#### ASSEMBLY INSTRUCTIONS, FIG. 2

1. Join the tip (8) with the collection cup (7). See fig. 2a.
2. Put the non-return cover (10) on the headpiece (11). See fig. 2b.
3. Tighten by turning it clockwise. See fig. 2c.
4. Insert the suction head (2c) into the connector of the nasal aspirator.
5. Join the head with the base. See fig. 2d.

Choose a suitable tip.

1. The narrow tip (8) is recommended for use when the mucus is loose and in a liquid form.
2. The large tip (9) is recommended for use when the mucus is in a thicker, more solid form.

Insert the assembled suction head into the connector, and rotate it clockwise to secure it in the base. Make sure the head is connected correctly.

## CLEANING

### Manual cleaning

Disassemble all parts. Clean all head elements in warm water with mild detergent and rinse thoroughly after each use. Use a cleaning brush for better effects. Silicone elements (fig. 1, parts 8 and 9) can be scalded.

**Warning!** After each use, be sure to disassemble all head parts for cleaning. The device is not waterproof. Never immerse the base in liquid. Clean with a cloth.

### Automatic cleaning

Assemble the nasal aspirator following the assembly instructions. Remove the plastic cover (6). Then connect the silicone drain pipe (15). Prepare clean water in a vessel. Place the end of the pipe in a sink. Press the ON/OFF button (2) to start the cleaning process (fig. 3). To ensure hygiene, clean after each use. **Do not clean the device while it is plugged in!**

## HOW TO USE

Use the nasal aspirator in a comfortable position. See fig 4.

### NOTE:

Choose one of three different suction strengths by pressing the mode button. Check the suction strength on the mode indicator: low – 1 light, mid – 2 lights, high – 3 lights. Adjust suction strength to your needs.

**WARNING!!!** Lay your baby on the back or try to hold its head during usage. Don't insert the tip into nostrils too deep. Never use on ears, mouth, or eyes.

How to remove mucus more efficiently? Press the ON/OFF button to turn the device on. Insert the suction tip into a nostril. Gently move the tip in small circles to remove mucus more effectively. We recommend using saline solution to soften dried nasal mucus. Spray the saline solution into the baby's nostrils. Wait for 3-4 minutes. Then suck the mucus away with the aspirator. Low pressure is formed if air cannot get into the nostril, so it's easier to suck out the nasal mucus if the suction tip is close to the nasal cavity.

Always track the collection cup and headpiece during use. If there is too much mucus, it may get into the suction port, which may cause malfunction of the device. If you notice that there is a lot of mucus in the collection cup or headpiece, turn off the device immediately and clean it. Use a cotton swab to dry the suction port and remove mucus. If the suction port has sucked a lot of mucus, shake it out or blow it out with a hairdryer. Remove the headpiece before cleaning.

### Comments

The nasal aspirator generates a vacuum to remove nasal secretion. Frequent nasal congestion and runny nose caused by infections or allergies can trigger rhinitis. In case of disease or cold, please consult the doctor.

Keep out of the reach of children.

Dispose of following regulations on waste electrical and electronic equipment.

## SPECIFICATION

Model: DQ22  
Standard: EN 62368-1:2014+A11:2017  
EN55032:2015+A1 1:2020  
EN55035:2017+A1 1:2020

External power supply: 5V DC / 1A  
Power consumed by the device: 3.7W  
Built-in rechargeable battery: 1500mAh

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## DECLARATION OF CONFORMITY EU No. 3/2022

In scope of directives: 2014/35/EU, 2014/30/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o.  
Puławska Street 430  
02-884 Warsaw  
Poland

We hereby declare under our responsibility that **Electric Nasal Aspirator EasyStart with power supply** (cat. no. 9/319)  
brand: **Canpol babies**, model: **DQ22**,

is in accordance with the following directives:

- Directive relating to the making available on the market of electrical equipment designed for use within certain voltage limits 2014/35/EU
- Directive relating to electromagnetic compatibility 2014/30/EU
- Energy related Products Directive 2009/125/EC
- RoHS Directive 2011/65/EU and 2015/863/EU

and is in accordance with the following harmonized standards:

EN 62368-1 :2014 + A11 :2017  
EN 55032-1:2015 + A11 :2020  
EN 55035-2:2017 + A11 :2020  
EN IEC 63000:2018

as well as other technician specifications:

The Commission Regulation (EU) No. 2019/1782

The last two digits of the year in which the  marking was affixed: 22

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( place and date of issue )

( name, surname and signature of authorized person, position )

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