nenor[®]

Velo

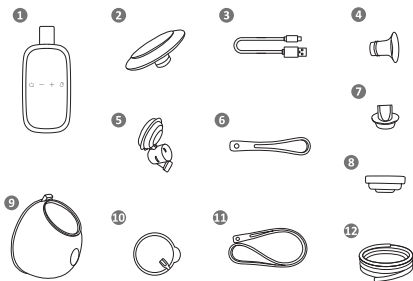
Table of contents


Spis treści

5	Instrukcja obsługi	PL
12	User manual	EN
18	Bedienungsanleitung	DE
26	Návod k použití	CZ
32	Használati utasítás	HU
39	Používateľská príručka	SK
46	Användarhandbok	SE
53	Käyttäjän käsikirja	FI
59	Brukerhåndbok	NO
66	Brugermanual	DK
72	Gebruikershandleiding	NL
79	Manual del usuario	ES
86	Manuale utente	IT
93	Manuel de l'utilisateur	FR
100	Manual de utilizare	RO

A

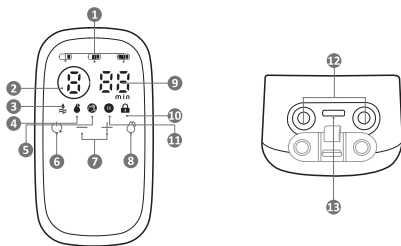B

C

1

2

3

4

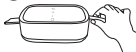
5

6

7

8

9

10

11

12

13

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USER MANUAL

Dear Customer,

The Neno Velo device you have purchased is a dual, electronic, three-phase breast pump for expressing breast milk from a woman's breast after child-birth. Read the instruction manual before use.

01. PRECAUTIONS

1. Always unplug the device after charging if you do not need to recharge the device's battery.
2. Do not place the product where it is in danger of falling into sinks, bathtubs or other bodies of water.
3. The breast pump and its components are not heat resistant - keep the device away from hot surfaces and open flames.
4. Do not use the device near flammable materials.
5. The breast pump should not be left unattended when connected to a power source.
6. Never use the device if it has been dropped or otherwise damaged.
7. Do not use a device whose power cord or plug is damaged.
8. Do not use the device in the bath or shower.
9. Do not place the device under water or in other liquids.
10. Do not wash the device under running water.
11. Do not connect the device to a power socket that has been in contact with water or other liquids.
12. If the device has been exposed to water or other liquids - do not touch the device directly and contact the manufacturer.
13. This appliance is intended for use by a single user. Use of the breast pump by more than one person carries a risk of harm to the users and voids the warranty.
14. Do not repair or modify the device yourself.
15. Use the breast pump only as described in the instructions. Do not use the device for any other purpose.
16. Inspect all breast pump components before each use.
17. Before using the breast pump for the first time, wash and disinfect all parts of the machine that will come into direct contact with your breast or milk.
18. After each use of the breast pump, wash all parts of the machine that come into direct contact with your breast or milk.

19. Take special care when using the breast pump or its accessories near children.
20. Do not use the breast pump while it is charging.

02. BEFORE FIRST USE

1. Connect the breast pump to the power supply. Charge the batteries for 3 hours.
2. Disconnect the cups with tubes from the pump. Disconnect the parts of the breast pump that will come into direct contact with the breast or milk: remove the breast shield from the milk container, then pull the valve down and the membrane up to disconnect from the connector.
3. Clean the device components (refer to the CLEANING AND DISINFECTION section).

NOTE!

Use the breast pump in an upright position. Use in any other position (e.g. lying down) will result in the breast pump not fitting properly to the breast and milk leaking out.

03. ELEMENTS

FIG. A

1. Pumping unit
2. Breast shield 24mm x2
3. USB-C charging cable
4. Breast shield reducer 20mm x2
5. Connector x2
6. Short silicone lanyard
7. Valve x4
8. Membrane x4
9. Milk container x2
10. Membrane cover x2
11. Long silicone lanyard
12. Tube x2

04. DESCRIPTION OF THE PUMPING DEVICE

FIG. B

1. Battery icons - While charging, the icons will illuminate one at a time, when the device is fully charged, the full battery icon will be highlighted.
2. Icon of the selected mode (L1-L8).
3. Stimulation phase icon.
4. Deep extraction phase icon.
5. Mixed phase icon.

6. Power button - Hold the button for 2 seconds to switch the device on or off. Press briefly to pause the breast pump.
7. „-“ and „+“ buttons. Press to reduce/increase suction power.
8. Mode change button/Button lock. Press briefly to select stimulation/deep suction/mixed phase. Press for 2 seconds to lock/unlock the device. The breast pump automatically locks after 20 seconds of inactivity.
9. Working time of the breast pump.
10. Button lock icon.
11. Pause icon.
12. Tube connector.
13. USB-C charging socket.

05. CLEANING AND DISINFECTION

Washing and disinfecting are two different operations. They must be carried out separately to protect your baby and to ensure that the breast pump works properly.

Washing - washing the surface of the unit and removing dirt.

Disinfection - killing organisms such as bacteria or viruses that may be present on the surface of the equipment.

Breast shield, membrane, valve, reducer, connector, milk container:

Wash and disinfect before first use.

Wash after each use.

A. WASHING:

- Disassemble all parts of the breast pump (except the pumping unit and tubes) and wash in warm soapy water for 5 minutes, then rinse. This should be done immediately after using the machine. This will remove milk deposits and prevent bacterial growth.
- Use only drinkable tap water for washing. Use bottled water if the tap water does not meet this requirement.
- When cleaning silicone parts, do not use brushes. Be careful not to pierce them through.
- Use a mild dishwashing liquid or soap.
- Allow the parts to dry naturally. Store them in a cool, dry place when you are not using them.

B. DISINFECTION:

Membrane, valve, reducer, connector, breast shield, milk container:

- Fill the pot or steriliser with water so that all the components inside (membrane, valve, reducers, connector, breast shield, milk container) are covered

when placed inside. Bring to the boil. Place breast pump parts in boiling water for 10 minutes. Do not boil.

- Allow the pieces to dry naturally. Store dry kit items in a cool, dry place when you are not using them.

NOTE: Tubes and membrane covers cannot be sterilised.

C. CLEANING THE PUMPING DEVICE:

- Wipe the pump unit with a soft, wet cloth and a mild dishwashing liquid/soap. Other cleaners or abrasives may damage the unit and its display.

- When not using the pump unit, store it in a cool, dry place.

06. ASSEMBLY AND USE OF THE BREAST PUMP

FIG. C

Attach the components as shown in FIG. C 1-13.

NOTE: Wash your hands thoroughly with soap and water before touching the breast pump and breast.

1. Fix the membrane in the connector.
2. Fix the valve in the connector.
3. Insert the connector into the milk container making sure it is fixed flush with the edges of the connector hole.
4. Install the membrane cover (press inwards with index finger and thumb against the connector to ensure a proper fit).
5. Fit the breast shield and push it into place so that it is flush with the edges of the milk container.
6. Attach one end of the tube to the connector located in the membrane cover.
7. Attach the other end of the tube to the connector at the bottom of the pumping unit.
8. If using a single breast pump - Connect the tube to the connector located under the plug at the bottom of the pumping unit. Close the other connector with the silicone cap. When using a breast pump for two breasts - Connect both tubes to the connectors located under the plug at the bottom of the pumping unit.
9. Place the breast shield on the breast so that the nipple is inside the drainage channel. Press the breast shield against the breast using your thumb and index finger.
10. Turn on the breast pump by pressing the power button and select the corresponding operating mode.
11. Once pumping is complete, remove the tube from the membrane cover.
12. Carefully transfer the milk into a bottle/storage container.
13. Disassemble and clean all breast pump components after use.

Tips:

- a. Before each use, check the parts of the breast pump for damage. If necessary, replace damaged parts with new ones.
- b. Always check the cleanliness of components before using the unit.
- c. If the diameter of the breast shield is too large, use the supplied reducer by inserting it inside the funnel.

NOTES:

The breast pump starts in stimulation mode. Set the level of suction strength comfortable for you using the +/- buttons:

- Stimulation mode - fast pumping that mimics your baby's natural sucking of breast milk to stimulate milk flow. After 2 minutes, the breast pump will automatically activate the deep pumping mode. Adjust the suction level using the +/- buttons to find the highest level of suction that is comfortable for you. If milk starts to flow from your breast before two minutes have elapsed, press the mode change button to manually switch to deep extraction mode.
- Deep pumping mode - slower pumping that gently and efficiently sucks milk from the breast. In intake mode, increase the suction until you feel slight discomfort (not pain) and reduce the suction by one degree.
- Mixed mode - mimics the baby's sucking reflex. Can be used after pumping to help empty the breast and increase lactation.

NOTE! Use the breast pump in an upright position. Use in any other position (e.g. lying down) will result in the breast pump not fitting properly to the breast and milk leaking out.

07. CHARGING THE BREAST PUMP

Fully charge the breast pump battery before putting it into storage if you intend not to use the device for a long period of time. Charge the device battery before it is completely depleted. This extends the life of the battery. If the breast pump has been stored in a hot place, the device battery may not work at first. Allow the breast pump to cool down for one hour before you start using it.

Indications:

- Flashing battery symbols - breast pump under charge
- Flashing battery symbol with one dash - low battery charge
- Battery symbol with two dashes - half charge of battery
- Battery symbol with three dashes - battery fully charged

NOTES:

Do not use the breast pump while it is charging.

08. REPLACEMENT OF CONSUMABLES

Consumable components are subject to natural wear and tear during operation of the breast pump and are therefore not covered by the manufacturer's warranty.

Worn components of the breast pump that come into contact with milk should be replaced. A worn component has a crack, deformation, tarnishing or a distinct change in colour.

- The membrane should be replaced every 8 weeks (if you use the breast pump 1-3 times a day) or every 3-4 weeks (if you use the breast pump more than 3 times a day).
- The valve should be replaced every 2-3 months (if you use the breast pump 1-3 times a day) or every 4 weeks (if you use the breast pump more than 3 times a day).
- The breast shield and connector should be replaced every 6 months if you use the breast pump more than 3 times a day.

09. LONG-TERM STORAGE

The breast pump has a built-in rechargeable battery. To protect itself from damage, the device enters storage mode after a long period of inactivity. To deactivate the storage mode, connect the device to power and charge it for 2 hours. Before you put the breast pump away for a long period of time, make sure that:

All components of the breast pump are clean.

You have cleaned the pumping device.

The device's battery is fully charged.

10. FAQ

1. **The device does not start:** make sure the breast pump is plugged into a working electrical outlet.
2. **No/poor suction:** make sure all breast pump parts are positioned correctly. Take the breast pump apart and make sure all parts are clean and undamaged. Reassemble the breast pump according to the instructions in the ASSEMBLY OF THE BREAST PUMP section. When pumping, make sure the breast shield is fully seated against the breast.
3. **More milk has flowed from the breast than the container can hold:** Use a dampened (not wet) cloth to wipe the breast pump and tubes. Wash the parts of the breast pump that hold milk according to the instructions in the CLEANING AND DISINFECTING section. When all parts of the machine are dry, reassemble.

11. SPECIFICATION

Milk container volume: 180ml x2

Battery capacity: 3.7V 2000mAh lithium-polymer

Output power: 5V/2A

Operating time: up to 105min

Charging time: approx. 3h

Dimensions of breast pump: 12x7x4cm

Weight of breast pump: 236g

Charging cable: USB-C

Length of longer leash: 42cm

Length of shorter lanyard: 15cm

Suction power:

1. Stimulation mode (8 levels): 6-24 kPa
2. Deep extraction mode (8 levels): 9-37 kPa
3. Mixed mode (8 levels): 7-26 kPa

Permissible variation ± 5 kPa.

12. WARRANTY/ COMPLAINTS CARD

The product comes with a 24-month warranty. Warranty conditions can be found at: <https://neno.pl/gwarancja>

Details, contact and service address can be found at:

<https://neno.pl/kontakt>

Specifications and kit contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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