NoseNest

Automatyczny aspirator do nosa

Automatic nasal aspirator

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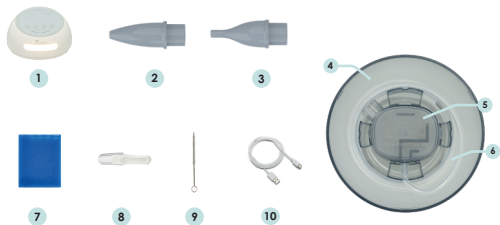
User manual


Create happy beginnings

A

C

B

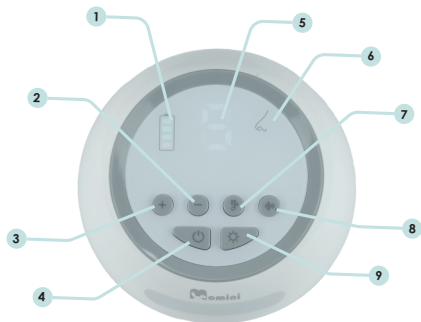1

2

3

4

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01. SECURITY:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

02. SET CONTENTS [PAGE 2 - A]:

1. Aspirator.
2. Round tip.
3. Pointed tip.
4. Cover.
5. Collection chamber.
6. Tube.
7. Hygienic filter.
8. Nose cleaning tweezers.
9. Cleaning brush.
10. USB-C charging cable.

03. DISPLAY DESCRIPTION [PAGE 2 - B]:

1. Battery charge indicator.
2. Intensity level decrease button (to decrease, press briefly, to mute music or noise, hold down the button).
3. Button to increase the intensity level (to increase it, press it briefly, to increase the volume of the music or noise, hold the button down).
4. On/off button (hold 3 seconds to turn the device on or off, a short press pauses or resumes).
5. Suction intensity level indicator (6 levels).
6. Aspirator operation indicator.
7. Music on/off button (hold down for 3 seconds to turn off the music, a short press changes the melody).
8. Noise on/off button (hold 3 seconds to turn off the noise, short press changes the noise).
9. Light intensity change button (3 light modes, short press changes the mode).

04. DESCRIPTION OF THE DEVICE ASSEMBLY [PAGE 2 - C]:

1. Insert the hygiene filter into the suction channel in the collection chamber and close the cover of the collection chamber.
2. Place the silicone tip on the tab of the collection chamber.
3. There are two holes in the main unit, exhaust (A) and intake (B).
4. Connect the collection chamber to the suction port with a tube.

Note: The collection chamber cover can only be closed from one fixed side. Please place the collection chamber in the direction of the arrow up.

05. HOW TO USE:

1. Choose a silicone tip appropriate for your child's age. The round tip is suitable for babies aged 0-3 years and the pointed tip is suitable for children over 3 years of age.
2. Insert the silicone tip into the nostril, rotate gently inside, do not hold in one place for too long.
3. If there are dried mucus lumps in the nostrils, they should be softened with physiological saline or removed using the tweezers included in the kit.
4. When using the aspirator, please use low suction level first and increase the level slowly, do not start at the highest level.
5. When using, pay attention, if the nasal mucus is sucked into the tube, please stop the aspirator immediately to prevent the mucus from being sucked into the device and affecting normal use.

06. MAINTENANCE AND CARE OF THE DEVICE:

1. Turn off your device. Then dismantle the collection chamber. To avoid damaging the machine circuit, the main unit cannot be flushed with water.
2. The hygienic filter is disposable and does not require cleaning. Do not reuse it to prevent bacteria from multiplying.
3. The collection chamber, lid and silicone tip can be rinsed with water, sterilized with steam, boiling water or disinfected with UV light. The inside of the silicone tip can be cleaned with a small brush. After cleaning the collection chamber, insert a tube into the exhaust port to dry the collection chamber.

07. PRECAUTIONS:

The nasal aspirator will automatically stop after 10 minutes. It can remember the suction level, music volume and light brightness used before the last time it was turned off. Do not use the nasal aspirator on the outside of your nose. Do not use the product as a toy for children. If your nose has sores, inflammation, or signs of bleeding, do not use the nasal aspirator. Handle the product carefully when using. Do not use while charging. Do not use the device if it is damaged. Do not repair, disassemble or modify the device yourself. Any irregularities should be assessed by the service. Do not drop or hit the device to avoid malfunction. Do not wash the device under water or submerge it. Do not use corrosive agents to clean the device. Store it in a cool, dry place. During use, make sure the child's mouth is open and there is air circulation in the nasal cavity to avoid discomfort due to negative pressure. When using the nasal aspirator, do not block the other nostril, thus avoiding negative pressure and discomfort. Clean the product

before each use. Disinfect the suction tip and secretion reservoir after use to avoid cross-contamination. The product is not a toy, keep it out of the reach of children.

Material: ABS, Silicone
Cup capacity: 30 ml
Charging method: USB-C charging
Charging time: about 3 hours
Battery: 2000mAh
Voltage: 3.7V
Input voltage: 5V-1A
Suction power: 65-70kPa
Noise: below 80 decibels

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed-out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in health damage or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of March 27, 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment

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[EN] Summary declaration of conformity: The manufacturer declares, under its sole responsibility, that the device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances. The full content of the declaration of conformity is available at www.momini.eu/deklaracje

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EN | More information about the product is available on the manufacturer's website at:

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