Momini

Sniffley

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
Automatyczny aspirator do nosa

Automatic nasal aspirator

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
User manual


Create happy beginnings

A

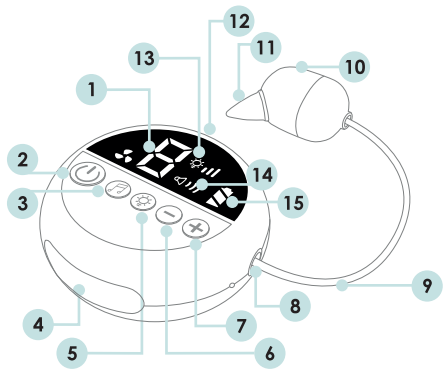B

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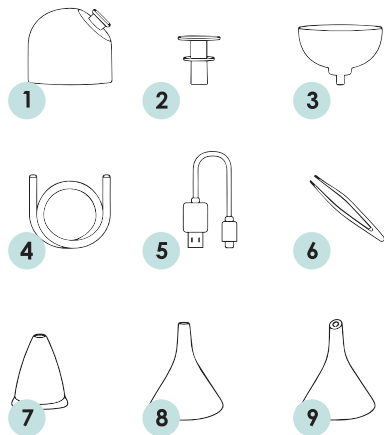Waga netto produktu: 235g

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01. SECURITY:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

02. DEVICE DESCRIPTION [PAGE 2 - A]:

1. Mode indicator / 2. Power button / 3. Music button / 4. Lamp / 5. Light adjustment button / 6. Suction power decrease button / 7. Suction power increase button / 8. Suction / 9. Tube / 10. Collection cup / 11. Silicone tip / 12. Charging port / 13. Brightness indicator / 14. Music/volume indicator / 15. Battery level indicator.

03. ACCESSORY DESCRIPTION [PAGE 2 - B]:

1. Cup cover / 2. Diversion column / 3. Lower part of the cup / 4. Tube / 5. USB charging cable / 6. Tweezers / 7. Conical tip / 8. Flat tip / 9. Angle tip.

Note: Before using the nasal aspirator, make sure you understand the features of each tip and choose the appropriate one based on your child's age to ensure comfortable and effective use.

Conical tip - Shallow Depth - 0-2 years.

Flat tip - deep - from 0 years.

Angled tip - very deep - from 0 years.

Power button [page 2 - A2]: Long press turns the device on/off. A short press starts/stops suction.

Music button [page 2 - A3]: A short press adjusts the volume to levels 1 to 3. The button sequence is: 1 → 2 → 3 → Off → On. Pressing it again will switch to different music.

Light button [page 2 - A5]: Off mode: Long press turns the light on/off. Once turned on, a single press cycles through brightness levels 1 to 3.

Power-on mode: Once the device is turned on, a single press switches brightness levels from 1 to 3 and then turns off.

Suction power reduction button [page 2 - A6]: After turning on the device, a single press of the "-" button reduces the suction power. You can select suction levels from 1 to 9.

Suction power increase button [page 2 - A7]: After turning on the device, a single press of the "+" button increases the suction power. You can select suction levels from 1 to 9.

04. DESCRIPTION OF DEVICE USE:

1. Install the separation insert on the bottom of the collection cup.
2. Place the collection cup on the bottom cover.
3. Select a tip and install it on the collection cup.
4. Connect the collection cup to the main unit's air inlet port (suction) with a hose.
5. Long press the "power" button to turn on the device (long press

again to turn it off).

6. Use the "-" or "+" suction power button to adjust the suction power to the desired level.

7. Use the "melody" button to activate an optional mode suitable for your child.

8. Gently place the tip inside your baby's nose to effectively cleanse their mucus.

9. After clearing the mucus from your baby's nose, remove the collection cup.

10. After removing the collection cup components, rinse them immediately. (Note: The main body of the device should not be washed with water!)

11. After cleaning, use a soft cloth to dry the surface of the device and store it in a cool place away from direct sunlight.

05. TIPS:

For better results, if the mucus in your baby's nose is dry, it is recommended to use saline to moisturize before using the electric nasal aspirator.

06. CLEANING AND STORAGE:

1. Always turn off the power after use.
2. Remove the suction tip and collection cup and clean them with cleaning agent and water.
3. Keep the parts and body of the device clean and dry after use.
4. The suction tip and collection cup can be sterilized by boiling in water or steam, but not in the microwave.
5. The main body of the product is not waterproof, so do not rinse it with water or clean it at high temperatures. Please use a soft damp cloth to clean the main body.
6. Avoid storing the product in direct sunlight or humid environments.

07. SECURITY MEASURES:

1. It is important that the baby's mouth is open during use to ensure proper airflow in the nasal cavity and prevent discomfort caused by negative pressure.
2. Avoid blocking the baby's other nostril when using the nasal aspirator to prevent discomfort from negative pressure in the head.
3. When the collection cup is half full, it is recommended to empty it before continuing use as this prevents mucus from being sucked into the pump and potentially damaging the device.

08. PRECAUTIONS:

1. Do not turn on the nasal aspirator while charging.
2. The device is intended for use by adults only.
3. Do not expose the device to high temperatures, high humidity or direct sunlight.
4. The device is specially designed to remove mucus from the nose. It

should not be used for any other purposes!

5. Clean and dry the parts and body of the device after each use.
6. Charge the battery to 100% every 2 months if you don't use it for a long time.
7. Do not immerse the entire device in water when cleaning.
8. During use, the child/infant should not hold their breath, close their mouth or swallow. Excessive vacuum pressure may cause an imbalance in ear pressure.
9. Do not use excessive force when placing the suction tip into the nasal cavity.
10. If your child/infant experiences any discomfort while using the device, stop using it immediately and consult a doctor.
11. Do not use the nasal aspirator on the outside of the nose.
12. Do not use the product as a toy for children.
13. If your nose has sores, inflammation or signs of bleeding, do not use the nasal aspirator.
14. Do not repair, disassemble or modify the device yourself. Any irregularities should be assessed by the service.
15. Do not use the device if it is damaged.
16. Do not use corrosive agents to clean the device.

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment

from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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[EN] Summary declaration of conformity: The manufacturer declares, under its sole responsibility, that the device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility and Directive 2011/65/UE of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Directive Commission delegated act (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/UE of the European Parliament and of the Council as regards the list of restricted substances. The full content of the declaration of conformity is available at www.momini.eu/deklaracja

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EN | More information about the product is available on the manufacturer's website at:

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