# Momini

## NasoClear

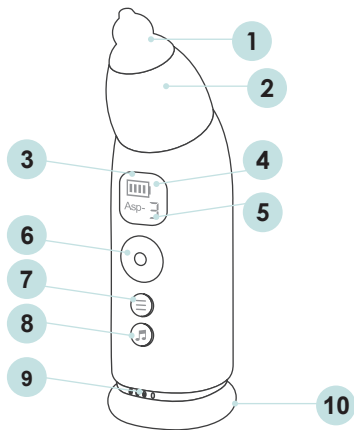
Create happy beginnings


**Automatic nasal aspirator**  
Automatyczny aspirator do nosa

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
A

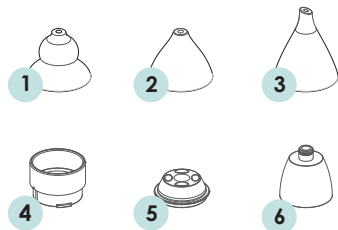B

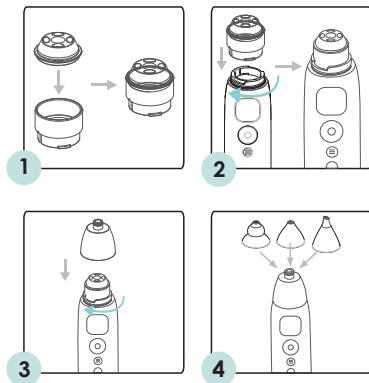C

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Bateria: 600 mAh

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**01. SECURITY:**

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

**02. DEVICE DESCRIPTION [PAGE 2 - A]:**

1. Replaceable durability
2. Collective cup
3. Organizing
4. Battery level
5. Suction modes
6. Power On/Off/Pause button
7. CHOKE switch
8. Music/light switch
9. Speaker holes
10. Charging port

**03. DESCRIPTION OF ACCESSORIES [PAGE 2 - B]:**

1. Pumpkin tip: for newborns
2. Straight tip for sucking secretions
3. Slant tip for children over 2 years old
4. Runny nose container
5. Runny nose container cover
6. Outer cover of the cold case container

**Note:** Before using the aspirator to control yourself, the results are determined by the user and is selected, based on the assessment of the child, who is the control and follow-up. In newborns using pumpkin-type silicone used through the oblique, it may reach too deep.

**04. INSTALLATION AND USE [PAGE 2 - C]:**

1. Closing the lid on the runny nose container.
2. Mount on your device by turning the update with your watch.
3. Put the outer cover of the catarrh container on.
4. Select and install the suction silicone power supply.
5. Start by 3 buttons to turn on/off the aspirator, short press to start or detect the operation of the device.
5. Short press the button to select one of three modes.
6. To process or detect software touch the music note button briefly; snap tabs: light on or off.

**05. CAME:**

- When in use, the application is tightly applied to prevent outside air from entering, making suction easier.
- If your child's nose is runny/mucous, it is recommended to soften it by using a warm towel or a drop of saline to moisturize it before using the nasal aspirator.

**06. CLEANING AND STORAGE:**

1. Always turn off the power after use.
2. Remove the suction tip, container, lid and cover, then clean them with cleaning agent and water and dry them.
3. The suction tips, container, lid and cover can be sterilized, for example by immersing them in boiling water at 100°C for a few minutes, but you cannot use a microwave or other high-temperature devices for sterilization.
5. The main body of the product is not waterproof, so do not rinse and/or immerse it in water or clean it at high temperatures. Use a soft, damp cloth to clean.
6. Avoid storing the product in direct sunlight or humid environments.

**07. SECURITY MEASURES:**

- When using the device, it is recommended that the child keep his mouth open to ensure proper air flow in the nasal cavity and not have the other nostril blocked to avoid discomfort caused by air pressure.
- Pay attention to the amount of nasal secretions when using the aspirator. If there is too much of it, the container may overflow and cause a runny nose. Empty the container and complete nasal cleaning.

**08. PRECAUTIONS:**

1. Do not turn on the nasal aspirator while charging.
2. The device is intended for use by adults only.
3. Do not expose the device to high temperatures, high humidity or direct sunlight.
4. The device is specially designed to remove mucus from the nose. It should not be used for any other purposes!
5. Clean and dry the parts and body of the device after each use.
6. Charge the battery to 100% every 2 months if you don't use it for a long time.
7. Do not immerse the entire device in water when cleaning.
8. During use, the child/infant should not hold their breath, close their mouth or swallow. Excessive vacuum pressure may cause an imbalance in ear pressure.
9. Do not use excessive force when placing the suction tip into the nasal cavity.
10. If a child/infant occurs when using the device, discontinue use immediately and consult the product.
11. Do not use the nasal aspirator on the outside of your nose.
12. Do not use the product as a toy for children.
13. If there are wounds, inflammation or bleeding in the nose, do not use the device.
14. Do not repair, disassemble or modify the device with your own hand. Any additional information may be disputed by the website.
15. Do not use the device, the device is damaged.
16. Do not use corrosive agents to clean the device.

**09. Power supply: 3.7V**

Input voltage: 5V/1A

Battery: 600 mAh

Material: ABS + PC, food-grade silicone

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**[EN] Rules for handling electrical and electronic equipment:** Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment

from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m<sup>2</sup> devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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**[EN] Summary declaration of conformity:** The manufacturer declares, under its sole responsibility, that the device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Directive Commission delegated act (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances. The full content of the declaration of conformity is available at [www.momini.eu/deklaracje](http://www.momini.eu/deklaracje)

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**[EN]** More information about the product is available on the manufacturer's website at:

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