**Momini**

**SleepSafe**

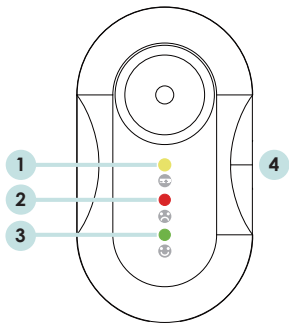
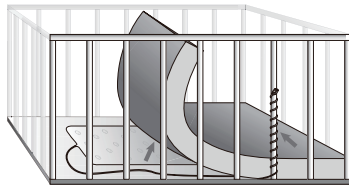
*Create happy beginnings*

**Monitor snu**  
Sleep monitor

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**A****B**

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Masa netto: 600g

**01. SAFETY:**

Before using the product, read this user manual and follow the instructions contained therein when using the device.

**02. DESCRIPTION OF THE REMOTE CONTROL [PAGE 2 - A]:**

1. Yellow indicator - a blink indicates low batteries.
2. Red indicator/diode - a blink signals an alarm.
3. Green indicator/diode - a short blink signals the child's breathing or movement.
4. Turn the device on/off.

**03. SENSOR INSTALLATION:**

1. Turn the device on (confirmed by a sound signal and flashing of all LEDs on the remote control).
2. Put the child in bed.
3. The green LED will blink when the child breathes or moves.
4. Before taking the child out of the bed, turn the device off to avoid the alarm going off.
5. The cable should be positioned so that it is not too loose or forms a loop and should be out of the reach of the child.
6. If the child's breathing is not detected for more than 20 seconds, or the breathing rhythm is slow, an audible alarm will sound and the red LED will flash.

**04. INSTALLING THE PAD [PAGE 2 - B]:**

1. Place the sensor pad on the bottom of the cot. Place the pad where the child will sleep. When the child is lying down, it should be as close to the center of the sensor pad as possible.
2. Make sure the sensor pad is positioned at the height of the child's chest.
3. The pad should be on a firm, hard surface.
4. If using the device outside the cot, the child should sleep on a separate sensor control pad and sensor.

**IMPORTANT:** When the alarm sounds when the child is sleeping. This may be due to:

1. The child's position that prevents detection of breathing.
2. If the baby is lying on a mattress that is too thick, which may interfere with monitoring, the alarm may also go off. It is not recommended to add an additional mat.
3. Too many clothes on the baby, which may affect the detection of breathing rhythm.
4. Slow breathing of some children during sleep.

**05. PRECAUTIONS:**

1. The device should not be used in an environment with potential interference, such as engine noise or proximity to devices that generate vibrations (e.g. air conditioning).
2. Do not place the pad on a soft or expanding surface, as this may interfere with the operation of the device.
3. Change the batteries in the device regularly and do not expose it to extreme temperatures or sunlight.
4. When cleaning the device, avoid contact with the battery and electronic components.
5. The alarm volume of the device is designed to remind the user to monitor. However, this may scare and worry people nearby, so pay attention to this.
6. It is not recommended to use the device when the child's body and limbs are wrapped (because completely wrapped arms and legs may interfere with movement detection).
7. The device can only monitor the child's breathing rate, it cannot prevent changes in breathing rate or provide assistance if breathing becomes irregular or stops. The device should not replace parental supervision. Always stay close to the child so that you can hear the alarm if the monitor detects a change in breathing.
8. This product can only be an aid and cannot replace a human!

**Power supply: AA batteries (1.5V x 2)**

Current: 0.2mA

Operating environment: Temperature from 5° to 40°C

Humidity: 15% ~ 90%

Atmospheric pressure: 700 ~ 1060 hPa

Storage environment: Temperature from -25° to 70°C

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**[EN] Rules for handling electrical and electronic equipment:** Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m<sup>2</sup> devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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**[EN] Short declaration of conformity:** The manufacturer declares, under its sole responsibility, that the product complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, Directive 2014/53/EU of the European Parliament and of the Council of 16 April 2014 on the harmonization of the laws of the Member States relating to the making available on the market of radio equipment and repealing Directive 1999/5/EC and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive (EU) 2017/2102 of the European Parliament and of the Council of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at [www.momini.eu/deklaracje](http://www.momini.eu/deklaracje)

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**EN |** More information about the product is available on the manufacturer's website at:

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