PL EN DE ES PT CZ SK SL EE

2

PL

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EN

- 01. **SAFETY:** Before using the product, read this manual and follow its recommendations during use.
- 02. **DEVICE DESCRIPTION [PAGE 2 - A]:** 1. On/Off/Vibration - press and hold to turn on. Short press changes vibration modes or pauses. Press and hold again to turn off. / 2. Heating - short press changes modes (yellow light - medium mode, red light - high mode, blue light - heating off).
- 03. **USAGE METHODS [PAGE 2 - B]:** Wash your hands and breasts. Sit comfortably and lean slightly forward. Do not lie down or tilt the massager. Apply the massage pad and gently massage the breast towards the nipple. Use the narrow part of the massager and gently press to massage/unblock clogged ducts. Place the massager on different parts of the breast to stimulate milk flow and increase production.
- 04. **PRECAUTIONS:** Charge the product before first use (~ 2 hrs). Full charge is indicated by a steady blue light. / Clean with a soft cloth before and after use. Do not immerse in water or use chemicals. / The product will automatically turn off after 20 minutes, avoid using the same area for longer than 20 minutes. / Regularly check the device. Do not disassemble or replace any parts yourself. / Inspect the product before use, if damaged, stop using immediately. / Keep away from direct sunlight, heat sources, and children - this is not a toy. Store in a cool, dry place.
- Voltage: 3.7 V ~ / Current: 810mA / Power: 3W / Battery life: 80-90 minutes

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Used electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m<sup>2</sup> devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005.

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[EN] Short declaration of conformity: The manufacturer declares, under its sole responsibility, that the Calma device complies with Directive 2014/53/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the electromagnetic compatibility, Directive 2014/53/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the making available on the market of electrical equipment intended for use within specified voltage limits and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive (EU) 2017/102 of the European Parliament and of the Council of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at [www.momini.eu/deklaracja](http://www.momini.eu/deklaracja)

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