 **Momini**

BlissPump

Create happy beginnings

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Pojemność baterii: 3,7 V, 1400 mAh

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01. SECURITY:

Before using the product, read this manual and follow the guidelines during operation.

02. DEVICE DESCRIPTION - 2 SETS [PAGE 2 - A]:

1. Device / 2. Cylinder (silicone) / 3. Valve (silicone) / 4. Pump body (polypropylene) / 5. Flange insert 21mm (silicone) / 6. Flange insert 24mm (silicone) / 7. Milk container cap (polypropylene) / 8. Silicone ring for milk container (silicone) / 9. Milk container (nylon polyethylene) / 10. Charging cable

WARNING [PAGE 2 - B]: Measure your nipple size and choose the appropriate flange insert. The box contains 21mm/24mm flange inserts.

03. ASSEMBLY OF THE PUMP [PAGE 2 - C]:

Wash your hands and ensure the device is clean.

1. Place the cylinder on the pump body.
2. Insert the pump body into the device.
3. Insert the valve into the milk container cap. Place the silicone ring on the bottom of the cap. Attach the cap to the container and turn until it clicks.
4. Insert the milk container into the bottom of the device until you hear a click.
5. Do not exceed the maximum milk container capacity. If needed, turn off the pump and empty or replace the container.

04. DISASSEMBLY OF THE PUMP [PAGE 2 - D]:

1. On/Off/Pause - hold the button to turn the device on or off. When it is on, press briefly to pause or restart the pump. After 10 minutes of inactivity, the device will turn off automatically.
2. Phase change. A short press switches between the available phases: simulation, massage, expression, and sustained expression.
3. Suction power adjustment.
4. Charging: flashing symbol means low level and will turn off automatically. 3 lit symbols mean fully charged.
5. Timer - the device will automatically turn off after 30 minutes.
6. Level display from 1 to 9.
7. Sustained expression phase: 5 light, 1 strong suck
8. Massage phase: 9 power levels for breast massage before milk expression
9. Simulation phase: two short, one long, three short, two long sucks.
10. Expression phase: constant frequency and strength. It has a gentle sucking rhythm

05. LOADING [PAGE 2 - D]:

Fully charge the product before first use. Connect the USB plug to a power bank, computer or adapter.

06. DEVICE USER MANUAL [PAGE 2 - E]:

1. Make sure all parts of the breast pump are clean/sterilized and that

the device has been assembled correctly (follow the instructions in this user manual)

2. Wash your hands and breasts before the massage.
 3. Sit up straight and lean forward slightly. Maintain good posture when expressing milk. DO NOT lie on your side and DO NOT tilt the breast pump.
 4. Adjust the collar so that it fits snugly around the nipple and breast.
 5. After pumping, if you don't need to use the milk right away, close the bottle and store the milk in the refrigerator.
 6. Clean and dry all parts of the breast pump after each use.
- NOTE:** Do not fill the bottle with too much milk, do not exceed the maximum capacity. In this case, turn off the device, replace the bottle or pour the milk into another container.

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08. CLEANING AND STERILIZATION:

1. The main unit (with motor) and the charging cable cannot be washed in water or sterilized. If their surface is dirty, clean them with a soft cloth. Don't use chemicals.
2. When sterilizing in a microwave oven, use an appropriate sterilization container. You can also use a bottle sterilizer.
3. The cylinder, valve, massage cover and nipple are made of silicone. Due to the delicate material, clean and rinse them with water. They are not suitable for rubbing and long-term sterilization. Otherwise, it may cause rapid aging or deformation.
4. The pump body, flange, hygiene cover, bottle, bottle cap and threaded cap can be washed in water. After cleaning, they can be sterilized using a steam or ultraviolet sterilizer, microwave (place the parts to be disinfected in a container or disinfection bag) or boiling water (the amount of water cannot be less than the parts to be disinfected, and the parts cannot directly touch the bottom of the pot or metal edge).
5. After cleaning and/or sterilization, the device components must be dried before use.

09. WARNINGS:

1. All components of the device must be dry before use to prevent damage to the engine.
2. The battery can only be replaced by authorized service centers. Dispose of used batteries in accordance with local regulations.

3. If the product is not used for a long time, the battery will run out. It must be connected to electricity and charged.
4. The pump motor and other electrical parts cannot be washed in water or sterilized in a steam sterilizer.
5. Use the charging cable provided with this product.
6. This is not a toy. The appliance must NOT be used by children or persons with physical or mental limitations.
7. If you have problems with lactation or pain, consult your doctor before using the breast pump.
8. Store in a dry and cool place away from children.

● Voltage: 5V---
Current: 650mA
Power: 3.25W

Battery capacity: 3.7V, 1400mAh
Battery life: 80-90 minutes

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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[EN] Short declaration of conformity: The manufacturer declares, under its sole responsibility, that the BlissPump device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, Directive 2014/53/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the making available on the market of electrical equipment intended for use within specified voltage limits, Regulation (EC) No 1935/2004 of the European Parliament and of the Council of 27 October 2004 on materials and articles intended to come into contact with food and repealing Directives 80/590/EEC and 89/109/EEC, Commission Regulation (EU) No 10/2011 of 14 January 2011 on plastic materials and articles intended to come into contact with food, together with Commission Regulation (EU) 2020/1245 of 2 September 2020 amending and correction of Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food, and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in equipment electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive of the European Parliament and of the Council (EU) 2017/102 of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at www.momini.eu/deklaracja

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EN | More information about the product is available on the manufacturer's website at:

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