
Momini

MilkySilky

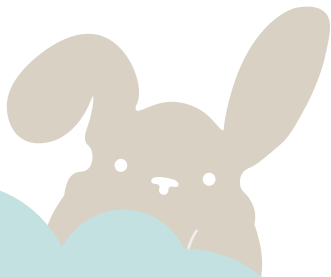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01. SECURITY:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

02. DEVICE DESCRIPTION [PAGE 2 - A]:

1. Top cover (PP) / 2. Diaphragm (silicone) / 3. Pump body (PP) / 4. Flange (PP/SEBS) / 5. Non-return valve (silicone) / 6. Device / 7. Bottle cap (PP) / 8. Nipple (silicone) / 9. Cap (PP) / 10. Bottle (PPSU) / 11. Tube connector / 12. Air tube (PVC) / 13. Charging cable

03. INSTALLATION OF THE BREAST PUMP [PAGE 2 - B]:

Wash your hands and make sure the device is clean.

1. Insert the non-return valve into the breast pump body.
2. Attach the collar to the body.
3. Insert the membrane into the body.
4. Screw the diaphragm cover to the body.
5. Screw the bottle onto the body.
6. Start the end of the air tube into the diaphragm cover port.
7. Single/Double Side Switch [Side 2-C]: (A) Insert the tube into one slot and lock the other to use single pump; (B) Insert the tubes into both slots to use double pump.
8. Do not exceed the maximum capacity of the milk bottle. An overfilled bottle may cause leaks through the holes in the pump. Turn off the breast pump and empty or replace the bottle to continue suckling, and store any excess milk in a closed bottle in the refrigerator. Once you're done, wash and dry all non-electronic parts of your breast pump.

04. CHARGING [PAGE 2 - D]:

Fully charge the product before first use. Insert the plug into the Type-C socket on your device and connect the USB port to your computer or adapter. The device must be turned off while charging. A flashing light means the battery is charging, and a steady light means the device is charged. Then disconnect them from the power supply.

05. BUTTON OPERATING INSTRUCTIONS [PAGE 2 - E]:

1. On/Off: long press to turn on; short press to turn off the device.
2. Changing phases: the device has 4 phases: massage, stimulation, deep expression and deep expression with support.
3. Pause: short press stops expression, press again to resume. The pump will automatically turn off after 10 minutes of inactivity.
4. Suction power adjustment: increases or decreases level from 1 to 9
5. Power: a flashing LED means that the device needs to be charged. Automatic shutdown occurs when the power level is too low.
6. Timer: automatic shutdown after 30 minutes.
7. Level display: 1 to 9.
8. Sustained expression phase: 5 light, 1 strong suck

9. Massage phase: 9 power levels for breast massage before milk expression
10. Simulation phase: two short, one long, three short, two long sucks.
11. Expression phase: constant frequency and strength. It has a gentle sucking rhythm.

06. DEVICE USER MANUAL

1. Make sure all parts of the breast pump are clean/sterilized and that the device has been assembled correctly (follow the instructions in this user manual)
 2. Wash your hands and breasts.
 3. Sit up straight and lean forward slightly. Maintain good posture while pumping. DO NOT lie on your side; you cannot tilt the breast pump to avoid milk backflow.
 4. Position the center of the collar over your nipple and press it against your breast to ensure regular latching and prevent air from entering.
 5. Long press the start button to adjust the mode and suction level according to your preferences.
- NOTE: Do not fill the bottle with too much milk, do not exceed the maximum capacity. In this case, turn off the device, replace the bottle or pour the milk into another container.

07. STORING AND HEATING MILK:

1. You can store expressed milk in the refrigerator for 48 hours at a temperature of approximately 4°C. When freezing it, leave space at the top of the container/bottle.
2. To preserve the nutritional value of expressed milk, it can be heated in water at 40°C or use a bottle warmer.
3. Do not thaw frozen expressed milk in the microwave or in boiling water (this will damage its nutritional value).

08. STERILIZATION INSTRUCTIONS:

1. The device, charging cable, air tube connector cannot be washed or disinfected. If they are dirty, clean them with a soft cloth.
2. Top cover, air tube: If dirty, just wash in water and dry before use.
3. Pump body, bottle, bottle cover, cap: dishwasher safe (temperature cannot exceed 90°C). After cleaning, they can be sterilized using a steam or ultraviolet sterilizer, microwave (place the parts to be disinfected in a container or disinfection bag) or boiling water (the amount of water cannot be less than the parts to be disinfected, and the parts cannot directly touch the bottom of the pot or metal edge).
4. Diaphragm, collar, nipple, valve: are made of silicone. Due to the delicate material, clean and rinse them with a gentle stream of water. They are not suitable for rubbing and long-term sterilization at high temperatures. Otherwise, they may cause rapid aging or deformation.

09. WARNING:

1. All components must be dry before using the device to prevent water from damaging the engine

2. If the product is not used for a long time, the battery will run out. Plug it in and wait until the screen lights up, indicating that it is fully charged.

3. The pump motor and other PVC electrical parts cannot be washed or sterilized. If they are dirty, clean and dry them. Make sure there are no drops of water in the tube before continuing use.
4. After sterilizing the membrane, please dry it before use.
5. When the battery light flashes after starting, it means the battery level is insufficient. It must be charged in good time.
6. Use only the charging cable provided by the manufacturer.
7. Maintain correct posture when expressing milk, do not lie on your side.
8. If you have problems with lactation or pain, consult your doctor before using the breast pump.
9. Don't fill the bottle to the brim; this may cause milk to leak out of the side holes in the pump body. If it overflows, turn off the pump, replace the bottle with a larger one and continue expressing.
10. After pumping, make sure that there are no milk residues in the parts of the pump after they have dried. If you don't need to feed your baby immediately, store the milk in the refrigerator.
11. Store the product in a dry place, at room temperature, out of the reach of children. Avoid direct sunlight on the product. The product is not a toy and is not suitable for use by children.

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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[EN] Short declaration of conformity: The manufacturer declares, under its sole responsibility, that the MilkySilky device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, Directive 2014/35/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the making available on the market of electrical equipment intended for use within specified voltage limits, Regulation (EC) No 1935/2004 of the European Parliament and of the Council of 27 October 2004 on materials and articles intended to come into contact with food and repealing Directives 80/590/EEC and 89/109/EEC, Commission Regulation (EU) No 10/2011 of 14 January 2011 on plastic materials and articles intended to come into contact with food, together with Commission Regulation (EU) 2020/1245 of 2 September 2020 amending and correction of Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food, and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive of the European Parliament and of the Council (EU) 2017/2102 of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at www.momini.eu/deklaracje

www.momini.eu/deklaracje
 Importer: Beauty Factor Sp. z o.o. Plac Białokory 2, 00-095 Warszawa, Polska
 Model: MilkySilky

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[EN] More information about the product is available on the manufacturer's website at:

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