NebuEasy

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READ THIS USER'S MANUAL CAREFULLY BEFORE USING.

THE PICTURES ARE FOR REFERENCE ONLY.

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COMPRESSOR NEBULIZER
MODEL NO.: MCN-DS600C

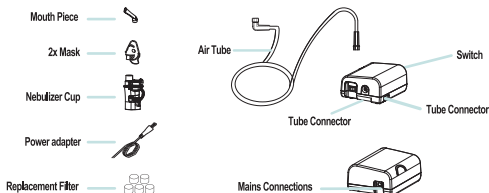
1. INTRODUCTION

Thank you for purchasing the Compressor Nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your Compressor Nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

2. PRODUCT IDENTIFICATION

3. IMPORTANT SAFEGUARDS

Note: Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product:

Caution: Failure to read and observe all precautions could result in personal injury or equipment damage.

Product cautions:

1. To avoid electrical shock: Keep the unit away from water - Do not immerse the power cord or the unit in liquid - Do not use while bathing - Do not reach for a unit that has fallen into water - immediately unplug the unit.
2. Never operate the unit if it has any damaged parts (including power cord), if it has been dropped or submerged in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
5. Do not use other attachments unless recommended by manufacturer.

Operating Cautions:

1. Connect this product to an appropriate voltage outlet for your model.
2. Do not operate this product unattended.
3. Never run this unit if it has a damaged cord or plug, if it has been dropped, or in any way, into water, if it does not work properly. Return it to a service center for repair.
4. If any abnormality occurs, discontinue use immediately until the unit has been examined and repaired.
5. Always unplug the power plug immediately after use.
6. Never block the air openings of the main unit or place it where the air openings may be obstructed.

Storage Cautions:

1. Do not store the unit under direct sunlight, high temperature or humidity.
2. Keep the unit out of reach of small children.
3. Always keep the unit unplugged while not in use.

Cleaning Cautions:

1. Do not immerse the unit in water. It may damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guide.

4. OPERATING YOUR COMPRESSOR NEBULIZER

Note: Prior to initial operation, the nebulizer should be thoroughly cleaned.

1. Place your Compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated. Important: Prior to initial operation, the nebulizer should be thoroughly cleaned referring to the 'Cleaning Procedures' in the manual.
2. Gently twist the top part of the nebulizer counter-clockwise to disassemble the nebulizer.
3. Fill the bottom section of the nebulizer with the medication prescribed by your physician. Be sure that the cone is put inside the bottom section.
4. Gently twist the top part clockwise to re-assemble the nebulizer. Be sure the two sections fit well.
5. Attach one end of air tube to the base of the nebulizer.
6. Attach the compressor air pipe to your choice of the top section located on the front of the other end of it.
7. Attach mouth piece or mask per your choice to the top section of nebulizer.
8. Plug the power adapter into an appropriate electrical outlet. Make sure that at this stage, the power switch is at "OFF" status.
9. Press power switch to begin your prescribed treatment.

Important:

The compressor motor has a thermal protector which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- a. Switch off the unit.
 - b. Unplug the unit from the electrical outlet.
 - c. Wait 30 minutes for the motor to cool down before another treatment.
- Make sure the air openings are not obstructed.
11. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

**5. CLEANING**

It is recommended that the nebulizer cup, mouthpiece and mask are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Rinsing (after each treatment):

1. Disconnect the air tube, nebulizer cup, mouthpiece and mask.
2. Rinse the nebulizer, mouthpiece and mask with water.
3. Gently twist the nebulizer to open it.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

Disinfections:

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece and mask.
2. Complete Rinsing Steps 1-3.
3. Wash nebulizer cup, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.
4. Submerge these parts in the vinegar and water solution for thirty minutes.
5. Complete Rinsing Steps 3-5.

Cleaning the compressor:

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads, which may damage the finish.

Filter change:

1. Do not use cotton or any other materials. Do not wash or clean the filter. Use only Safe Touch's filters. And do not operate without a filter.
2. Change the filter every 30 days or when the filter turns gray.
3. Changing procedure:
 - A. Remove the filter cover
 - B. Replace the used filter with a new one.
 - C. Replace the filter cover.

6. SPECIFICATIONS

Model:	MCN-DS600C
Product:	DC Compressor Nebulizer
Power supply:	Power adapter(AC100~240V 50H/60 DC 12V 1A)
Power consumption:	≤ 11W
Compressor pressure Range:	Approx.16~22 Psi/110~151 Kpa
Operation pressure Range:	Approx.6~10 Psi/41~69Kpa
Liter flow Rang:	Approx.2.5~5 lpm
Nebulization rate:	≥ 0.20 ml/min
Particle size:	≤5um Approx.63%
MMAD:	Approx. 4.42um
Capacity of medication cup:	Max. 7ml
Noise level:	≤49dBA (acc. to DIN EN 13544-1 section 26)
Expected service life:	400h
Dimension:	Approx. L117 X W90 X H58mm
Weight:	Approx.208g
Operating temperature and humidity:	10 ~40°C, 30 ~ 85 % RH, 800~1060hPa
Storage and delivery condition:	-20~70°C, 20 ~ 75 % RH, 800~1060hPa
Safety level:	Type B ClassII

Protection against electric shock

-Class II Equipment

-Type BF applied parts:
-Mouth piece & mask.

Protection against harmful ingress of water

- IPX0 (Ordinary)

Degree of safety in the presence of flammable anesthetics or oxygen

- No AP/AG (Not suitable for use in the presence of flammable anesthetics or oxygen)

Mode of operation - continuous.

7. CONTACT US

If you have any question about the product, please contact us. We will reply you within 24 hours.

**Authorized Representative:**

MedPath GmbH, Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

**Dongguan Aidisy Machinery & Electronic Equipment Co., Ltd.**

ADDRESS: Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu Dongcheng District, Dongguan City, Guangdong Province, 523118, P.R.China.

Importuotojas: Beauty Factor Sp. z o.o. Plac Bankowy 2, 00-095 Warsaw, Poland

8. UE Declaration of Conformity:

DongguanAidisy Machinery & Electronic Equipment Co.Ltd.

File No.: AIDISY-DCN-CE-02

Version: V5.0

UE Declaration of Conformity

Manufacturer:	Whose Single Authorized Representative:
Dongguan Aidisy Machinery & Electronic Equipment Co., Ltd.	MedPath GmbH
Address:	Address:
Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu, Dongcheng District Dongguan City, Guangdong Province, 523118 P.R. China	Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany
SRN: CN-MF-000026848	SRN: DE-AR-000000067

We, the manufacturer, declare at our sole responsibility that following products

Product name	Model	Basic UDI-DI	Intended purpose
Compressor Nebulizer for Conscious Patient	MCN-S600A, MCN-S600B, MCN-S600C, MCN-S600D, MCN-S600E, MCN-S600F, MCN-S600G, MCN-S600MA, MCN-S600MB, MCN-S600MC, MCN-S600MD, MCN-S600ME, MCN-S600MF, MCN-S600MG, MCN-S600MH, MCN-S600MI, MCN-DS600A, MCN-DS600B, MCN-DS600C, MCN-DS600D, MCN-DS600E	Basic UDI-DI: 69571303MCN0001 GX	The Compressor Nebulizer is intended to provide air to the pneumatic nebulizer in order to aerosolize medications for inhalation by the patient for respiratory disorders

meet the provisions of Regulations (EU) 2017/745,

The medical device has been assigned to class IIa according to rule 20 in Annex VII of MDR 2017/745. It bears the mark

The product concerned has been designed and manufactured under a quality management system according to Annex IX of Regulations (EU) 2017/745.

Compliance of the designated product with the Annex IX of Regulation (EU) 2017/745 has been assessed and certified by the Notified Body

SGS Belgium NV
SGSHouse Noorderlaan
87 2030 Antwerp Belgium

Certificate No.:	Certificate No.: CN19141056
Issue date:	SGS Belgium NV Notified Body 1639
Expiry date:	24 MAY 2024

The above mentioned declaration of conformity is exclusively
under the responsibility of

Company:	Dongguan Aidisy Machinery & Electronic Equipment Co., Ltd
Address:	Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu, Dongcheng District Dongguan City, Guangdong Province, 523118 P.R. China

2024/05/01

Place, date

Legally binding signature, Function

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MedPath GmbH, Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

Dongguan Aidisy Machinery & Electronic Equipment Co., Ltd.

ADRES: Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu Dongcheng District, Dongguan City, Guangdong Province, 523118, P.R.China.

Uvoznik: Beauty Factor Sp. z o.o. Plac Bankowy 2, 00-095 Warszawa, Poland

07. EU IZJAVA O SUKLADNOSTI:

Dongguan Aidisy Machinery & Electronic Equipment Co.Ltd.

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Za gore navedenu izjavu o sukladnosti isključivu odgovornost snosi:

Tvrtka:	Dongguan Aidisy Machinery & Electronic Equipment Co., Ltd
Adresa:	Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu, Dongcheng District Dongguan City, Guangdong Province, 523118 P.R. China

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