
# Momini SippyPump

*Create happy beginnings*

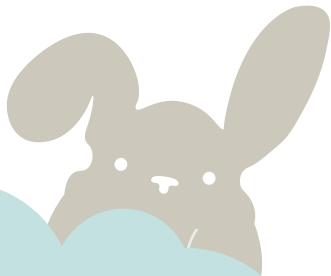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#### 01. SAFETY:

Before using the product, please read this user manual and follow the instructions contained therein when using the device.

#### 02. DEVICE DESCRIPTION [PAGE 2 - A]:

1. Device / 2. Membrane cover (PP) / 3. Membrane (Silicone) / 4. Breast shield (PP, 25mm) / 5. Shield insert (Silicone, 21mm) / 6. Bottle (PP) / 7. Bottle lid (PP) / 8. Teat (Silicone) / 9. Bottle lid (PP) / 10. USB charging cable.

#### 03. BUTTON INSTRUCTIONS [PAGE 2 - B]:

1. On/Off: Long press to turn on/off. Short press to pause. The pump will automatically turn off after 10 minutes of pause.
2. Mode change button: Default multi-function mode "M" every time the device is turned on. Short presses to switch modes.
3. Suction adjustment buttons: There are 9 suction levels in each mode. Short presses of the "+" button increase and "-" button decrease suction level.
4. Multi-function mode: Variable frequency to get more milk.
5. Massage mode: Gentle suction simulation to get milk flow.
6. Express mode: Slow and deep suction for complete emptying.
7. Suction simulation mode: Simulate baby's natural sucking.

#### 04. HOW TO ASSEMBLING THE BREAST PUMP [PAGE 2 - C]:

Wash your hands and make sure the device is clean.

1. Align the diaphragm valve with the valve groove on the breast shield.
2. Insert the diaphragm cover axle into the breast shield, then turn the diaphragm cover over the breast shield and fasten.
3. Install the assembled breast shield onto the motor, then slide the collar into the breast shield.
4. Screw the bottle onto the motor, making sure it is in place.
5. Do not exceed the maximum capacity of the milk bottle. If you reach the maximum capacity, immediately turn off the pump, replace the bottle, and then continue pumping.

#### 05. HOW TO DISASSEMBLE THE BREASTS [PAGE 2 - E]:

1. Remove the breast shield assembly from the motor.
2. Press the two clasps on the diaphragm cover towards the center and fold it upwards.
3. Detach the diaphragm from the breast shield and close the bottle tightly with the lid, then store in a safe place.

#### 06. CHARGING [PAGE 2 - D]:

Fully charge the product before first use. Insert the plug into the Type-C socket on the device and connect the USB port to a computer or adapter. The device must be turned off while charging. A flashing indicator light means that the battery is charging, and a steady light means that the device is charged. Then, disconnect it from the power supply.

#### 07. INSTRUCTIONS FOR USE [PAGE 2 - F]:

1. Make sure that all parts of the breast pump are clean/sterilized and that the device has been assembled correctly (follow the instructions in this instruction manual)
2. Wash your hands and breasts.
3. Sit up straight and lean slightly forward. Maintain a correct posture while pumping. DO NOT lie on your side, and do NOT tilt the breast pump to avoid milk backflow.
4. Place the center of the collar on the nipple and press it against the breast to ensure regular suction and prevent air from entering.
5. Long press the start button to adjust the mode and suction level according to your preference.

#### 08. WARNINGS:

1. This breast pump will automatically turn off after 30 minutes of operation.
2. All parts must be dry before using the device to prevent water from damaging the motor.
3. If the product is not used for a long time, the battery will run out. Please plug it in, wait until the screen lights up, indicating that it is fully charged.
4. The pump motor and other PVC electrical parts cannot be washed or sterilized. If they are dirty, clean and dry them. Make sure there is no water drop in the tube before continuing to use.
5. After sterilizing the membrane, please dry it before use.
6. When the battery light flashes after starting, it means that the charge level is insufficient. It should be charged.

7. Only use the charging cable provided by the manufacturer.

8. If you have problems with lactation or pain, consult your doctor before using the breast pump.
9. Do not fill the bottle to the brim; this may cause milk to leak out of the side holes of the pump body. If it is overflowed, turn off the pump, replace the bottle and continue pumping.
10. After pumping, make sure that there is no milk residue in the breast pump parts after drying. If you do not need to feed your baby immediately, store the milk in the refrigerator.
11. Store the product in a dry place, at room temperature, out of the reach of children. Avoid direct sunlight on the product. The product is not a toy and is not suitable for use by children.

#### ● Battery Specification: 3.7V, 2000mAh

Voltage: 5V  
Power: 5W  
Computer USB Port: DC5V/1A  
Electric Power Adapter: Input: AC100-240V/50Hz-60Hz; Output: DC5V/1A  
USB Power Bank: DC5V/1A

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**[EN] Rules for handling electrical and electronic equipment:** Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m<sup>2</sup> devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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**[EN] Short declaration of conformity:** The manufacturer declares, under its sole responsibility, that the SpicyPump device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, Directive 2014/35/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the making available on the market of electrical equipment intended for use within specified voltage limits, Regulation (EC) No 1935/2004 of the European Parliament and of the Council of 27 October 2004 on materials and articles intended to come into contact with food and repealing Directives 80/590/EEC and 89/109/EEC, Commission Regulation (EU) No 102011 of 14 January 2011 on plastic materials and articles intended to come into contact with food, together with Commission Regulation (EU) 2020/1245 of 2 September 2020 amending and correction of Regulation (EU) No 102011 on plastic materials and articles intended to come into contact with food, and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in equipment electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive of the European Parliament and of the Council (EU) 2017/2102 of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at [www.momini.eu/deklaracje](http://www.momini.eu/deklaracje)

[www.momini.eu/deklaracje](http://www.momini.eu/deklaracje)  
 Importer: Beauty Factor Sp. z o.o., Plac Bankowy 2, 00-085 Warszawa, Polska  
 Model: SpicyPump/ SpicyPump Twin  
 SN: 04255906601060656/ SN: 04255906601060663  
 Marka: Momini  
 E-mail: [contact@momini.eu](mailto:contact@momini.eu)

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**EN |** More information about the product is available on the manufacturer's website at:

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