LOLA & LYKKE®

SMART ELECTRIC  
**BREAST PUMP**

ELEKTRISCHE  
**MILCHPUMPE**

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## INSTRUCTIONS FOR USE

Lightweight, quiet, wireless pump with USB-C charger – perfect for mums on the go. Mother's milk is the best nutrition for babies. The Lola&Lykke® Smart Electric Breast Pump gives you and your baby optimal support during the breastfeeding period.

 Lola&Lykke Oy  
Konepajankuja 1  
00510 Helsinki  
Finland

EN 14350:2020

- EN Smart electric breast pump**  
Instruction for use
- CZ Chytrá elektrická odsávačka mléka**  
Návod k použití
- DA Smart elektrisk brystpumpe**  
Brugsanvisning
- DE Intelligente elektrische Milchpumpe**  
Gebrauchsanweisung
- ES Extractor de Leche Eléctrico Inteligente**  
Manual de instrucciones
- ET Nutikas elektriline rinnapump**  
Kasutusjuhend
- FI Sähkökäyttöinen rintapumppu**  
Käyttöohjeet
- FR Tire-lait électrique intelligent**  
Instructions d'utilisation
- IT Tiralatte elettrico intelligente**  
Istruzioni per l'uso
- LT Išmanaus elektrinio pientraukio**  
Naudotojo vadovas
- LV Viedais elektriskais piena pumpis**  
Lietošanas instrukcija
- NL Slimme elektrische borstpomp**  
Gebruiksaanwijzing
- NO Smart elektrisk brystpumpe**  
Bruksanvisning
- PT Extrator de Aleitamento Elétrica Inteligente**  
Instruções de utilização
- SE Smart elektrisk bröstpump**  
Instruktioner För Användning

# 1 Your Product

EN

Thank you for choosing the Lola&Lykke® Smart Electric Breast Pump. Breast milk is the natural choice for your baby with a multitude of well documented health benefits. Although breastmilk is always best, providing breastmilk for your baby when you need to be away is not always easy. Our Smart Electric Breast Pump was designed to provide a modern and convenient way for mothers to express breast milk.

Our breast pump's 2-Phase Expression Technology mimics the natural sucking pattern of the baby. The automatic switch between functions and a memory button for easy tracking and recording of pumping sessions makes breastfeeding with our pump effortless. The super soft silicone breast shield helps create effective suction resulting in comfortable and fast milk flow. Our breast pump includes an anti-colic, slow flow teat which allows for your baby to replicate natural sucking actions learned at the breast when drinking expressed breastmilk through a bottle.

Exclusive breastfeeding is recognised as the best choice for your baby and provides all the nutrients your baby requires for the first 6 months of life. Breastfeeding helps to develop a strong bond between mother and baby and provides numerous long terms benefits for both.

## 2 Important Safeguards

When using electrical products, especially when children are present, basic safety precautions should always be followed. This product has been tested according to EN 14350:2020 Child care articles – Drinking Equipment.

### READ ALL INSTRUCTIONS BEFORE USE

The defined signal words identify all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breast pump or yourself. When used in conjunction with the following words, the defined signal words stand for:

-  **WARNING** Can lead to serious injury or death.
-  **CAUTION** Can lead to minor injury.
-  **NOTE** Can lead to material damage.
-  **TIP** Useful or important information that is not related to safety

### **WARNING : For your child's health and safety (important drinking equipment safety measures):**

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never use feeding teats as a soother.
- Always use this product with adult supervision.
- For use with breast milk only.
- Unsuitable common methods of heating, cleaning (including where applicable unsuitable common cleaning agents), storage and use might damage the product.
- Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant (sterilising solution) for longer than recommended, as this may weaken the teat.

**WARNING: To avoid fire, electrocution, or serious burns:**

- Always unplug electrical product immediately after use except for when charging.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- The breast pump and accessories are not heat-resistant: keep away from heated surfaces or open flames.
- Do not use near flammable materials.
- The breast pump should never be left unattended when plugged into a power source.
- Never operate an electrical device if it has a damaged cord or plug, is not working properly, or if it has been dropped or damaged. If damage is found, immediately discontinue use.
- Do not use an electrical outlet device that has been exposed to water or other liquids including:
  - Do not use while bathing or showering.
  - Never place or drop into water or other liquids.
  - Do not run water over breast pump.
  - If a device has been exposed to water or other liquids, do not touch, unplug the device from electrical outlet, turn off and contact manufacturer.


**WARNING: To avoid health risk and reduce the risk of injury:**

- This is a single-user product. Use by more than one person may present a health risk and void the warranty.
- This device cannot be serviced or repaired. Do not repair yourself. Do not modify the device.
- Use the breast pump only for its intended use as described in this manual.
- Inspect all appropriate pump components before each use.
- Do not microwave or boil breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breast milk.)
- Clean and sanitize all parts that come into contact with your breast and breast milk prior to first use.
- Wash all parts that come into contact with your breast and breast milk after every use.
- Close supervision is necessary when the breast pump or accessories are used in the vicinity of children.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

**Your breast pump system includes**


- A - Cap
- B - Breast shield
- C - Connector
- D - Motor Unit
- E - Suction Bowl
- F - Valve
- G - Milk Bottle
- H - Slow flow teat
- I - Bottle base
- J - Bottle cap set
- K - Charging cable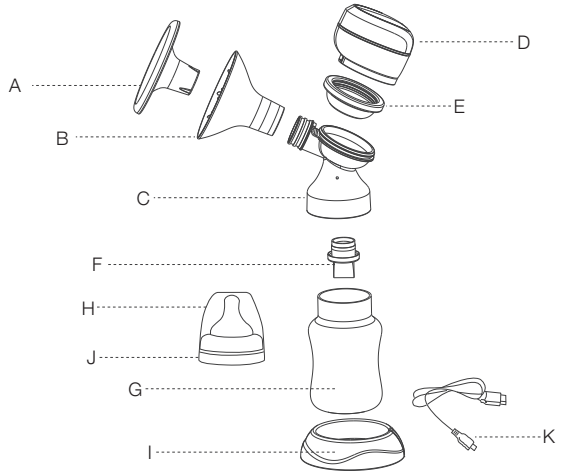

**Does not contain Bisphenol-A, DEHP (Phthalate) or Latex**

Power in: 110-240V 50-60Hz      Power Out: 5V 1A  
 Size: 81x63x67mm      Weight: 650g  
 Battery Capacity and Type: 3.7V 1400mAh Li-ion

**Display Icons**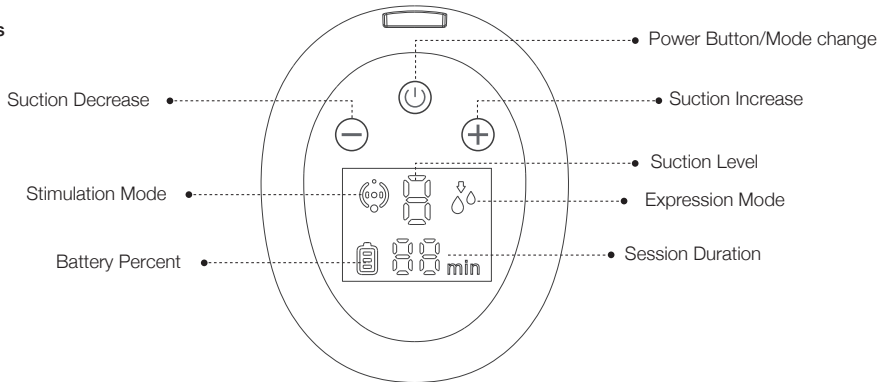

- Power Button: Press 2 seconds to start or to turn off the breast pump.
- Mode Change: Tap the screen to change mode.  
 Initial set-up: The initial stimulation mode will automatically change to expression mode after 2 minutes of use. Digital display features a memory button and timer for easy tracking and recording pumping sessions. When switched on, the pump's memory will automatically use the suction settings from the previous pumping session. The pump will automatically switch off after 30 minutes use.
- Suction Level: + suction increase - suction decrease (Stimulation Mode: L1-L6 Expression Mode: L1-L9).
- Battery Percent: Indicates remaining battery life. Please charge breast pump when low in battery.
- Session Duration: Indicates the length of your pumping session. The pump will automatically switch off after 30 minutes use.


## 4 Getting Started

EN

### Note

You should do the following before using your breast pump for the first time:

- 1 . Plug the breast pump into an external power source. Charge the battery for 2 hours.
- 2 . Separate all parts that are exposed to the breast or breast milk prior to cleaning.
- 3 . Clean the pump – see section 6

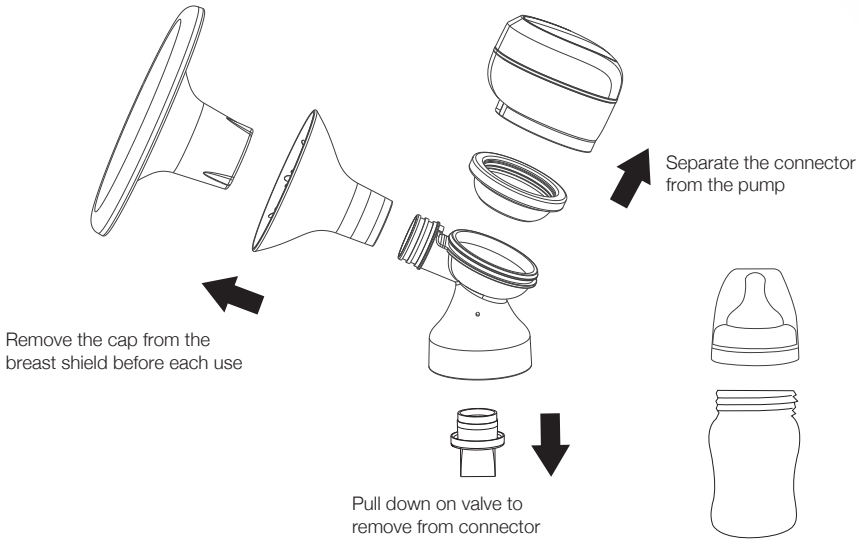## 5 Cleaning Overview

Washing and sanitizing are two different activities. They must be done separately to protect you, your baby and the performance of your breast pump.

**Wash** – To clean the surfaces of the parts by physically removing contamination.

**Sanitize** – To kill living organisms, such as bacteria or viruses, that may be present on the surfaces of the parts.

When to Wash	Breast Shield 	Breast Pump Kit 	Bottle & Cap 
Before 1st use	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> <li><input checked="" type="checkbox"/> sanitize</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> <li><input checked="" type="checkbox"/> sanitize</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> <li><input checked="" type="checkbox"/> sanitize</li> </ul>
After each use	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> wash</li> </ul>
Once per day	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> sanitize</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> sanitize</li> </ul>	<ul style="list-style-type: none"> <li><input checked="" type="checkbox"/> sanitize</li> </ul>

**Supplies Needed:**

- Mild dish soap
- Clean dish towel or soft brush
- Clean wash basin
- Drinking-quality water
- Clean pot for boiling water

**Parts to wash or sanitize:**

- Breast shield
- Milk bottle
- Bottle cap
- Connector
- Valve


**Caution**

- Separate and wash all parts that are exposed to breast milk immediately after use. This will help to remove breast milk residue and prevent growth of bacteria.
- Only use drinking-quality tap or bottled water for cleaning.
- When cleaning the valves do not use small objects such as brushes. Be careful not to puncture.
- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk except motor unit.


**2. Wash**  
Before first use and after use

Sink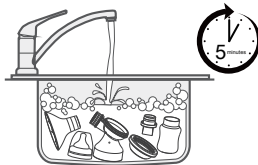

- Step 1 – Rinse all separated parts that touch the breast and breast milk in cool water to remove breast milk residue.
- Step 2 – Soak in warm soapy water for 5 minutes, then wash and rinse.
- Step 3 – Allow parts to air dry. Store dry parts in a cool place when not in use.

**1. Disassemble**

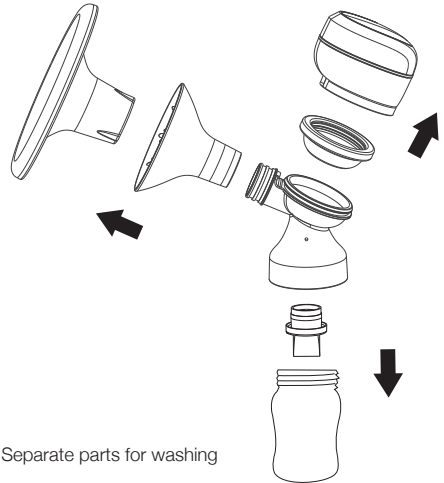Separate parts for washing

**Note**

- Take care not to damage parts of the breast pump kit during cleaning.
- Store the dry breast pump kit in a clean bag/container until next use.
- Wash hands thoroughly.

**3. Sanitize**  
Before first use and once per day

Stovetop

- Step 1 – Fill pot with water to cover all parts. Bring to a boil.
- Step 2 – Place parts in boiling water for 10 minutes.
- Step 3 – Allow parts to air dry. Store dry parts in a cool place when not in use.

## 7 Motor Unit


EN

Supplies needed:

- Mild dish soap
- Clean dish towel
- Clean sink or bowl
- Drinking-quality water

Parts needed:

- Motor unit

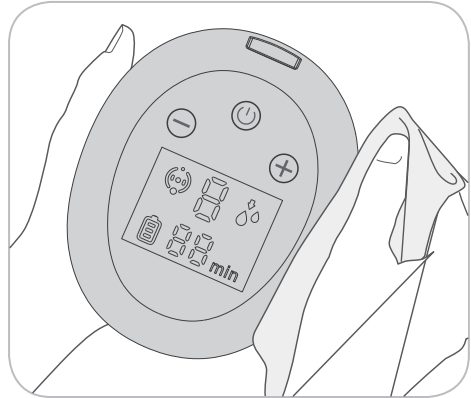
Wipe pump unit with a clean, damp cloth.

### Warning

- Do not immerse the motor unit in water; do not run water over the motor unit.
- When cleaning the motor unit display, only use a damp, soft cloth with mild dish soap. Other cleaners or abrasive cloths may degrade the display performance.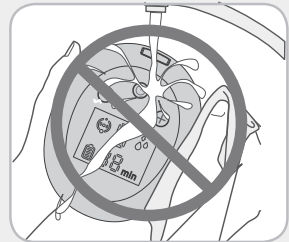

## 8 Breastfeeding Information

### Common pumping questions & answers

#### How often should you breast pump?

A breast pump is a replacement for when you are separated from your baby. It is important to pump when the baby would normally be breastfeeding. For example, a working mother may pump 2-3 times during an 8-hour work day.

#### How long should your pumping session last?

Pumping times can vary from mother to mother. Pumping sessions usually last from 15 to 30 minutes.

#### How should your breasts feel before and after pumping?

Before pumping, your breasts will have a firm, heavy feeling. After pumping, your breasts should feel soft with no firm areas. Firm areas may indicate that the breast is not completely drained.

## 9 Assembling Your Breast Pump


EN

### Parts Needed:

- Motor unit
- Breast shield
- Milk bottle
- Suction bowl
- Connector
- Valve


### Tip

- Check breast pump kit components for wear or damage before use. Replace if necessary.
- Always inspect all parts prior to use for cleanliness.
- To prevent damage to the breast pump, all components must be completely dry before use.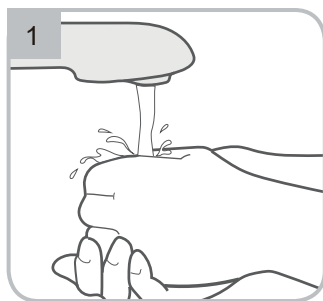


### Tip

Wash hands thoroughly with soap and water before touching breast pump, breast pump kit and breast.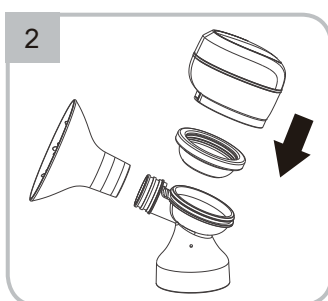


### Tip

Press the suction bowl and motor unit into the connector.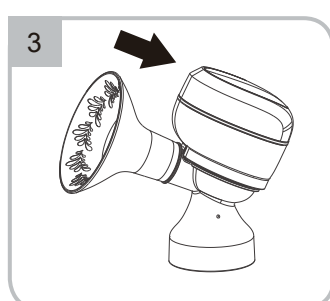


### Tip

Push the breast shield onto the connector.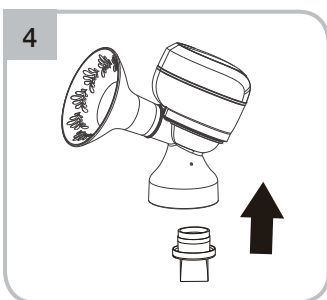

### Tip

Put the valve into the connector.

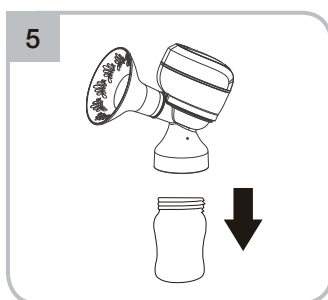### Tip

Lock the milk bottle into the connector by rotating clockwise until tight.

## 10A. Rechargeable Battery

- Before first use, charge the breast pump battery for 2 hours by connecting the pump to a power source with the USB-C connector. The battery icon  will appear fully charged on the display when charging is complete.
- Proceed to Section 11, Operating Your Breast Pump, to begin your pumping session.

## 10B. Battery Care

Your breast pump contains a lithium-ion rechargeable battery.

### Tip

- Before first use, fully charge your breast pump for 2 hours.
- You can use your breast pump while charging the battery. The battery will charge as soon as the breast pump is connected to a power source. It will not charge while the pump is running.
- Keep your pump in a cool place.
- Keep your battery charged.
- Fully charge the battery before storing your pump for an extended period of time.
- Recharge the battery before it completely drains. This is better for the lifecycle of the battery.
- If your breast pump has been stored in a hot location, it may not run on battery power right away. To resume normal battery function, allow the breast pump to cool for one hour. During this time, you can use your breast pump when plugged into external wall power.

## Battery Charge Status Codes





Battery charge status/Low battery, needs charging

(flashing)

Fully charged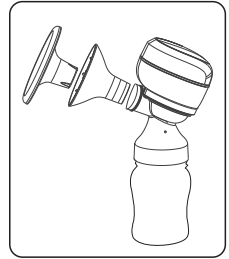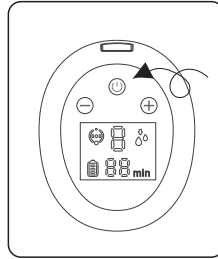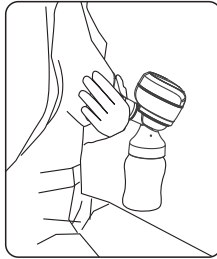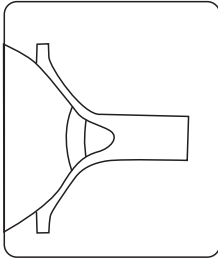

1

Place the breast shield on your breast so that your nipple is properly centered in the suction tunnel.

2

Hold the breast shield and connector on to your breast with your thumb and index fingers. Support your breast with the palm of your hand.

3

Turn on the breast pump by tapping the power button and choose the mode you prefer.

4

Turn off the breast pump by tapping the power button after using and place the dust cap over the breast shield.

**Notes:**

1

Your breast pump will begin in Stimulation Phase . Adjust the pumping level using the   controls to find a comfortable level.

**Stimulation Phase** = a fast pumping rhythm that mimics the baby's natural sucking pattern to stimulate milk flow.

2

After 2 minutes, your pump will automatically switch to Expression phase. . Adjust the pumping level using the   controls to find the maximum level of comfortable vacuum for you. If your milk begins to flow before 2 minutes, press the mode change button  to manually switch to expression mode.

**Expression Phase** = a slower pumping rhythm for gentle and efficient milk collection after milk has started flowing. 

3

Once you are pumping in the Expression Phase, increase the speed/vacuum  until pumping feels slightly uncomfortable (not painful), then decrease slightly.

## 12 Product features

EN

**Intuitive Smart Capabilities:** 6 Stimulation levels and 9 Expression levels  
**2-Phase Expression Technology:** Stimulation/Expression. Two clinically tested pumping rhythms for personalized comfort. **4-Phase Smart Functionality:** Stimulation/Expression/Automatic/Memory.

**Anti-back Flow Design for Hygiene**  
Closed system to prevent breastmilk from backing up to the pump, prevent infection, and keep the milk fresh.

**Super Soft Silicone Breast Shield**  
The flexible, ultra-soft silicone breast shield feels comfortable, reduces pressure and helps massage the breast to promote letdown and optimize milk flow.

**Quiet and Lightweight**  
The quiet breast pump allows for discreet pumping anywhere. Lightweight, portable pump. Designed for mums on the go who pump several times a day.

**Easy to Clean and Assemble**  
The pump has a mere five parts to assemble and clean, making it very easy to use.

**Smart Touch Screen Technology**  
Convenient pumping experience. Digital display features a memory button and a timer for easy tracking and recording pumping sessions.

**USB-C Rechargeable Battery**  
Durable lithium-ion battery with 2-hour battery life. Micro USB-C port with standard USB-C cable for connecting to power source.

**Streamlined Wireless Design**  
The pump has been designed with style in mind. No tubes, no wires – easy to use anywhere.

**BPA-free Compatible Pump**  
Pump and accessories are made from BPA-free materials. Pump is compatible with standard baby bottles.

**Anti-Colic Slow Flow Baby Bottle Teat**  
An advanced BPA-free anti-colic bottle teat with a slow flow rate suited to a newborn is provided with the pump.

## 13 Storing Breast Milk

- As soon as you are finished pumping, place the breast milk bottle or breast milk storage bag in a cool place like a cool bag if not at home.
- Transfer breast milk bottles or breast milk storage bags to the refrigerator or freezer once you are home.

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly Expressed Breast milk	Maximum 4 hours at up to 25°C / 77°F	3-5 days at approx. 4°C	6 months at approx. -16°C	12 months at approx. -18°C
Thawed Breast Milk (previously frozen)	1 - 2 hours at up to 25°C / 77°F	24 hours at approx. 4°C / 40°F	Never refreeze	

- When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion. Label bottles/bags with the date of pumping.
- In order to preserve the components of the breast milk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).
- Gently shake the bottle/bag to blend any fat that has separated.
- Do not thaw frozen breast milk in a microwave oven or in a pan of boiling water (danger of burning)

## 14 Preparing & Feeding Breast Milk

EN

### Preparing Breast Milk

#### Caution

- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- Do not microwave breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breast milk.)
- Thaw breast milk overnight in the refrigerator. Thawed breast milk is safe in the refrigerator for 24 hours. Do not refreeze thawed breast milk.
- Quickly thaw breast milk by holding the bottle or bag under warm running water.
- Place the sealed bottle or bag in a bowl of warm water for 20-minutes to bring it to body temperature.
- If you are adding expressed breast milk to a container of already frozen breast milk, make sure to add a lesser amount than the already frozen amount.

### Feeding Breast Milk

- It is recommended that breastfeeding is well established prior to bottle feeding your baby.
- Always inspect the bottle, nipple and other components immediately before and after each use. If nipple appears cracked or torn, discontinue use immediately.
- To prevent possible choking hazard, test strength of nipple by pulling on bulb portion of the nipple.
- Do not attempt to enlarge the nipple hole.
- Infants must not be bottle fed without adult supervision.
- Nipple should not be used as a pacifier.
- Baby bottle feeding nipple teat type: Slow flow anti-colic 

## 15 Meaning of Symbols

### Glossary of Symbols

-  Power Button/Mode Change
-   Suction Decrease/Increase
-  Stimulation Mode
-  Expression Mode
-  Suction Level
-  Battery Percent
-  min Session Duration
-  This symbol indicates do not dispose the device together with unsorted municipal waste (in accordance with local regulations).
-  Symbol showing that the manufacturer has taken all necessary measures to ensure that the product complies with the applicable safety legislation for distribution within the European Community.

EN 14350: 2020 This symbol means that the product complies with the European standard for Child Care Articles: Drinking Equipment for children up to 48 months.

## 16 Troubleshooting

EN


<p>If breast pump does not power on</p>	<ul style="list-style-type: none"> <li>• Check that the breast pump is plugged into a working electric outlet.</li> <li>• If there is no power, display does not light up, and there is no error alert appearing.</li> </ul>
<p>If there is low or no suction</p>	<ul style="list-style-type: none"> <li>• Check that all of the breast pump and kit connections are secure.</li> <li>• Disassemble the pumping kit and check that all pieces are clean and not damaged.</li> <li>• Assemble the pump set as directed in Assembling your Breast Pump (Section 9).</li> </ul>
<p>If there is a breast milk overflow</p>	<ul style="list-style-type: none"> <li>• Turn off the breast pump and unplug the power cord from the electrical outlet.</li> <li>• Use a damp (not wet) cloth to wipe the breast pump.</li> <li>• When all components are dry, reassemble.</li> </ul>

## 17 Long-term Storage


Your breast pump contains a rechargeable battery. To preserve your breast pump, when not in use for an extended amount of time, it will automatically transition into storage mode.

To remove from storage mode, plug your breast pump into an external power source and fully charge the battery for up to 2 hours.


**Before storing your breast pump for an extended amount of time remember to:**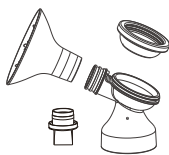

Clean breast pump kit  
(Section 6)

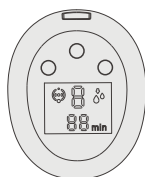Clean motor unit  
(Section 7)

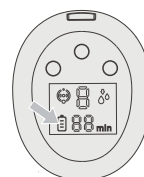Make sure breast pump battery is fully-charged

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