# AngelSounds JPD- 100S9

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Voděodolnost: IP22

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## Product Introduction


AngelSounds JPD-100S9 is a lightweight and portable detector for listening to the heartbeats of a child's fetus.

*Note: For proper functioning, the fetus must be at least 16 weeks old. Consult your doctor about the use of the device.*

## Package Contents

AngelSounds JPD-100S9, gel, jack-jack cable, headphones, battery, manual

## Basics of use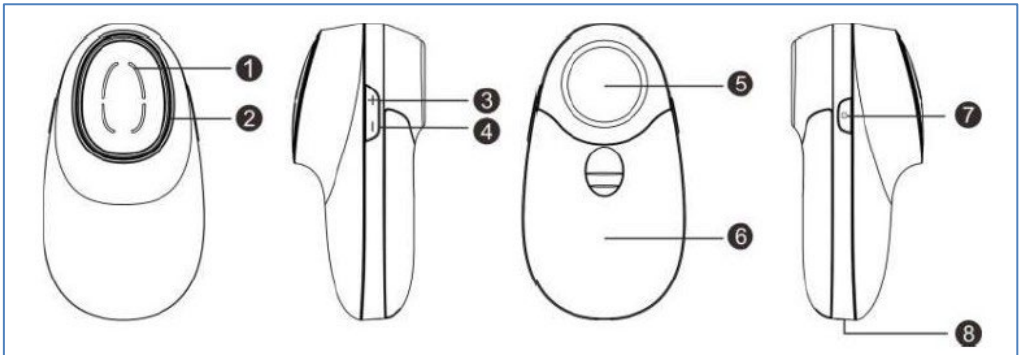

1 - Loud speaker 2 - Status indicator

3 - Adjust the volume


4 - Volume adjustment 5 - Signal transmission 6 - Battery cover 7 - Power on/off

8 - Connection for recording cable or headphones



To turn on the device, press the button (7), the LED lights up (2).

You can listen to heart sounds either from a loud speaker or by connecting headphones to the appropriate jack (8).

Audio can also be recorded. Connect your device and mobile phone with a cable and start audio recording.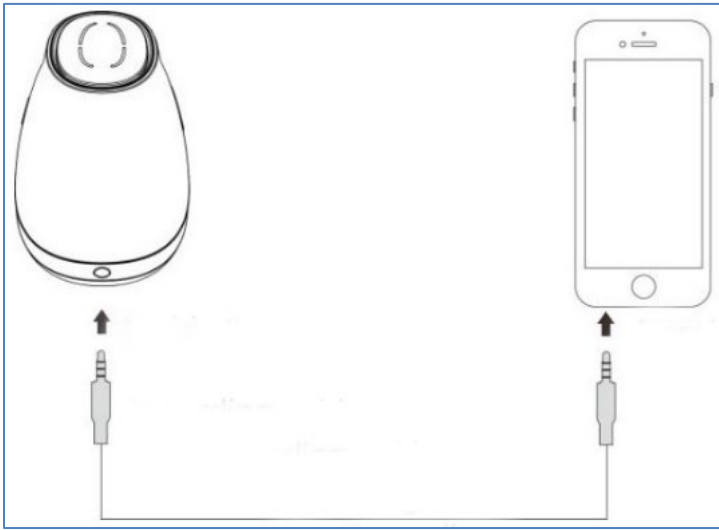

## Battery

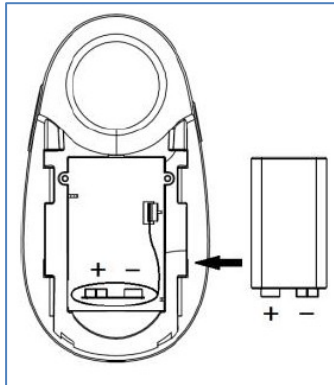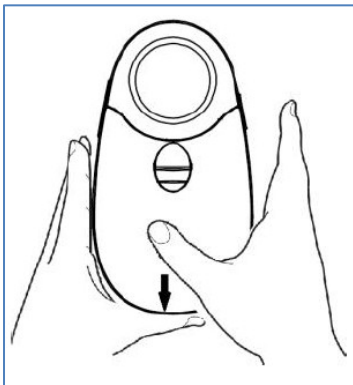
Battery included. It is located directly on the device, but is not connected. Before first use, the battery must be connected as shown in the following picture. Remove the cover (6) and connect the

battery.

## How to find the heart of the fetus

Apply a little gel to the probe (5) and place it on your stomach. Move it slowly, trying to find the heart of the fetus. After finding the sound, continue to move the probe slowly, looking for the place where the sound signal will be clearest. The usual position of the fetus can be seen in the following picture. However, the position of the fetus may vary depending on the stage of pregnancy and other conditions.

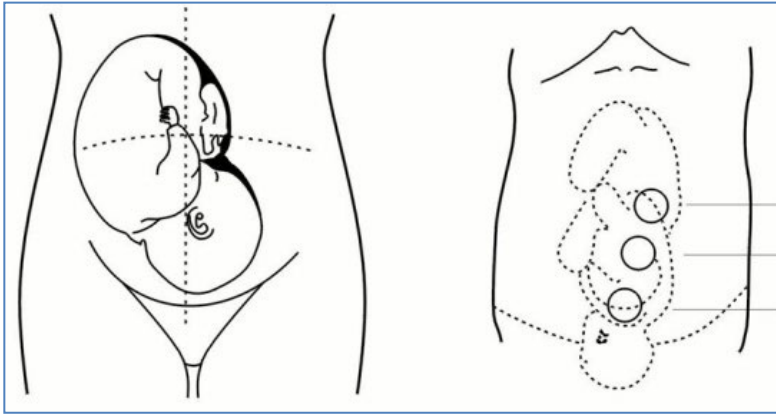## Safety Warnings

Avoid using the device in an environment with a potential risk of explosion, high or very low temperature or humidity, near an open flame. We recommend that you do not use the device outdoors.

Always wipe off the gel from the device after monitoring.

Do not use the device if it shows signs of damage!

If you're cleaning your device, always keep it turned off.

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