AngelSounds JPD-200S

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Voděodolnost: IP22

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Product Introduction


AngelSounds JPD-200S is a lightweight and portable detector for listening to the heartbeats of a child's fetus.

Note: For proper functioning, the fetus must be at least 20 weeks old. Consult your doctor about the use of the device.

Package Contents

AngelSounds JPD-200S, jack-jack cable, headphones, battery, manual

Basics of use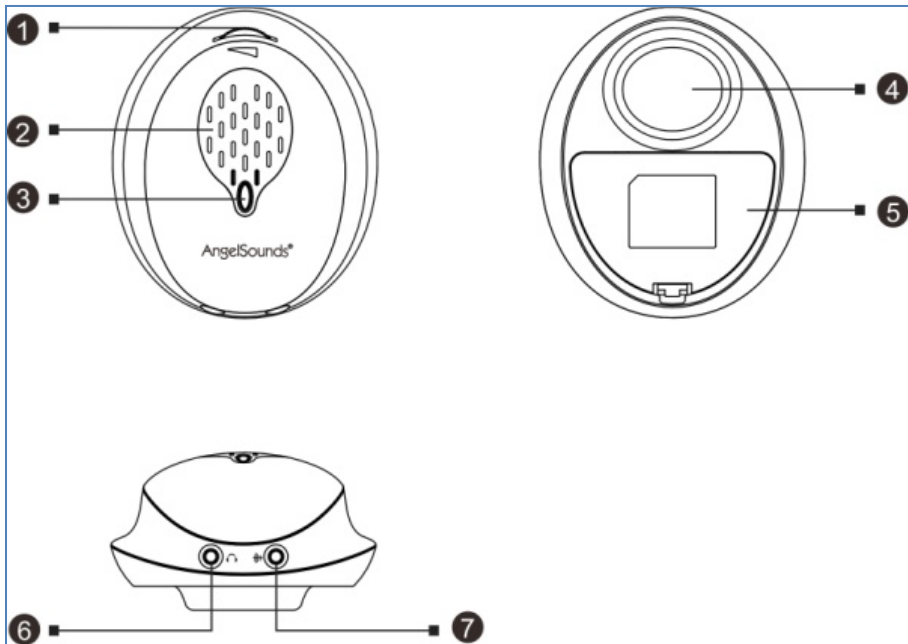

1 - Power on/off; Volume Adjustment

2 - Loudspeaker

3 - Status indicator

4 - Signal transmission

5 - Battery cover

6 - Headphone connection


7 - Connection for recording cable

Turn on the device by turning the wheel (1), the LED lights up to turn it on.

Gradually turning the wheel adds volume, turning to the opposite side gradually reduces the sound until the device is completely turned off.


You can listen to heart sounds either from a loud speaker or by connecting headphones to the appropriate jack (6).

Audio can also be recorded. Connect your device and mobile phone with a cable and start audio recording.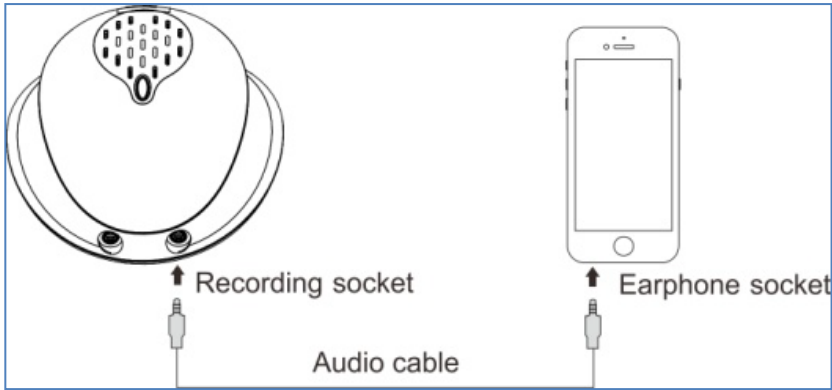

Battery

Battery included. It is located directly on the device, but is not connected. Before first use, the battery must be connected as shown in the following picture. Remove the cover (5) and connect the battery.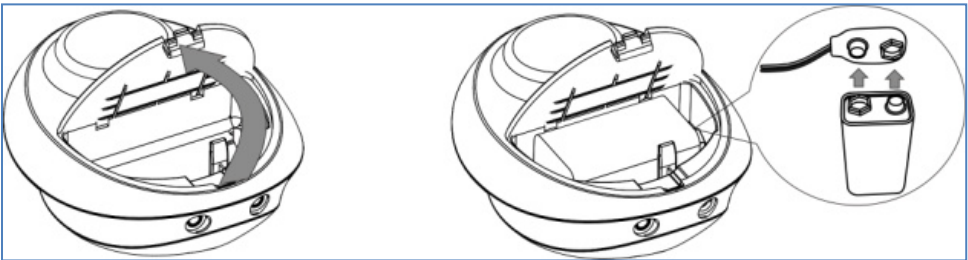


Indicator statuses

Indicator status	Operational status
Lights up green	The device is turned on
Flashing green	The device is turned on and captures the sound of the fetus
Flashing orange	Device needs to be recharged

How to find the heart of the fetus

Apply a little gel to the probe (4) and place it on your stomach. Move it slowly, trying to find the heart of the fetus. After finding the sound, continue to move the probe slowly, looking for the place where the sound signal will be clearest. The usual position of the fetus can be seen in the following picture. However, the position of the fetus may vary depending on the stage of pregnancy and other conditions.

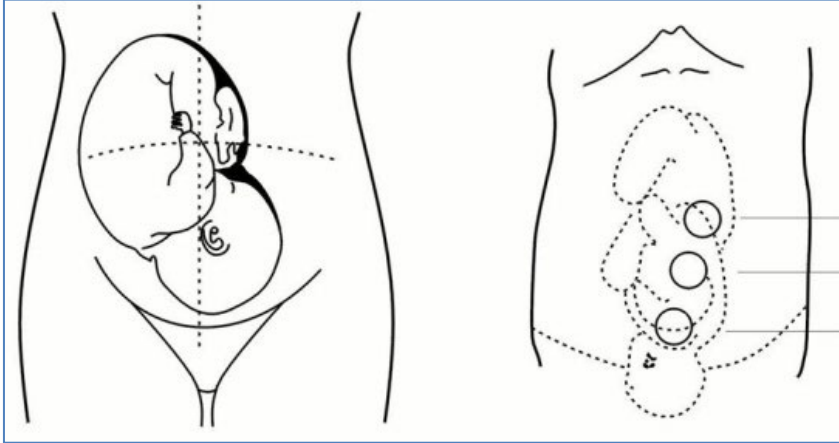Safety Warnings

Avoid using the device in an environment with a potential risk of explosion, high or very low temperature or humidity, near an open flame. We recommend that you do not use the device outdoors.

Always wipe off the gel from the device after monitoring.

Do not use the device if it shows signs of damage!

If you're cleaning your device, always keep it turned off.

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