Instructions for use
Spare Teats

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Spare Teats

Ideal for feeding breast milk

Note: Only use Medela original accessories.

Make sure breastfeeding is well established before introducing a teat.

Cleaning and Hygiene

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts when using a brush.
- Inspect all items before each use.
- Replace parts if damaged or worn out.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Food colorings may discolor components which will not affect performance.
- Follow these cleaning instructions unless you are told otherwise by your healthcare professional.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant („sterilizing solution“) for longer than recommended, as this may weaken the teat.

Prior to first use and once daily

- Disassemble, wash and rinse all parts.
- Place parts in a saucuppan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

As an alternative to boiling:

Dishwasher: Clean parts in top rack.

Microwave: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.

Steam sterilizer: Follow the manufacturer's instructions.

After each use

- Disassemble and wash all parts in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.
- After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- Do not place in direct sunlight.
- Avoid touching the inside of bottles and lids.

For your child's safety and health

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soothe.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding
- Keep all components not in use out of the reach of children.
- Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material Silicone

Disposal According to local regulations

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