Dear parents,

Congratulations on your decision to purchase Calma. For over 50 years, Medela has pioneered innovative products which help to enhance babies' health through the benefits of breast milk. Your baby will always prefer breastfeeding. However, if you are separated from your baby, Calma provides an excellent alternative to feeding breast milk. With Calma, your baby can maintain its natural feeding behaviour learned on the breast. Any interference with the breastfeeding relationship between the mother and her baby can thus be minimised.

Medela wishes you and your baby all the best!

Instructions for use Calma breast milk feeding solution

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NL

Preparing for feeding

- Wash your hands thoroughly (for approx. 1 min.) with soap and water before you touch Calma.
- All components must be dry before use.

Steps to perform

- Pump and store the breast milk (see the storage guidelines).
- Assemble Calma as described below.
- Warm the breast milk in a milk bottle with the Calma multi-lid on top. You can use Calma in combination with any Medela bottle. Heat the milk to max. 37°C.
- Put Calma on the bottle just before feeding.

Tip: Use the Calma multi-lid on a milk bottle for transport. For storing Calma, place the multi-lid on Calma's base.

Important safety information

Read all instructions before use. SAVE THIS INSTRUCTION MANUAL.

Device

- Only use original Medela accessories.
- Keep Calma away from radiators and open flames.
- Do not expose Calma to direct sunlight.
- Inspect before each use: pull the transparent silicone part in all directions to make sure it is still intact. Throw the part away at the first sign of damage or weakness.

Plastic bottles and component parts become brittle when frozen and may break when dropped.

Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over. Take appropriate care in handling bottles and components. Do not use the breast milk if bottles or components become damaged.

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Use Follow the cleaning instructions. Do not use Calma while sleeping or overly drowsy. This is a single-user product. Use by more than one person may present a health risk.

For your child's safety and health **WARNING!**

- Always use this product with adult supervision.
- Never use feeding teats as a sootheir.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check the food temperature before feeding (risk of burning).
- Keep all components not in use out of reach of children.
- Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Cleaning and hygiene

Clean all parts immediately after use with soapy water to avoid the drying up of milk residues and to prevent growth of bacteria.

- Only use drinking-quality water for cleaning.
- Do not clean Calma with a sharp or hard object (such as a brush, a toothpick or sharp fingernails).
- If you clean Calma in a dishwasher, parts may change in colour. This has no impact on the function.
- To avoid contamination, do not touch the inside of the parts after cleaning.
- Store Calma in an air-permeable, clean bag/container or an air-permeable, clean papercloth until its next use.
- For hygiene reasons, we recommend replacing Calma after 3 months.

Before using for the first time

- Disassemble Calma into its individual parts (see picture 1).
- Rinse all parts with cold water (approx. 20°C), then clean them in a gentle way with plenty of warm soapy water (approx. 30°C). Rinse again with cold water (approx. 20°C) for 10–15 seconds (see pictures 2–4).
- Cover all parts with water and boil for 5 minutes (picture 5a). Alternatively, you could use a Quick Clean Microwave Bag in accordance with the instructions (picture 5b).
- Dry all parts with a clean cloth or leave to dry on a clean cloth (picture 6).

Tip: When you boil the Calma parts, you could add a tea-spoon of citric acid to avoid a build-up of lime scale. Note that minor traces of the acid might remain on the Calma parts. Use drinking-quality water to wash them off.

After each use

- Disassemble Calma into its individual parts (see picture 1).
- Rinse all parts with cold water (approx. 20°C), then clean them in a gentle way with plenty of warm soapy water (approx. 30°C). Rinse again with cold water (approx. 20°C) for 10–15 seconds (see pictures 2–4).
- Dry all parts with a clean cloth or leave to dry on a clean cloth (see picture 6).
- As an alternative to the previous steps, you could clean Calma in the dishwasher. If you do so, place the parts on the top rack or in the cutlery section.

Once a day

- Disassemble Calma into its individual parts (see picture 1).
- Cover all parts with water and boil for 5 minutes (picture 5a). Alternatively, you could use a Quick Clean Microwave Bag in accordance with the instructions (picture 5b).
- Dry all parts with a clean cloth or leave to dry on a clean cloth (picture 6).

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- Take Calma off the bottle (see picture 15).
- Take the top with the silicone part off the base (picture 16).
- Turn the bottom edge of the silicone part upwards. Take the silicone part off the top (picture 17).
- Clean Calma as previously described (see pictures 1–6).

Storing and thawing breast milk

Storage guidelines for freshly expressed breast milk (or breast milk from healthy term babies)

Room temperature	4–6 hours at 19 to 26°C (66 to 78°F)
Refrigerator	3–8 days at 4°C (39°F) or lower (6 to 12 months –18 to –20°C (0 to 4°F)
Freezer	–18 to –20°C (0 to 4°F)

Thawed breast milk
Do not refreeze!

Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (at the back of the glass shelf above the vegetable compartment).

You can freeze expressed breast milk in milk bottles or "Pump & Save" bags. Do not fill the bottles or bags more than ¾ full to allow space for expansion (picture 5b).

Label the bottles or bags with the expiration date.

Do not thaw frozen breast milk or warm breast milk in a microwave/a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

In order to preserve the components of the breast milk, thaw the milk in the refrigerator separately. Alternatively, hold the bottle/"Pump & Save" bag under warm water (max. 37°C).

Gently swirl the bottle or "Pump & Save" bag to mix any fat that has separated. Avoid shaking or stirring the milk.

Material: Bottle, cap, multi-lid: polypropylene. Teat: silicone. Base, top: polypropylene and thermoplastic elastomer.

Disposal: According to local regulations. Conforms to EN 14350.

For more detailed information on the use of Calma, see our **online instructions for use** (www.medela.com/calma).

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