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ELECTRIC BREAST PUMP

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ELEKTRISCHE BORSTKOLF

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EN INSTRUCTIONS FOR USE

ELECTRIC BREAST PUMP

Please read these instructions carefully before use and keep them for future reference.

Intended use and indications for the electric breast pump

SAFETY WARNINGS

WARNING!

- This product is intended for domestic use only by an adult, and must be used in accordance with the instructions for use provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- The appliance is for personal use and must not be sold by the purchaser or lent to other users.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements. The mains adaptor is for indoor use only and should therefore not be exposed to sunlight, rain or other elements.
- If the appliance is used outdoors, it must be powered only by internal batteries and must not in any case be exposed to sunlight, rain or other elements: failure to comply with this warning may expose the user to risk of electric shock.
- This appliance is not a toy. This appliance should not be used by children or persons with reduced mental faculties, or who do not have the required experience or knowledge. The product can be used by persons with reduced physical or sensory capacity as long as under supervision or after they have received instructions for the safe use of the appliance, and have understood the hazards involved.
- Keep the appliance, its components and the mains adaptor with the power cord connecting to the motor unit out of children's reach. The power cord connecting the mains adaptor and the motor unit, and the tubes connecting the motor unit to the extraction kit, if left within children's reach, may cause strangulation. The appliance also contains small parts which, if ingested by a child, may cause suffocation.
- Please ensure that children do not play with the appliance, its components or the mains adaptor.
- Always place the appliance only on a flat, stable surface, away from cribs and cots.
- Before connecting the mains adaptor to the mains socket, ensure that the information on the power data plate and the mains adaptor are compatible with your electricity distribution network and the corresponding sock-

- et. The reference power data plate can be found on the mains adaptor.
- If the mains adaptor plug supplied with the appliance is not compatible with the mains socket, use only certified adaptors, if the current regulations permit them, or contact our free phone number (800-188-898) or qualified technical personnel.
 - Connect the mains adaptor to a socket that is easily accessible only by an adult.
 - When disconnecting the appliance from the mains electricity supply, first pull the mains adaptor plug out of the socket and only then disconnect the mains adaptor outlet cable from the motor unit. • Do not pull on the cord of the mains adaptor or the appliance itself if connected, to disconnect the mains adaptor from the socket.
 - Do not perform any cleaning or sanitising operations on the breast pump while it is operating, when it is powered by alkaline batteries (not supplied) or when it is connected to the mains through its mains adaptor. Cleaning and sanitising operations must be carried out in accordance with these instructions for use.
 - In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the milk has been collected, or to immediately store the collected milk in the most appropriate way.
 - Before each use clean and sanitise the silicone coverings (3), the pump body (4) and the valves (5). Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8), according to the instructions for the bottle.
 - Before each use, check that the membranes are not damaged: these must never have lacerations or cuts. If it is torn or cut, do not use the appliance.
 - The cleaning and sanitising to be performed by the user must not be carried out by children and/or people with reduced mental capabilities or lacking experience or the required knowledge.
 - Do not wet the appliance or its mains adaptor or put it into water or other liquids and never place it under running water. If any liquid has entered the appliance or the mains adaptor, contact our free phone number (800-188-898) or qualified technical personnel before further use.
 - If the appliance, the mains adaptor or its plug show any sign of damage, they must not be used. If the appliance or the mains adaptor show any sign of damage, contact our free phone number (800-188-898) or qualified technical personnel before further use.
 - Warning, never use a mains adaptor of a type different from that supplied by Artsana S.p.A. with the appliance (model GQ07-060100-DG, manufactured by: Dong Guan City GangQi Electronic Co. since doing so would make the appliance non-compliant with the applicable specific technical standards and EC directives, impairing its safety and integrity. If the mains adaptor is faulty, contact our free phone number (800-188-898).
 - Do not attempt to open or repair the appliance or the mains adaptor when faulty or damaged. In the event of faults or damage to the appliance in gen-

eral, to the motor unit or to the mains adaptor, contact our free phone number (800-188-898). Any repair/service carried out directly by the user may lead to severe risks to her or to any third party's safety as well as irreparably damaging the appliance and entails immediate termination of the warranty.

- If the packaging appears to be damaged, check the state of all the components and parts of the appliance. If the mains adaptor and the motor unit proved to be undamaged, wash all the components of the kit following the instructions given in the paragraph "CLEANING" as a precaution.
- Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children; they may be a source of danger.
- After removing any packaging materials, ensure that the appliance has not been damaged. If in any doubt, do not use the appliance and contact only technically qualified personnel, at the point of purchase or on our free phone number (800-188-898).
- Make sure that the appliance is placed on a level, stable surface when used in order to avoid spillage and prevent it from falling.
- Do not place the appliance on delicate surfaces or surfaces that may be sensitive to humidity and heat.
- Do not use the appliance while taking a bath or shower, or when in a damp place, or close to bath-tubs, sinks, washbasins, or in any other situation in which there are liquids that may come into contact with the appliance.
- Make sure that, when in use, the kit is not too far from the motor unit so that the latter does not fall.
- When using any electrical appliance, it is necessary to follow a few basic safety rules:
 - Never touch the appliance with wet or damp hands.
 - Keep the appliance and the mains adaptor away from heat sources.
 - Do not touch the appliance and the mains adaptor: if they accidentally fall into water, or accidentally come into contact with water or milk. Immediately disconnect the power supply using the master switch in your home and then disconnect the plug of the mains adaptor from the socket before touching the appliance. Do not use the appliance but contact qualified personnel or ring our free phone number (800-188-898).
 - If the appliance and/or the mains adaptor and/or the connection cord between the mains adaptor and the motor unit are faulty and/or not operating correctly, immediately switch off the appliance, disconnect the plug of the mains adaptor from the mains socket, do not attempt to tamper with or use the appliance but contact qualified personnel or ring our free phone number (800-188-898).
- The appliance must be cleaned and dried thoroughly at the end of every operating cycle.
- If the appliance is not going to be used for a prolonged period, it should be thoroughly cleaned and the batteries (if installed) removed before storing

it in a clean and dry place away from light and heat sources and according to the storage and transport instructions.

- If you decide not to use the appliance any more, remove the alkaline batteries, if present, and remove the output cable from the mains adaptor (after disconnecting the plug from the mains socket) to make it unusable, do not dispose of it as normal urban waste but as waste electrical and electronic equipment (WEEE) as required by EU Directive 2012/19/EU or by the laws in force in the countries where it is sold.
- Always remove alkaline batteries if you decide not to use the device for a long time. Failure to comply with this warning may cause leaking of acid with consequent damage and chemical contamination of the appliance.
- To avoid any risk of fire or electric shock, ensure that the cable connecting the mains adaptor and the appliance does not pass under any carpets or close to radiators, stoves and heaters.
- Operate the appliance at a temperature range between +5°C and +40°C. Use outside these temperature limits, and in particular above the upper limit, may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- The device was designed to only work with parts supplied in the original pack and with 150 ml capacity feeding bottles. It may not perform as declared if used with other parts or feeding bottles of greater capacity.
- The appliance must not be used continuously for more than 30 minutes. When you have finished using it, the appliance must be switched off for at least 30 minutes. If the appliance is used for longer than the permitted maximum operating time, even though it has a system to automatically switch it off after 40 minutes, this may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- product features might be subjected to change without notice.
- If you have any doubts on interpretation of the contents of this instruction booklet, contact your dealer or call our free phone number (800-188-898).

WARNINGS - REMOVING AND INSERTING BATTERIES

- Batteries must only be replaced by an adult.
- To replace the batteries: remove the cover, remove the run-down non-rechargeable batteries from the compartment, insert the new non-rechargeable batteries ensuring you match the polarity markings, replace the cover. Failing to match the polarity markings when replacing batteries may cause them to overheat, damaging the appliance and causing leakage of acid.
- Only use alkaline non-rechargeable batteries of the same or equivalent type recommended for the correct operation of this appliance.
- Never mix different types of batteries or new and used batteries.
- The poles/terminals of a battery must never be placed in short circuit.
- Always remove the alkaline batteries if you do not expect to use the appliance for a long time.
- Do not use rechargeable batteries for this appliance, as they may reduce or prevent its correct operation.
- Do not use lithium or rechargeable batteries. this appliance is not designed to operate with these types of battery. Using lithium or rechargeable batteries may create hazardous conditions.
- Always remove run-down non-rechargeable batteries from the device to prevent leaks of liquid that may damage it and/or anything near to it. Leakage of liquid may cause chemical contamination of parts designed to come into contact with food.
- If the batteries are leaking liquid, replace them immediately, making sure you clean the battery compartment and thoroughly wash your hands if they have come into contact with the liquid. Check that any leaked liquid has not come into

- contact in any way with the parts of the kit, in particular with those designed to come into contact with the milk.
- Do not throw run-down batteries into a fire or release them into the environment or dispose of them as normal urban waste. Batteries must be disposed of in compliance with the laws in force in the country where the appliance has been sold.
 - Remove the batteries from the appliance before you dispose of it.

INDEX OF SYMBOLS

Product catalogue reference code

WARNING!

Read the Instructions for use – Follow the Instructions for use

For indoor use only

ON/OFF (press-press)

IP 21

Appliance protected against penetration of solids (dimensions/diameter ≥ 12 mm) and liquids. Device protected against vertically dripping water.

Alternating current

Direct Current

Complies with European Directive 2012/19/EU

Mains adaptor polarity connection

Battery insertion polarity

The appliance complies with the essential requirements of applicable EC Directives

CLASS II Appliance

THE BREAST PUMP AND BREASTFEEDING

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking means not only satisfying a primary physiological need, but also, and more importantly, a precious moment in their relationship with their mothers, full of emotions and tactile, auditory, taste, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and well-being to both the mother and her child. Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives.

Chicco's electric breast pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can extract milk, store it, and give it to baby when required.

To guarantee the greatest flexibility and convenience, the electric breast pump operates both with AA size 1.5V alkaline batteries (not supplied) and with a special mains adaptor (model GQ07-060100-DG, manufactured by: Dong Guan City GangQi Electronic Co.

LIST OF COMPONENTS

- 1- Membrane
- 2- Membrane cover
- 3- Soft silicone cover
- 4- Pump body
- 5- Valve
- 6- NaturalFeeling feeding bottle
- 7- 0m teat with cap and ring
- 8- Airtight cap
- 9- Base
- 10- Motor unit
- 11- Display
- 12- Battery compartment
- 13- On/off button
- 14- Button for decreasing level of extraction/stimulation
- 15- Button for increasing level of extraction/stimulation
- 16- Stimulation/extraction button
- 17- Silicone tube for using the breast pump in single mode (both 00009199300000 and 00009199400000)
- 18- Silicone tube with splitter kit for using the breast pump in double mode (only 00009199400000)
- 19- Mains adaptor
- 20- Appliance power cable connector
- 21- Battery level indicator
- 22- Extraction/stimulation level indicator
- 23- Timer
- 24- Stimulation indicator
- 25- Extraction indicator
- 26- Active mode indicator (only on 00009199400000)
- 27- (only on 00009199400000)

* Components from (1) to (9) included in the Double Electric Breast Pump product 00009199400000 in pairs (2x) to enable use in double extraction / stimulation mode.

SET-UP

- Before using the Breast pump for the first time, please clean all the components* thoroughly as shown in the "-**CLEANING**" paragraph.
- Then assemble the breast pump as follows:
- Push the valve (5) down into the circular duct in the base

of the pump body (4). (Fig. 1)

Note: fit the valve preferably with the tab positioned inside the pump body.

- Push the outer edge to prevent damaging the valve and make sure that both closing edges of the assembled valve are joined together. If not, reposition the valve, making sure that the two edges are correctly aligned.

For better comfort, the silicone cover (3) may be assembled on the pump body (4). Insert the silicone cover well into the cone of the pump body, and seal it around the outer rim of the pump body (Fig.2). Press the pads of the cover to expel any air trapped inside.

- Fit the membrane cover (2) onto the silicone membrane (1) and insert the membrane and cover (pre-assembled) onto the pump body (4) (Fig.3)
- Screw the NaturalFeeling feeding bottle (6) onto the threaded base of the pump body (4). (fig.4)
- Use the practical base (9) to lay the assembled product on a flat surface and improve its stability.
- To use the breast pump with its mains adaptor (19), attach the connector (20) to the socket on the motor unit (10) and then insert the mains adaptor plug into a wall socket. (Fig. 5)
- To use the breast pump with alkaline batteries, open the cover of the battery compartment (12) and insert four AA size 1.5V batteries (not included) making sure you match the polarity markings. Close the compartment. The duration of the alkaline batteries may change based on the used brand and their charge status.
- **WARNING:** As a precaution, always remove the batteries from the motor body (10) if the breast pump is powered by the mains adaptor (19) and/or if it is not used for a long time.
- For the Single Electric Breast Pump 000091993000000 connect the silicone tube (17) with the membrane cover (2) (Fig. 6), whereas for Double Electric Breast Pump 000091994000000 connect the silicone tubes from the splitter kit (18) with the membrane covers.
- Insert the rigid connector of the silicone tube (17) for the Single Electric Breast Pump 000091993000000 or (18) for the Double Electric Breast Pump 000091994000000 in the appropriate location on the motor body (10) (Fig. 7).
- Take care that you do not lose the connector, if it is lost you may not be able to use the appliance.

*Excluding the motor body (10) and mains adaptor (19) and the silicone tubes (17) or (18).

USING THE BREAST PUMP

- Before each use, sanitise all the components* with a cold sanitizing method using sterilising liquids (following the manufacturer's instructions), or a hot one (steam) using specific devices, or by boiling them (5 minutes), and letting all the hot sanitized components fully cool down before use.
- Wash your hands and assemble the breast pump as shown in the **PREPARATION** paragraph.

- Cleanse your breast appropriately using water or one of the handy Chicco breast cleansing wipes.
- Sit down comfortably and relax.

Note: while using the breast pump, to ensure correct operation of the appliance, take care not to cover the hole on the left-hand side of the pump body (4) with your hand.

Note: when switched on, the breast pump always starts in stimulation mode to facilitate milk extraction, using the factory settings (level 5). The stimulation indicator (24) will appear on the display.

Note: To avoid all danger deriving from spilt liquid (milk), we recommend that you never fill the feeding bottle beyond the highest level indicated on its graduated scale.

Single Mode

REF 000091993000000 and REF 000091994000000

- Fit the cup over your breast ensuring that the nipple is centred properly. You can use the breast pump both with and without the soft silicone cover (3). We recommend trying both configurations before choosing the best and most comfortable one for you. Press the on/off button (13) on the motor body (10) to switch on the breast pump, the display (11) will light up.
- Only for 000091994000000 - check the icon (26) that indicates which operating mode is active on the breast pump.

For single mode the icon shows a single circle. If this is not the case, press the buttons "+" (15) and "-" (14) at the same time to change the breast pump operating mode from double (two circles) to single (one circle).

If you are using the breast pump powered by alkaline batteries, the indicator on the display will show the battery charge status (21). If it is powered by the mains adaptor, this indicator will not appear. The battery indicator shows 4 levels based on the battery charge status.

- After approximately 2 minutes of stimulation, the appliance automatically switches to the extraction mode using the factory settings (level 5). The extraction indicator (25) will appear on the display.
- You can stop the stimulation stage and switch to the expression stage by briefly pressing the stimulation/extraction button (16).
- If necessary, repeat stimulation by pressing the stimulation/extraction button (16) again. Press the stimulation/extraction button (16) to return to extraction.
- You can use the "+" (15) and "-" (14) buttons on the keypad to adjust extraction/stimulation based on the level of pull felt on your breast and/or based on the amount of milk expressed. We recommend adjusting the extraction/stimulation rate gradually until you reach the level best suited to you.
- If the level of extraction/stimulation chosen causes discomfort or pain, reduce it to a suitable level by pressing the "-" button (14).
- You can alternate milk extraction from both breasts.
- Press the on/off button (13) to switch the breast pump off.
- When the breast pump is switched back on it will be set to the stimulation/extraction levels stored from the previous session.

Double Mode - only for REF 000091994000000

- Press the on/off button (13) on the motor body (10) to switch on the breast pump, the display (11) will light up.
- Press the buttons "+" (15) and "-" (14) at the same time to change the breast pump operating mode from single (factory setting) to double. The icon (26) that indicates the breast pump operating mode must show two circles for double mode.
- Fit the cup over your breast ensuring that the nipple is

centred properly.

- Fit the second cup over your second breast, ensuring that the nipple is centred properly.
 - After approximately 2 minutes of stimulation, the appliance automatically switches to extraction mode using the factory settings (level 5). The extraction indicator (25) will appear on the display.
 - You can stop the stimulation stage and switch to the expression stage by briefly pressing the stimulation/extraction button (16).
 - If necessary, repeat stimulation by pressing the stimulation/extraction button (16) again. Press the stimulation/extraction button (16) to return to extraction.
 - You can use the "+" (15) and "-" (14) buttons on the keypad to adjust extraction/stimulation based on the level of pull felt on your breast and/or based on the amount of milk expressed. We recommend adjusting the extraction/stimulation rate gradually until you reach the level best suited to you.
 - If the level of extraction/stimulation chosen causes discomfort or pain, reduce it to a suitable level by pressing the "-" button (14).
 - Press the on/off button (13) to switch the breast pump off.
 - When the breast pump is switched back on it will be set to the stimulation/extraction levels stored from the previous session.
- WARNING:** immediately after each single use, carefully clean all the components* as shown in the "**CLEANING**" paragraph.
- To feed the milk immediately to your baby, unscrew the feeding bottle from the pump body (4) to remove it. Screw the ring with the teat (7) (included in the pack) onto the feeding bottle. This way, you will obtain a handy, ready-to-use baby feeding bottle.
 - If, on the contrary, you close the bottle with the airtight cap (8) (included in the package) you can store the expressed milk in the fridge for up to 48 hours at 4°C or for up to 3 months at -18°C in the freezer.
 - **WARNING:** When the breast pump is not used, always disconnect the mains adaptor from the mains.
- *Excluding the motor body (10) and mains adaptor (19) and the silicone tubes (17) or (18).

CLEANING

- Before carrying out any cleaning operation, disconnect, if connected, the mains adaptor from the mains and the power cord connector from the socket (20) on the motor body.
- Separate all the components from the breast pump.
Note: remove the valve (5), located on the circular conduit in the pump body (4) by gently pulling the tab on the side of the valve.
- **WARNING:** Make sure that the valve is always removed in a gentle way; do not use any tool for cleaning it because it may be damaged, making the breast pump impossible to use.
Note: to remove the silicone tube, first detach the membrane (1) and the membrane cover from the pump body, to avoid any spillage. Then, holding the end inserted into the membrane cover, gently pull it off.
- Carefully wash all the components* using hot water and washing up liquid, thoroughly rinse and dry carefully us-

ing a soft and clean cloth.

- Do not use abrasive products to clean the breast pump and its components.
- Clean the motor body and the mains adaptor with a soft, clean and dry cloth. Warning! Do not wet the appliance or put it into water or other liquids.
- The tube may be rinsed and suitably dried.
- All the components, with the exception of the motor body (10), the mains adaptor (19), the valve (5), the membrane (1), the silicone cover (3) and the tube (17) may also be washed in a dishwasher using a delicate programme. This washing method, which is more aggressive than hand washing, may discolour or dull and degrade the material over time.
- Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8), according to the instructions included with the bottle.
- After washing, always store the product in a clean, dry place.

*Excluding the motor body (10), mains adaptor (19) and the tube (17).

ACCESSORIES:

- NaturalFeeling feeding bottle 150 ml **REF** 00080711000xxx / **REF** 00080811000xxx.
 - NaturalFeeling milk container **REF** 00002257200000
 - Where XXX=variable identification code identifying the country of sale.
 - Two nursing pads, which are only to be used for absorbing excess breast milk, are supplied free with the appliance (**REF** 20061770000).
- No accessories other than those recommended must be used.

Trouble	Cause	Solution
The breast pump does not pump or pumps very little.	Valve not assembled or not properly assembled.	Assemble the valve correctly.
	Cup and motor body not assembly correctly.	Check that you have performed the procedure described in the "PREPARATION" paragraph correctly. Make sure that the silicone tube is firmly connected to both the motor unit and the membrane cover. Check that the tube is not damaged. Do not twist the tube during milk expression.
	The cup does not fit well on the breast	Fit the pump body over your breast, taking care that your nipple is properly centred. Try with/without the soft silicone cover.
	Suction level not correctly adjusted.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "+" or "-" buttons.
Milk expression is painful.	Run-down and/or insufficiently charged batteries.	Replace the batteries.
	Suction level too high.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "-" button.
	Failure of mains adaptor and/or appliance.	Check that mains power is present. Check the position of the batteries in the battery compartment.
The breast pump does not switch on	Incorrect connections.	Check all the connections.
	Run-down batteries.	Replace the batteries.

TECHNICAL SPECIFICATIONS

- REF 00009199300000 ELECTRIC BREAST PUMP Model N0041 and REF 00009199400000 DOUBLE ELECTRIC BREAST PUMP N0000
- Maximum vacuum level in extraction mode: -0.35 ± 0.03 bar**
(** measured using a sample cylinder - confidence level 95%).
- Type of appliance: intermittent operation; 30 minutes ON - 30 minutes OFF
- Motor body weight: 325 g excluding batteries
- Dimensions: 98x70xh105 mm
- Made in China

BATTERY POWERED MODE:

6V DC – 4x1.5V AA size (LR06) batteries

DECLARATION OF EC COMPLIANCE

The Chicco-branded appliances Electric Breast Pump 000091993000000 Mod. N0041 Double Electric Breast Pump 000091994000000 Mod. N0000 comply with all the essential requirements and other provisions indicated in the EU Directives 2014/30/EU (electromagnetic compatibility), 2014/35/EU (Low Voltage), and 2011/65/EU (RoHS). A copy of the EC declaration can be requested from the manufacturer: Artsana S.p.A via Saldarini Catelli, 1 - 22070 Grandate (Como) Italy.

The products Electric Breast Pump REF 000091993000000 Mod. N0041 and Double Electric Breast Pump REF 000091994000000 Mod. N0000 comply with EU Directive 2012/19/EU.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for

taking the appliance to a special waste disposal site at the end of its life. If the unwanted appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. . For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damage caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws

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Artsana S.p.A – via Saldarini Catelli, 1 - 22070 Grandate (Como) – Italy