

## PAP+ Teeth Whitening Led Kit



FOLLOW US @hellococoeu

### PAP+ Formula



Scientifically proven ingredient for teeth whitening, which is a safe alternative to peroxide. It has no negative side effects during the oxidation of tooth stains, which allows effective whitening without pain and sensitivity of the teeth.




Hydroxyapatite is the main component of teeth. The PAP + formula contains a synthetic version that replaces minerals back into damaged parts of the enamel, which over time will help strengthen and protect the layers of teeth.


### Package Contents




1x Innovative mouthpiece with 24 LEDs powered by a smartphone.

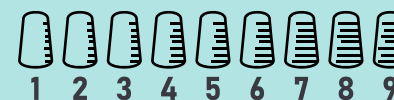

1x white tooth scale for measuring whitening effects.

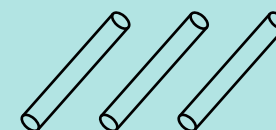

3x Whitening pen with whitening gel. Safe for enamel and gums.

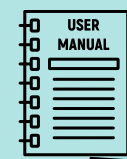

1x manual.

### How to use



1 Twist the bottom of the pen until you can see how the gel is released.



2 Apply a thin layer of gel to the front of the cleaned teeth, avoiding the gums.



3 Insert the LED mouthpiece into your mouth, connect it to your smartphone, turn it on and let it work. The device switches itself off automatically after 16 minutes.



4 Use once a day for 7 days, then repeat the treatment after 6 weeks.



5 For best results, do not drink, eat or smoke for 1 hour after bleaching.

### COMPOSITION

Aqua, Glycerin, Propylene Glycol, Phthalimidoperoxycaproic Acid (PAP), Hydroxyapatite (HAP), Carbomer, Cocos Nucifera Oil, Menthol, Xylitol, Aloe Barbadensis Leaf Juice, Punica Granatum Seed Oil, Cranberry Seed Oil, Mica, Sodium Hydroxide, Aroma/ Peppermint Oil

### TERMS OF USE

Use in accordance with the enclosed instructions. Keep out of reach of children. Do not use the gel on the lip. The product should only be used for teeth whitening. The whitening agent will not whiten any existing dental fillings, such as crown-works, implants, bridgeworks. Do not use if you are allergic to the above ingredients. If the gel is on your face or clothes, you should wash them immediately. Put the cap on top of the pen after you have finished whitening your teeth. after you have finished whitening your teeth. One whitening lasts 16 minutes. We recommend using the whitening treatment for 7 days. Subsequently, we recommend repeating the whitening after 6 weeks. Follow the instructions provided.

### NOTIFICATIONS

1. Not suitable for ceramic teeth and prostheses.
2. Not suitable for tetracycline teeth.
3. Not suitable for children under 12, pregnant and lactating women.
4. Not suitable for removing enamel and damaged teeth.
5. Not suitable for discoloration of teeth caused by lesion.

### STORAGE

Store the product in closed original packaging, in a dry place, away from heat and direct sunlight.