TESLA
NASAL ASPIRATOR NA300
USER MANUAL

EN

DE

CZ

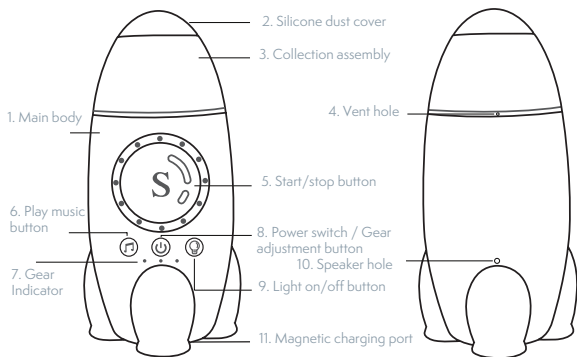
SK

PL


HU

SL

DESCRIPTION


INCLUDED ACCESSORIES

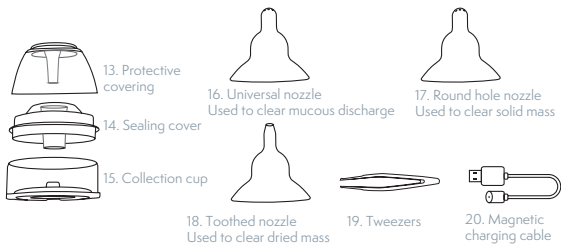
ASSEMBLY INSTRUCTIONS

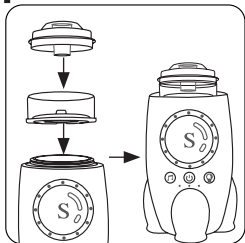1


1. First insert the base of the collection cup into the main body and tighten it, and then tighten the sealing cover.

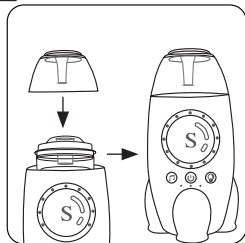2

2. Snap the protective cover of the collection cup into the main body and turn it clockwise to tighten it.

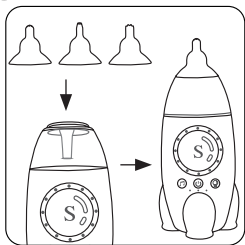3


3. Select the appropriate nozzle and install it on the collection cup.


OPERATION INSTRUCTIONS

1. Turn on/off: Long press the „8“ to turn the device on. The Gear Indicator lights up. Long press the „8“ again to turn the device off.
2. Gear Position Adjustment: After turning the device on, short press the „8“, gears 1 - 3 are cycled in sequence.
3. Music Playback: Long press the „6“ to turn music playback on, short press „6“ to switch to the next music track, long press „6“ again to turn the music off.
4. Start/Stop Button: Short press „5“ to turn the suction function on to clean the nasal cavity, and short press „5“ again to turn it off.
5. Light Button: Short press „9“ to turn the colorful light on, and press „9“ again to turn the light off.
6. Device Charging: Connect the Magnetic charging cable to the Magnetic charging port. When charging the device, the first to third lights flash in an incremental cycle. When fully charged, the three lights are always on. The device cannot be used while charging.
7. Gear Position Indicator: When the device is turned on, the default setting is the first gear and the first light is on. The three lights correspond to three gears. When the device is in a state of low power, the 3 gear lights flash three times. The device should then be charged.

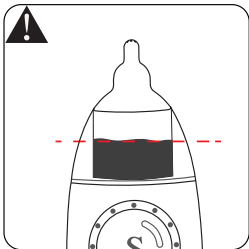CAUTION

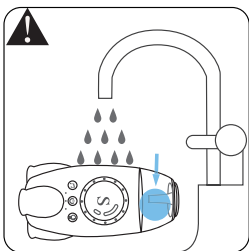
1. When cleaning the nasal cavity and the fluid accumulation in the collection cup exceeds half of the volume, remove the collection cup and empty it before continuing to use it, otherwise the air pump might get blocked, and the device may be damaged.


2. When cleaning the device, pay attention to tightening the collection cup before flushing to prevent water from entering the air pump.

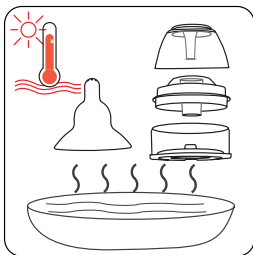CAUTION

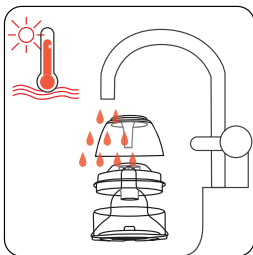Tip: Use boiling water or steam to disinfect the suction nozzle and collection cup before use. Note that a microwave oven cannot be used for disinfection.

Tip: After use, please clean the suction nozzle and collection cup with hot water to prevent build up of bacteria.

SAFETY INSTRUCTIONS

This product is only used to clean the nasal cavity.

Prevent solid and hard objects above 3 mm from entering the nozzle.

Take care not to drop the product as this might damage it.

Use only the accessories provided by the manufacturer to avoid damage to the device due to inappropriate accessories.

This product is not a toy, please keep it in a safe place.

PRECAUTIONS BEFORE USE

It is recommended to sterilize the suction nozzle and collection cup before use, such as soaking in boiling water at 100 °C for a few minutes and then install them on the main body of the device for use.

Do not use high-temperature machines such as microwave ovens for disinfection.

Do not immerse the whole product in liquid.

Please check whether the seal is damaged and the accessories are complete.

Wash your hands before use.

PRODUCT FEATURES

This product is an efficient nasal cavity cleaning device. It is soft, efficient and easy to clean. It is specifically designed for babies and children.

The main body of the device is made of soft silicone, and is waterproof and drop resistant.

The device has three levels of suction for maximum comfort level and different applications.

The aspirator is equipped uses light and music to distract baby during this, usually unpleasant activity.

The collection cup and suction nozzle made of food-grade materials are safe and hygienic, simple to operate and easy to clean.

PRODUCT STORAGE

Use cleaner or detergent to clean this product.

The suction nozzle and collection cup can be sterilized by boiling water or steam, but not by a microwave oven.

The whole device is waterproof if the collection cup is not removed. After use, make sure the collection cup is tightened and then you can rinse the whole device with water. The collection cup can be tightened and rinsed with water. Do not disassemble the collection cup. In case of flushing, prevent water from entering the air pump from the suction point which could cause malfunction.

Avoid storing the product in direct sunlight or humid environment.

Avoid using corrosive chemicals to clean this product.

SAFETY PRECAUTIONS

When using, make sure that your baby's mouth is open, to ensure air circulation in the nasal cavity, and to avoid discomfort caused by negative pressure.

When the nasal aspirator is working, do not block the other nostril, so as to avoid negative pressure on the head and cause discomfort.

Please clean this product before each use.

Disinfect the suction nozzle and transparent collection cup after use to prevent cross-infection.

Do not immerse the entire device in liquid.

When the capacity of the collection cup reaches half during use, it should be cleaned before continuing to use it to prevent the liquid from being sucked into the air pump and damaging the device.

INFORMATION ABOUT DISPOSAL AND RECYCLING

This product is marked with the symbol for separate collection. The product must be disposed of in accordance with the regulations for disposal of electrical and electronic equipment (Directive 2012/19/EU on waste electrical and electronic equipment). Disposal together with regular municipal waste is prohibited. Dispose of all electrical and electronic products in accordance with all local and European regulations at the designated collection points which hold the appropriate authorisation and certification in line with the local and legislative regulations. Correct disposal and recycling help to minimise impacts on the environment and human health. Further information regarding disposal can be obtained from the vendor, authorised service centre or local authorities.

EU DECLARATION OF CONFORMITY

Hereby, Tesla Global Limited declares that the radio equipment type TSL-PC-NA300 is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: tsl.sh/doc

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