

Scholl Velvet Smooth™ with diamond crystals* – electronic foot file

The extra-coarse roller heads of the Scholl Velvet Smooth™ for very hard skin now come with diamond crystals. The Scholl Velvet Smooth™ removes hard skin in a few minutes. It is very easy to use. Use the file in gentle, circular movements over the hardened skin and see the old skin disappear. The quantity of diamond crystals is 1% (i.e. about 5-15mg) of silicon carbide, which is used for the heads.

It has an ergonomically designed soft-touch handle.

A specially designed Micralumina™ replaceable head.

Gentle and effective rotating movement.

Battery-powered.

Ready to use.

Package contents

1 electric foot file

1 roller head – extra-coarse

4 AA batteries

1 cover

Preparation and use

Instructions for use: Keep the packaging for possible future use.

Initial use:

Unscrew the lower part of the unit counterclockwise.

Remove the bottom and remove the red insulating liner.

Screw the lower part of the unit in a clockwise direction.

For use only on hard skin on feet.

Use only on dry (not wet) skin.

Do not use on other parts of the body.

Use only for the intended purpose.

Do not use in the same place for more than 3-4 seconds per application.

Stop using the device immediately if you feel irritation or discomfort.

Do not use the product on cracked, sore or irritated skin.

Do not use the device in water.

When using, keep long hair and clothing away from the device.

Not suitable for diabetics and people with poor circulation.

We recommend only using spare Scholl Velvet Smooth Micralumina™ with the device.

Normal use:

Before use, remove the protective cover from the roller head.

The roller head must be securely fixed to the device. Do

not use if the roller head is not attached to the unit.

Switch on the unit – turn the silver ring to the left and gently sweep the roller over hard skin. Hard skin will gradually peel off. Only use on dry skin. Do not put too much pressure on the unit or it will switch itself off.

Turn off the unit and check whether you have achieved the desired result. If not, repeat the procedure.

Attention: Do not use in the same place for more than 3-4 seconds per procedure. If the skin is injured or hurts, stop the procedure.

Turn off the unit – turn the silver ring to the right.

Wash your feet and remove the remnants of dry skin.

Dry your feet thoroughly.

Apply a moisturiser and rub it into your smooth feet.

Cleaning after use:

The Velvet Smooth Express Pedi is not waterproof.
When cleaning, remove the roller head.

Turn off the unit.

Remove the exchangeable head – press the appropriate button on the side of the unit and pull gently to remove the head.

Wash the head under running water.

Clean the body of the unit with a damp cloth.

Dry the head and the body of the unit with a dry cloth.

Reattach the head and attach the protective cover.

Replacing the roller head

The head should be replaced if it loses effectiveness. Only use Scholl Velvet Smooth™ Micralumina™ roller heads with this device.

Make sure that the unit is switched off.

Remove the exchangeable head – press the appropriate button on the side of the unit and pull gently to remove the head.

Fit the new head to the unit by pushing in the buttons on both sides of the head and then inserting it in the unit.

Before use, check that the roller head is properly attached to the unit.

Replacing the batteries:

Unscrew the base of the unit counterclockwise.

Separate the base from the unit and remove the batteries.

Insert four new AA batteries, as shown on the unit.

Put the unit base in its original place and screw it clockwise until it stops.